

## Endodontic Ultrasonic Tip with Micro-projection

# **BL TIP**



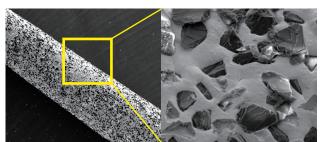

Diamond coated Tip

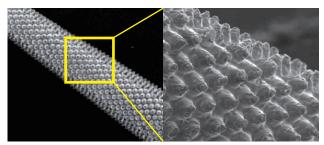

BL TIP with Micro-projections

## B&L non-surgical ultrasonic tip

## Longevity and safety with micro-projections

BL Tips have sharp abrasive micro-projections integrated onto the main body of the tip.

Unlike conventional diamond coated endodontic tips, whose coating wears off during surgery, BL Tips are more resilent and efficient in cutting

## Manageable elimination with a variety of shapes and sizes

The ends of the BL Tips come in a variety of shapes and sizes, making it convenient for the doctor to effectively control the amount of dentin

100μm Micro-projections: ideal for removing larger quantities of dentin. 50μm Micro-projections: ideal for more precision-based refinement.

#### Versatility

BL Tips are useful not only in accessing the root canal, but also in enlarging the coronal aspect of the root canal because the end of the tip works in every direction.

#### Item Code

Starter Kit: BLSKA(BL1,3,5 Satelec), BLSKE(BL1,3,5 EMS)


BL-1 : BL1A, BL1E BL-3 : BL3A, BL3E BL-5 : BL5A, BL5E

### Appropriate power level




E-Type not available for sale in the US.



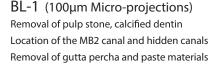



BL-3 (100µm Micro-projections)
Use at a higher power, higher speed
Removal of secondary dentin and pulp stones
Resistance to breakage

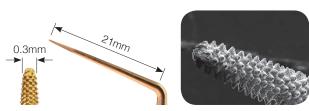

## BL-5 (50µm Micro-projections)

Smaller & longer than BL-1

Sharper and longer shape allows efficient preparation of the coronal half of the canal and creates a space for the insertion of a file.



