# Dental suction & ASPIRATION SYSTEM CLEANER/SANITIZER



DAILY SANITISER/

**CLEANER** 

ION LINE CLED

PSPIRATION,

**DenJet®MULTI** is a powerful effective foam-free clinical detergent/sanitiser. Formulated to clean & sanitise dental aspiration/suction lines & separation systems. **DenJet®MULTI is Non-hazardous**.

Use once or twice per day as part of the practice management. Instructions for the maintenance & sanitation of aspiration lines for assisting the removal of bioburden.

A highly effective formula is used that incorporates advanced chemicals for rapid cleaning and sanitisation of the aspiration lines.

Formulated into **DenJet®MULTI** is a highly effective non-chelating agent that will prevent the build-up of plaque, scale and biofilm throughout the entire lumen of the tubes. This will assist the removal of biosoils that are resistant to other detergents.

**DenJet®MULTI** incorporates second generation chemicals that are multi purpose and only daily rinsing is required. This reduces time and costs.

Mildly alkaline auxiliary agents enhance the activity of the sanitising agents and will aid the solubilisation of proteins and soils.

**DenJet®MULTI** is compatible with metal, silicone, plastic and rubber components.





PRODUCT CODES: DenJet®MULTI 5 Litre Container - DJM5

Specially formulated to be suitable for use with dental aspiration/suction pumps and lines.



Marketed by: **Majac Medical Products Pty. Ltd.** Tel: 1300 138 578 • Int: +61 7 3265 6355 Email: sales@majacmedical.com.au **www.majacmedical.com.au** 



FS 540017

Also available from:



Rely on Us<sup>\*</sup>





## Preventative Maintenance of **DENTAL ASPIRATION SYSTEMS**

### Routine cleaning is an essential part in keeping the aspiration lines in working condition and a guarantee of effective trouble free operation.

Sanitation of the waste system is essential to maintaining a hygienic aspiration system that will protect patients and staff from hazardous microorganisms.

#### **IMPORTANT:**

Strong domestic hypochlorite solutions should never be used to clean aspiration lines because the chlorine can damage rubber and metal components. These types of cleaners are usually caustic and therefore can damage some materials, they are also high foaming.

Phenolic compounds in some industrial disinfectants will damage plastics and rubber.



#### DenJet<sup>®</sup>MULTI is a mild alkaline cleaner/sanitiser.

**DenJet®MULTI** will remove protein and bio-soils from the suction tubes while eliminating contaminates before they form a biofilm that will narrow the lumen of the lines and fatigue the pump.

### When using DenJet<sup>®</sup>MULTI a weekly descale with another acidic cleaner is not necessary.

Because the MULTI formula incorporates a non-chelating agent that is effective in preventing any residual plaque from forming that may be resistant to other cleaners.





Marketed by: **Majac Medical Products Pty. Ltd.** Tel: 1300 138 578 • Int: +61 7 3265 6355 Email: sales@majacmedical.com.au www.majacmedical.com.au



Also available from: HENRY SCHEIN® DENTAL

20