

# FIT CHECKER™ ADVANCED WHITE Tube Pack

1x 42ml base

1x 42ml catalyst

1x 3g retarder

1x mixing pad



# FIT CHECKER™ ADVANCED WHITE Cartridge Pack

2x 48ml cartridge

3x mixing tip IIS

3x mixing tip IISS



# FIT CHECKER™ ADVANCED BLUE Cartridge Pack

2x 48ml cartridge 3x mixing tip IIS 3x mixing tip IISS


### Technique tip

FIT CHECKER™ ADVANCED contains A-silicone components. Contact with latex gloves, unpolymerised resin and eugenol-containing materials may hinder its setting reaction. Before cementing restorations, clean with alcohol to remove any residual silicone oil as this could interfere with the cementation process.



,'GC,'

GC Australasia Dental Pty Ltd 1753 Botany Road, Banksmeadow NSW 2019, Australia T: +61 2 9316 4499 F: +61 2 9316 4196 www.gcasia.info

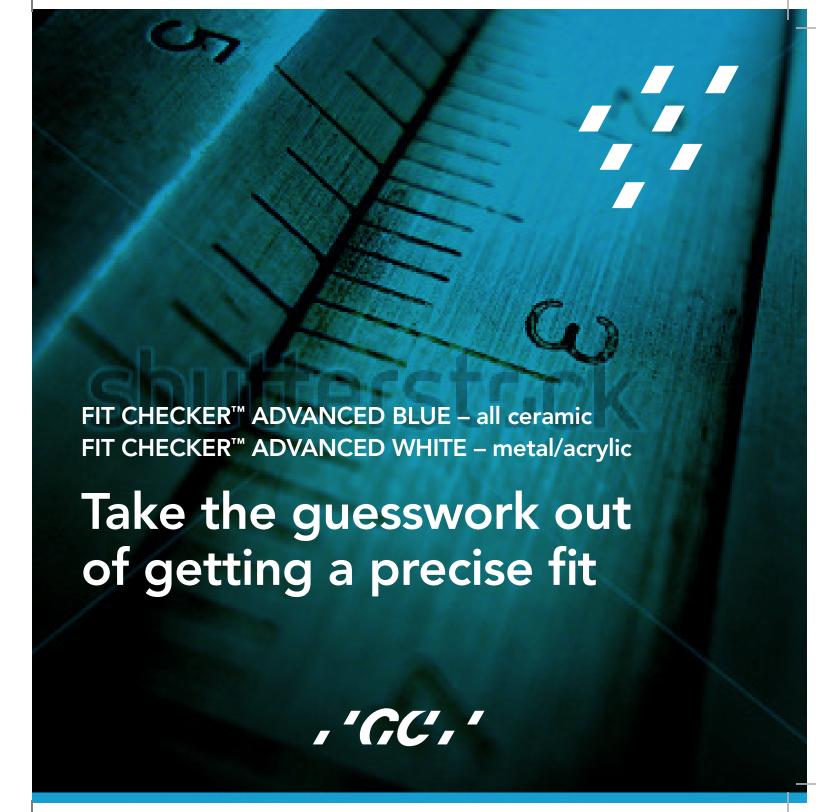

## FIT CHECKER™ ADVANCED WHITE

## The ideal colour for acrylic and metal

- FIT CHECKER™ ADVANCED WHITE offers ideal consistency and high wettability for easy flow onto dentures and oral tissues, now even easier to use in automix.
- Perfect for all kinds of prosthetic work, such as dentures and metal-based crowns and bridges.
- FIT CHECKER™ ADVANCED WHITE is not affected by saliva.







Check and adjust Ideal for metal crowns and bridges



## FIT CHECKER™ ADVANCED BLUE


## The ideal colour for all ceramic work – adjust the right spots and take the guesswork out of getting a precise fit

- Perfect for checking the fit for all kinds of ceramic work, especially resin, ceramic and zirconia.
- Can be used for bite registration and evaluation of occlusal contacts, as well as checking the fit of splints. Finally, it's possible to make certain of a perfect fit for indirect ceramic restorations, avoiding marginal gaps and

major occlusal adjustments after cementation.





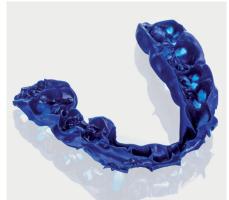

Check and adjust

Bite registration

#### FIT CHECKER™ **ADVANCED**

A Vinyl PolyEther Silicone for simple and precise checking and adjustment of all prosthetic work.

#### Snap set

Sufficient working time and rapid setting time (w/t 1 minute, s/t 1 minute in mouth).

#### Clear and precise visualisation

A perfect colour gradation for immediate visualisation of occlusal contacts and poorly fitting areas.

### No mess during adjustments

The set material does not get caught by the bur while relieving pressure spots.

## Easy to peel off

The Vinyl PolyEther Silicone peels off easily for a clean and fast procedure.

