EXABITE III

Vinyl Polysiloxane Bite Registration Crème

Contains:

Two (48ml) cartridges and 6 mixing tips (3 small, 3 large)



Physical Properties	EXABITE III
Working time 23°C (sec)	35
Working time 37°C (sec)	15
Minimum time in mouth (sec)	30
Strain in Compression (%)	0.7
Recovery from deformation (%)	99.2
Linear dimensional change (%)	-0.3
Shore A Hardness	94

EXABITE III

A perfect bite every time





GC Australasia Dental Pty Ltd 1753 Botany Road, Banksmeadow NSW 2019, Australia T: +61 2 9316 4499 F: +61 2 9316 4196 www.gcasia.info



Introducing EXABITE III – faster, harder and more accurate for optimal bite registration.

Getting the bite right is so important

Capturing an occlusal relationship can present difficulties and small errors in this crucial step may require major intra oral adjustments during seating. These intra oral adjustments are time consuming and can compromise the strength of ceramics. The recent strong growth of zirconia-based restorations, that are much more difficult to adjust, makes occlusal relationship even more important.

Snap set

EXABITE III is fast – it sets in just 30 seconds, with 15 seconds working time in the mouth.



Ideal consistency for closed bite technique

Multiple applications

EXABITE III is suitable for all bite registration techniques, whether you are taking an open bite registration for standard crown and bridge indexes or closed bites for edentulous or partial patients. The consistency of EXABITE III is ideal for capturing bites for splints and it adapts well over wax bite blocks when taking denture registrations. EXABITE III is also suitable for CAD/CAM fabrications, as it is easy to powder and obtain a flawless optical image.

EXABITE III takes the stress out of capturing a bite

– making it easier for the clinician, technician and patient.



Comfortable for the patient with 30 seconds setting time in the mouth

Easy to read

EXABITE III makes it much easier to read the occlusal detail with its contrasting green colour. Light reflecting off the surface results in greater detail being seen.



Easy for patients

Featuring a creamy consistency, EXABITE III offers no resistance during closure so patients can occlude in centric without effort. This means the potential for deviation of the mandible is virtually eliminated and an accurate registration can be completed. EXABITE III has no taste, no heat, no smell or odour making it very comfortable for patients.


Accurate for dentists and technicians


A Shore A hardness of 94 guarantees there will be no distortion or movement. This enables fast and easy trimming with no displacement.





Mouth Removal Time**

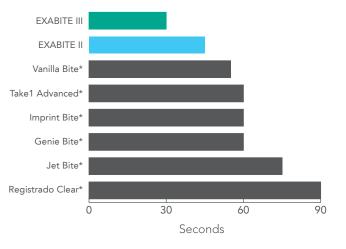

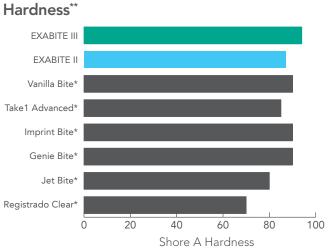

^{*} not a trademark of GC Corporation

^{**} Information sourced from manufacturers' directions for use