



Alcohol free, dual action cleaner and disinfectant for alcohol sensitive surfaces

Effective against enveloped viruses
1 minute contact time
(EN16615, 4-field test)

mikroqid® alcohol free liquid and Jumbo wipes

Our Plus

- alcohol free, making it ideal for use on all surfaces including dental chairs and alcohol sensitive materials
- added surfactants for boosted cleaning performance
- cleaning and disinfection, two complete actions in one product
- effective against bacteria, yeast and enveloped viruses
- rapid 1 minute contact time, making it ideal for use between patients
- ready to use solution

Application areas

For the disinfection of medical devices and other environmental surfaces. Especially suitable for surfaces sensitive to alcohol (e.g. leather, PVC and acrylic glass).

Instructions for use

mikroqid® alcohol free liquid is a ready-to-use solution. Apply mikroqid® alcohol free liquid undiluted to objects and surfaces and allow to take effect. Ensure that surfaces are thoroughly wetted. If necessary, remove excess product with a dye-free disposable wipe. You can apply the ready-to-use solution either as a spray or as a foam disinfectant.

Wipes

Wipe surfaces thoroughly with the impregnated wipe, ensure complete wetting and allow to take effect. Make sure that all visible soiling is removed before disinfection and the lid is tightly closed after use. If necessary, use several wipes for large areas.

Refilling

First disinfect hands and wear new protective gloves. During the refilling process, please ensure that the gloves are not contaminated by other activities. On every refill, subject the lid closure to disinfectant cleaning with an alcohol based surface disinfectant.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
MRSA	ready-to-use	1 min.
virucidal against enveloped viruses	ready-to-use	1 min.
Bovine viral diarrhea virus	ready-to-use	1 min.
yeastocidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
Vaccinia virus	ready-to-use	1 min.
Rotavirus	ready-to-use	1 min.



mikrozid[®] alcohol free liquid

Product data

Composition: 100 g solution contains the following active substances: 0.26 g Quaternary ammonium compounds, benzyl-C12-C18-alkyldimethyl, chlorides, 0.26 g Didecyldimethylammonium chloride, 0.26 g Quaternary ammonium compounds, C12-14-alkyl [(ethylphenyl) methyl] dimethylammonium, chlorides.
One dispenser or refill contains 200 ready-to-use Jumbo wipes.

Special advice

Use biocides safely. Always read the label and product information before use.

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit www.schuelke.com

Wearing gloves (e.g. of nitrile rubber) is recommended. Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container. Only recycle completely emptied packaging. Leftover product can be disposed of in accordance with EWC 070601. Do not use near ignition sources. Do not exceed usage of more than 50ml per m². Avoid contact with the eyes. Store out of the reach of children. Not for final disinfection of semicritical and critical medical devices.

Information for order

Item	Delivery form	Item no.
Mikrozid Alcohol Free Wipes Jumbo	TUB 200 Wipes	SK-109184
Mikrozid Alcohol Free Wipes Jumbo	REFILL 200 Wipes	SK-109190
Mikrozid Alcohol Free liquid 5L	5L bottle	SK-109193
Mikrozid Alcohol Free Liquid 1L	1L bottle	SK-109186

Related Products

- mikrozid[®] (alcohol based) wipes & liquid
- mikrozid[®] sensitive wipes 'premium'



Environmental information

schulke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail mail.uk@schuelke.com

☎ 0800 808 855
✉ info@henryschein.co.nz
🌐 www.henryschein.co.nz

 HENRY SCHEIN[®]
DENTAL

 Rely on Us[™]



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schulke UK
Schülke & Mayr UK Ltd
Cygnet House, Meadowhall
Sheffield, S9 1AT
Phone +44-114-254 35 00
Fax +44-114-254 35 01
mail.uk@schuelke.com

schulke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com