



## Lubrina

Fast, economical handpiece maintenance

 J.MORITA MFG.CORP.

### Lubrina features at a glance

- Servicing 4 handpieces in 100 seconds
- Built-in chuck maintenance system
- Purges and lubricates all types of handpieces
- Unique dual lubrication system
- Step air blow function
- Accommodates all attachments

Thinking ahead. Focused on life.

  
**MORITA**

# Time- and cost-saving efficiency

The Lubrina handpiece maintenance unit is a real time- and cost-saver, for any dental practice, servicing up to four handpieces in as little as 100 seconds. Purging and lubricating all types of handpieces, Lubrina is also extremely economical in spray deployment. What is more, Lubrina is the first lubrication unit with a built-in chuck maintenance system.

## Servicing 4 handpieces in 100 seconds

Requiring approx. 25 sec. per handpiece, Lubrina can service up to four handpieces in 100 sec. - including lubrication, purging and chuck maintenance.

Compared with manual lubrication, Lubrina requires only 1/3 of the lubricant.

## Built-in chuck maintenance system

Lubrina is the first handpiece maintenance unit with a built-in chuck maintenance system (only for J. MORITA'S instruments).

## Maintenance for various instruments

Whether contra-angle and straight handpieces, air turbines and air scalers, Lubrina offers you highly efficient and user-friendly handpiece maintenance - fully automatic. By replacing the joints, Lubrina can make the maintenance of the contra-angle of Tri Auto ZX and DentaPort ZX, the air turbine handpiece (ISO 9168 joint) and contra-angle and straight handpieces from other manufacturers (ISO 3964 joint).

## Unique dual lubrication system

Featuring a one-of-a-kind dual lubrication system, Lubrina enables lubrication of two instruments with one spray - or, if so desired, the use of two different sprays.

Lubrina requires no special spray. It can be operated with any authorized brand by using an optional spray base adapter.

## Air suction system

The air suction system eliminates lubricant mist, so there is less smell by oil. Excess lubricant goes to the lubricant absorber through a return circuit to keep the operating area clean.

## Lubricant indicator

To reduce maintenance failure, Lubrina is equipped with a lubrication sensor, signaling when spray is empty with an illuminated light.

## Selectable lubrication time option

Lubrina allows you to select between two lubrication time modes: short mode and long mode.



Lubrina services up to 4 handpieces simultaneously and various instruments including instruments from other manufacturers.



The Unique dual lubrication system

## Lubrina Specifications

|                          |                          |
|--------------------------|--------------------------|
| Name                     | Lubrina                  |
| Model                    | HIM-1                    |
| Rating                   | AC 100 - 240 V; 50/60 Hz |
| Power consumption        | 18 VA                    |
| Operating air pressure   | 0.4 - 0.6 MPa            |
| Recommended air pressure | 0.5 MPa                  |
| Air flow                 | 40 - 60 NL/min.          |
| Weight                   | approx. 10 kg            |
| Dimensions (mm)          | W 300 x H 370 x D 300    |

Distributed by

**J. MORITA EUROPE GMBH**

Justus-von-Liebig-Strasse 27A, D-63128 Dietzenbach, Germany

Tel: +49-6074-836-0, Fax: +49-6074-836-299, <http://www.jmorita-europe.com>

Manufactured by

**J. MORITA Mfg. Corp.**

680 Higashihama Minami-cho, Fushimi-ku, Kyoto, 612-8533 Japan

Tel: +81-75-611-2141, Fax: +81-75-622-4595, <http://www.jmorita-mfg.com>

