

Tri Auto ZX2

Lubrication


- * Only the contra angle needs to be lubricated.
- * We recommend using the Lubrina dental handpiece maintenance unit for lubricating the contra angle.



- (1) Cover the contra angle with a piece of gauze or other suitable cloth.




- (2) Screw the nozzle onto the spray can. Then insert it into the connection end of the contra angle, and spray for 2 seconds. Use gauze etc. to wipe excess spray off the outside of the contra angle.

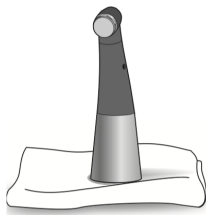

- (3) Stand the contra angle up on a piece of gauze to allow all the excess spray to drain out.

Spray Nozzle (1)

- * Keep this nozzle and use it again when replacing the spray can. For maintenance of contra angle, either spray (LS SPRAY or MORITA MULTI SPRAY) with the spray nozzle or LS OIL may be used.

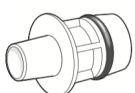

Before autoclaving, the contra angle must be lubricated with the LS SPRAY or MORITA MULTI SPRAY.

⚠ CAUTION

- Do not use any type of spray other than the LS SPRAY or MORITA MULTI SPRAY.
- Failure to lubricate the contra angle will result in a malfunction.

⚠ WARNING

- Prevent spray from splashing into your eyes etc. by always covering the contra angle with gauze or suitable cloth.

⚠ WARNING

- Never direct the spray towards a person.
- Never use the spray near an open flame.
- Hold both the contra angle and the spray can firmly when using the spray. Otherwise, the pressure of the spray could make the contra angle fly out of your hand.

⚠ CAUTION

- Always shake the spray can two or three times before using it. Use the can in an upright position.

⚠ CAUTION

- The motor handpiece could be damaged if the contra angle is attached without allowing the excess spray to drain out first.