BRILLIANT Crios

High performance – made brilliant



Ceramics – a smart solution in the kitchen.

Composite – a BRILLLIANT solution in the practice.



BRILLIANT Crios

Reinforced composite bloc for permanent restorations

Composite has long been the gold standard in direct filling therapy. With BRILLIANT Crios, the well-proven dental material now enters indirect CAD/CAM restoration technology. Compared to ceramics, the CAD/CAM bloc for permanent restorations, which is entirely made of composite, offers considerable advantages.

BRILLIANT CRIOS FOR PERMANENT AESTHETIC RESTORATIONS

REINFORCED COMPOSITE

- High flexural strength for resistant restorations
- effect and pleasant bite feeling

TOOTH-LIKE BEHAVIOUR

- Blends in extremely well to provide natural aesthetics
- High wear resistance and low abrasion on antagonist

EFFICIENT HANDLING

- No firing process required
- Can be modified and repaired
- Exceptional grinding precision for more freedom of preparation
- Effortless polishing for fast gloss

RELIABLE LUTING SYSTEM

- Secure bonding due to ONE COAT 7 UNIVERSAL
- Suitable luting material for every situation

SIZES

12 $12 \times 10 \times 15 \text{ (mm)}$


 $14 \times 12 \times 18 \text{ (mm)}$

Low Translucent (LT)

Bleach, A1, A2, A3, A3.5, B1, B2, B3, C2

High Translucent (HT)

BRILLIANT Crios is the ideal choice for single tooth restorations in the anterior as well as posterior region of teeth. This includes all conventional indications such as inlays, onlays, crowns and veneers.

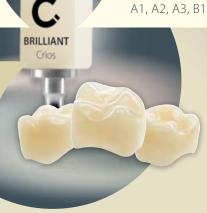










REINFORCED COMPOSITE

The outstanding mechanical properties of BRILLIANT Crios are the result of controlled, stress-free thermal curing. The multimodal composition of dental glass and amorphous silica in combination with a reinforcing resin matrix make BRILLIANT Crios the ideal material for permanent single tooth restorations.

A STRONG MATERIAL

The quality of BRILLIANT Crios is also reflected by its high flexural strength. The compact structure increases the material resistance and reduces the risk of fracturing.

BIAXIAL FLEXURAL STRENGTH

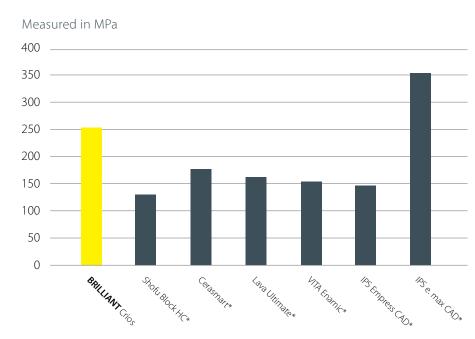


Source: internal data | * not a trademark of COLTENE

SHOCK-ABSORBING EFFECT

The dentine-like modulus of elasticity of BRILLIANT Crios reduces stress peaks during chewing, resulting in decreased risk of material fatigue. Due to its shock-absorbing effect, it is extremely well suited for implant restorations and provides a pleasant bite feeling. As tooth substance and BRILLIANT Crios have a similar modulus of elasticity, loading stress is considerably reduced at the margins.


MODULUS OF ELASTICITY

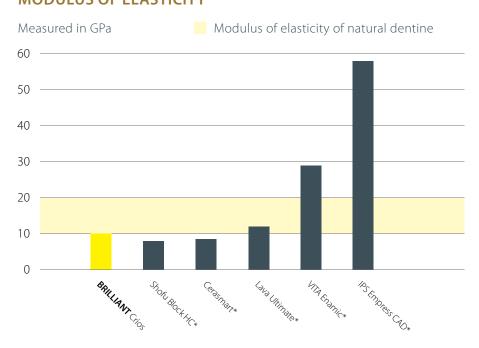

Source: R. Böhner et al., #597, IADR 2015, Antalya, Turkey * not a trademark of COLTENE

TOOTH-LIKE BEHAVIOUR

TWO-BODY WEAR

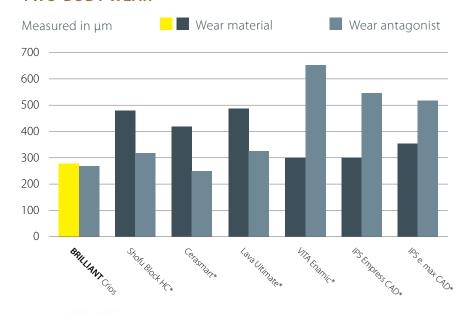

Source: Stawarczyk et al., J Mech Behav Biomed Mater 55, 1-11 (2015) | * not a trademark of COLTENE

WEAR RESISTANT AND GENTLE AT THE SAME TIME

Studies confirm the excellent abrasion resistance of BRILLIANT Crios.

Nonetheless, the wear-resistant composite material proves to be especially kind to the antagonist teeth in comparison to ceramic restorations. BRILLIANT Crios is suitable as a permanent restoration material – even under considerable chewing forces – and thus offers the best of both worlds.

NATURAL AESTHETICS WITH A GLOSSY RESULT

BRILLIANT Crios blends in extremely well and conveys natural aesthetics. A broad spectrum of 13 shades in two translucencies perfectly covers the needs in the anterior as well as posterior region of teeth. Immediately after grinding, BRILLIANT Crios displays an extremely homogeneous surface structure. The restoration can therefore be brought to a high gloss very quickly. Easy repolishing, for example during prophylaxis, is possible at any time if required.

EFFICIENT HANDLING




EXCEPTIONAL GRINDING PRECISION

BRILLIANT Crios is distinguished by superior precision. Even when ceramic materials have already start chipping, BRILLIANT Crios can still be ground with ease and precision in the case of extremely thin margins. This exceptional grinding precision offers greater freedom of preparation.





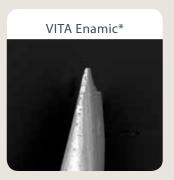



Ground tip 0.1 mm (Sirona inLab MC XL)

Source: C. Kopfmann et al., Eur. J. Prosthodont. Rest. Dent., EMDC Special, P21, 2015, Nuremberg | * not a trademark of COLTENE

TIME AND COST EFFICIENT


- Effortless polishing for fast gloss
- No firing process required
- Quick grinding at high precision
- Easy to modify and repair



RELIABLE LUTING SYSTEM

As part of its adhesive filling therapy, COLTENE offers a matched and proven luting system for BRILLIANT Crios. The key component of the luting concept is the universal adhesive ONE COAT 7 UNIVERSAL. The smooth BRILLIANT EverGlow composite or the dual-curing resin cements

DuoCem and SoloCem offer indication-appropriate solutions, depending on the wall thickness of the restoration and the luting substrate, ranging from natural dentine to a metal abutment.

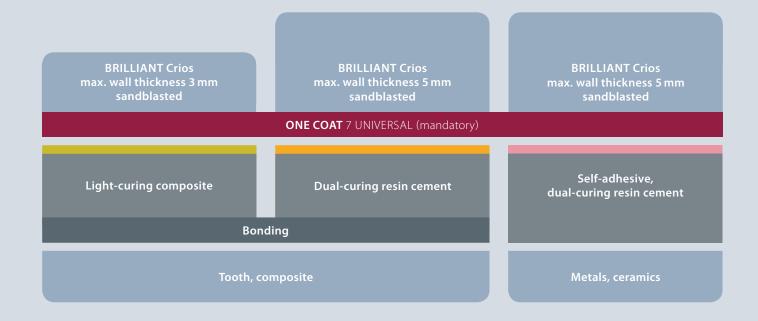







BRILLIANT EverGlow [™] Light-curing composite	DuoCem® Dual-curing resin cement	SoloCem® Self-adhesive, dual-curing resin cement
Ideal as luting material to dentine or composite for thin restoration walls.	Offers exceptional bonding values to dentine or composite, in particular for greater wall thickness.	Convincing bonding to metal or ceramic abutments, without requiring an additional bonding step.
Wall thickness ≤ 3 mm	Wall thickness ≤ 5 mm	Wall thickness ≤ 5 mm

SECURE BONDING

ONE COAT 7 UNIVERSAL

ONE COAT 7 UNIVERSAL is matched perfectly to the BRILLIANT Crios components, resulting in strong anchorage and a reliable adhesion. This is why the ONE COAT 7 UNIVERSAL adhesive is to be used in conjunction with BRILLIANT Crios.

The monomers (red) contained in ONE COAT 7 UNIVERSAL penetrate into the polymerised resin matrix of BRILLIANT Crios



Polymer chains are formed during polymerisation of the monomers and these are anchored in the resin matrix.




The double bonds contained in the resin matrix form chains during polymerisation of the monomers.

BRILLIANT Crios ONE COAT 7 UNIVERSAL

SHEAR BOND STRENGTH WITH ONE COAT 7 UNIVERSAL

Measured in MPa

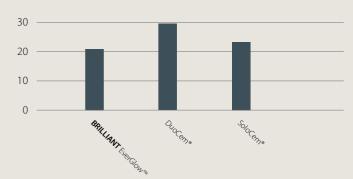

The BRILLIANT EverGlow, DuoCem and SoloCem luting materials demonstrate excellent bonding strengths to give a secure adhesion to BRILLIANT Crios.

Source: internal data

FURTHER PROPERTIES

ONE COAT 7 UNIVERSAL is a light-curing one component universal adhesive.

- Exceptional bonding values on enamel and dentine
- Excellent marginal seal
- Minimised postoperative sensitivity
- For all application techniques (Self-Etch, Selective Etch, Total Etch)



CLINICAL CASE

Restored teeth

26 with BRILLIANT Crios A2 HT 27 with BRILLIANT Crios A2 LT

Luting

ONE COAT 7 UNIVERSAL with BRILLIANT EverGlow A2/B2





ORDER INFORMATION

BRILLIANT Crios

60020095 Intro Kit CEREC 12

Each 1 \times A2 LT 12, A3 LT 12, A3.5 LT 12, A2 HT 12, A3 HT 12 $5 \times 0.1 \, \text{ml}$ ONE COAT 7 UNIVERSAL Single Dose

60020097 Intro Kit CEREC 14

Each 1 × A2 LT 14, A3 LT 14, A3.5 LT 14, A2 HT 14, A3 HT 14
5 × 0.1 ml ONE COAT 7 UNIVERSAL Single Dose



CEREC/inLab – 5 blocs				
	Low Translucent		High Translucent	
	12	14	12	14
Bleach	60019998 BL LT 12	60019985 BL LT 14		
A 1	60019999 A1 LT 12	60019986 A1 LT 14	60020007 A1 HT 12	60019994 A1 HT 14
A2	60020000 A2 LT 12	60019987 A2 LT 14	60020008 A2 HT 12	60019995 A2 HT 14
А3	60020001 A3 LT 12	60019988 A3 LT 14	60020009 A3 HT 12	60019996 A3 HT 14
A3.5	60020002 A3.5 LT 12	60019989 A3.5 LT 14		
B1	60020003 B1 LT 12	60019990 B1 LT 14	60020010 B1 HT 12	60019997 B1 HT 14
B2	60020004 B2 LT 12	60019991 B2 LT 14		
В3	60020005 B3 LT 12	60019992 B3 LT 14		
C2	60020006 C2 LT 12	60019993 C2 LT 14		

ONE COAT 7 UNIVERSAL

60019538	ONE COAT 7 UNIVERSAL Intro Kit

1 × 5 mlONE COAT 7 UNIVERSAL1 × 2.5 mlEtchant Gel S1 × 20 pcs.Application tips1 × 50 pcs.Brushes black1 ×Mixing well

1 × Cover for mixing well

60019539 ONE COAT 7 UNIVERSAL Refill

1 × 5 ml ONE COAT 7 UNIVERSAL

60019540 ONE COAT 7 UNIVERSAL Single Dose

 1×50 pcs. ONE COAT 7 UNIVERSAL Single Dose

 1×50 pcs. Brushes black

7054 ONE COAT 7.0 Activator Kit

 1×3 ml ONE COAT 7.0 Activator

 1×50 pcs. Brushes black

BRILLIANT EverGlow

60019720 BRILLIANT EverGlow Trial Kit Syringe

1 x 3 g A2 / B2 1 x 3 g A3 / D3 1 x 3 g A3.5 / B3 1 x 3 g Translucent

60019721 BRILLIANT EverGlow Trial Kit Tips

8 x 0.2 g A2/B2 8 x 0.2 g A3/D3 8 x 0.2 g A3.5/B3 8 x 0.2 g Translucent

DuoCem

60014381	DuoCem Intro Kit
$1 \times 5 ml$	DuoCem Dentin
$1 \times 5 ml$	DuoCem Trans
1×10 pcs.	Mixing Tip Short Fine
1 × 10 pcs.	Mixing Tip Short Super Fine

SoloCem

60013986	SoloCem Intro Kit
$1 \times 5 \text{ml}$	SoloCem Dentin
$1 \times 5 \text{ml}$	SoloCem Trans
1×10 pcs.	Mixing Tip Short Fine
1×10 pcs.	Mixing Tip Short Super Fine

© 2016 Coltène/Whaledent AG - www.coltene.com



Coltène/Whaledent Ltd.
The President Suite
Kendal House, Victoria Way
Burgess Hill, West Sussex
RH15 9NF / UK
Tel +44 1444 235486
Fax +44 1444 870640
info.uk@coltene.com

Coltène/Whaledent AG
Feldwiesenstr. 20
9450 Altstätten / Switzerland
T +41 71 757 53 00
F +41 71 757 53 01
info.ch@coltene.com

Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, Ohio 44223 / USA
T +1 330 916 8800
F +1 330 916 7077
info.us@coltene.com

