# HyFlex<sup>™</sup> CM & EDM



Stays on track



# **HyFlex™**EDM Electrical Discharge Machining

The new HyFlex EDM files constitute the 5th generation root canal files. HyFlex EDM NiTi files have completely new properties due to their innovative manufacturing process using electric discharge machining. Workpieces are machined in the EDM manufacturing process by generating a potential between the workpiece and the tool. The sparks generated in this process cause the surface of the material to melt and evaporate.

This creates the unique surface of the new Niti files and makes the HyFlex EDM files stronger and more fracture resistant. This entirely unique combination of flexibility and fracture resistance makes it possible to reduce the number of files required for cleaning and shaping during root canal treatments without having to compromise preservation of the root canal anatomy.

Just like HyFlex CM files, HyFlex EDM files have the already familiar controlled memory effect (CM). This results in very similar properties in terms of material flexibility and regeneration.

- → Up to 700 % higher fracture resistance
- → Specially hardened surface
- → Less filing required for treatment success

# SEQUENCE STEP BY STEP

ORIFICE OPENER

(optional) 25 / .12

Glidepath File **10 / .05** 

Preparation File **20 / .05** 

HyFlex<sup>™</sup> OneFile 25 / ~


# **Clinical Cases**

"These files definitely know their way around curves. That's for sure. I love them!"

**Dr. Antonis Chaniotis** 





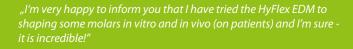

Dr. Eugenio Pedullà










Source: Dr. Antonis Chaniotis




FINISHING FILES (optional) 40/.04 50/.03

60/.02

- → High flexibility
- → Extreme resistance to fracture
- → Centred canal preparation

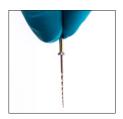

HyFlex files are manufactured utilising a unique process in which the crystallographic phase transitions from austenite to martensite at room temperature in contrast to conventional NiTi files, making the files extremely flexible and fracture resistant.

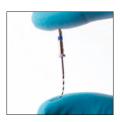

Due to the controlled memory, the files always follow the anatomy of the canal, thus significantly reducing the risk of ledging, transportation or perforation of a canal. Like stainless steel files, the HyFlex files can be pre-bent.



This is a crucial advantage in extremely curved root canals to avoid the creation of steps.

## Up to 700% higher fracture resistance


Tests have shown that HyFlex NiTi files are significantly more resistant to fatigue than conventional NiTi files. In the new HyFlex EDM NiTi files, this effect is further increased by 300-700 %. The new EDM manufacturing process

results in another form fit and hardness of the surface of the file.

- → Fracture resistance by visual inspection
- → Regeneration by thermal treatment

HyFlex NiTi have a built-in shape memory. They prevent stress during preparation by changing their spiral shape. They regain their shape after heat treatment. A normal autoclaving process is enough to return the files to their original shape and to regenerate the crystal structures and resistance to fatigue.

Should the file fail to regain its shape after heat treatment, it is plastically deformed, thus increasing the risk of fracture. These files should not be used after visual inspection. The number of reuses, which can be determined by the user by visual inspection, is higher than that of conventional NiTi files.

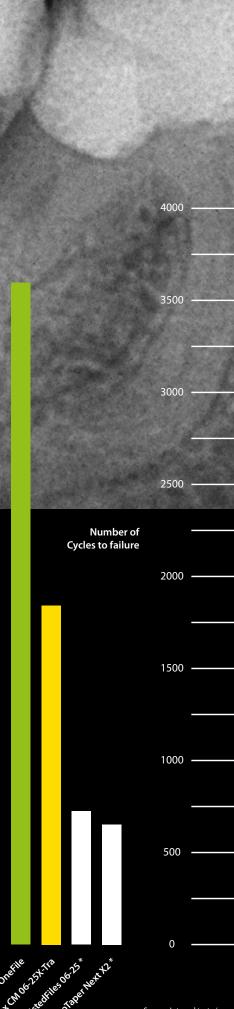

Source: Internal tests /
\* Not a COLTENE trademark



#### **HyFlex**<sup>™</sup>EDM Reduced Number of Files

Depending on the clinical situation, use of HyFlex EDM files reduces the number of files to 2-3 pieces particularly in straight and larger

All HyFlex EDM NiTi Files can be used at 400 rpm and at a torque of up to 2.5 Ncm (25 mNm) excepted the Glidepath files which can be used with 300 rpm and at a torque of up to 1.8 Ncm (18 mNm).




#### CORONAL MIDDLE APICAL Hand Files 25/.08 20/.04 25/04 20/06 30/04 40/04 HyFlex HyFlex HyFlex HyFlex HyFlex HyFlex CM CM CM CM

## **HyFlex**<sup>™</sup>CM Single Length Technique

The single length technique, in which all files are used except the short Orifice Opener up to the working length, allows accurate and anatomically accurate preparation using HyFlex

Again, all files can be used except the Orifice Opener up to the working length (500 rpm; 2.5 Ncm (25 mNm)).

### Regeneration by thermal treatment

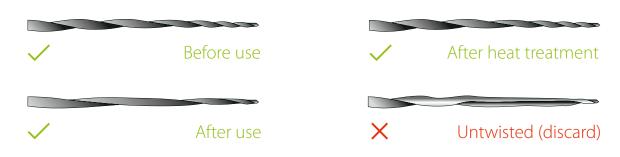



# **HyFlex**<sup>™</sup>GPF Glide Path File



The HyFlex GPF ensure optimal shaping of the glide path due to the controlled memory. They are characterized by good durability and resistance to fracture.

Despite the extreme flexibility that allows the files to follow unusual anatomies, the HyFlex Glide Path Files are very easy to use and ensure safe treatment.

# HyFlex Organizer \*\*Coltene No. 1 Control of Control of

The HyFlex CM Introkit Compact is an optimal choice for starting a successful endodontic treatment.

In this compact set, the reliable HyFlex CM Sequence is arranged for using the single length technique together with space-saving storage and guttapercha points tailored to the system. The set allows optimal access to the high-quality COLTENE ENDO product range. In a small starter set, the HyFlex EDM

Introkit Compact offers 4 file sizes with which treatments, particularly of straight canals, can be preformed very simply to ISO size 40. Like all HyFlex NiTi files, the HyFlex EDM are presterilised and together with the small, practical HyFlex Organizer and dimensionally correct Greater Taper guttapercha points, treatment can be started immediately.

#### **HyFlex CM Introkit:**

- 1 × HyFlex Organizer
- 1 x HyFlex CM NiTi File Refill Sequence 25 mm
- 1 × ROEKO Guttapercha Points Greater Taper 0.04, Assorted 20-45

#### **HyFlex EDM Introkit:**

- 1 × HyFlex Organizer
- 1 × HyFlex EDM Shaping Set Simple + 40/.04 Finisher File
- 1 × ROEKO Guttapercha Points Greater Taper 0.04, Assorted 20-45

# **HyFlex**NiTi Files & Accessories

#### **PRESTERILIZED**





#### Coltène/Whaledent AG Feldwiesenstrasse 20 9450 Altstätten/Switzerland T+41 71 757 53 00


T +41 71 757 53 00 F +41 71 757 53 01 info.ch@coltene.com

Coltène/Whaledent GmbH + Co. KG Raiffeisenstraße 30 89129 Langenau/Germany T +49 7345 805 0 F +49 7345 805 201 info.de@coltene.com

#### Coltène/Whaledent Ltd. The President Suite

The President Suite Kendal House, Victoria Way Burgess Hill, West Sussex RH15 9NF/UK T +44 1444 235486 F +44 1444 870640 info.uk@coltene.com

#### © 2018 Coltène/Whaledent AG - www.coltene.com

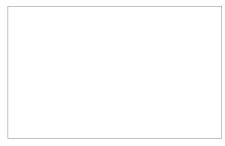



Content			25 mm	
2 pcs 15/.01; 2 pcs 15/.02; 2 pcs 20/.02		•••••••••••••••••••••••••••••••••••••••	60018753	
HyFlex CM NiTi File Refills, 6 pcs				
Size / Taper	19 mm	21 mm	25 mm	31 mr
25/.08	60011048			
15/.04		60011049	60011065	6001828
20/.04		60011050	60011066	6001828
25/.04		60011052	60011067	6001828
30/.04		60011053	60011068	6001828
35/.04		60011054	60011069	
45/04	·····	60011055	60011070	6001828
45/.04 50/.04		60011056	60011071	6001828
60/.04	·····	60011058	60011073	6001829
20/.06	·····	60011059	60011074	6001829
25/.06		60011060	60011075	
20/.06, X-tra spirals		60018699	60018700	6001870
25/.06, X-tra spirals		60018702	60018703	6001870
30/.06		60011061	60011076	6001829
35/.06		60011062	60011077	
40/.06		60011063	60011078	6001829
HyFlex CM NiTi File Sequence Single length				
Content / Sizes		21 mm	25 mm	31 m
20/.04; 25/.04; 30/.04; 40/.04; 20/.06; 25/.08 19 mm	1	60011064	6001 1079	6001 829
HyFlex CM NiTi File Sequence Crown down				
Content / Sizes		21 mm	25 mm	31 m
S (25/.08 19 mm; 35/.04; 30/.04; 25/.04; 20/.04; 15/.		60018689	60018692	6001869
M (25/.08 19 mm; 60/.04; 50/.04; 45/.04; 40/.04; 35	/.04)	60018690	60018693	6001869
L /25/00 10 mm, 40/06, 25/06, 20/06, 25/06, 20/	(06)	60019601	60019604	6001060
HyFlex Introkits	(.06)	60018691	60018694	
HyFlex Introkits HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact	5 mm Sequence, Sequence, ROEK(	ROEKO GP Greater Taper) D GP Greater Taper)	60018694	6001953
L (25/08 19 mm; 40/06; 35/06; 30/.06; 25/.06; 20/. HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25/. HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit	5 mm Sequence, Sequence, ROEK(	ROEKO GP Greater Taper) D GP Greater Taper)	60018694	6001953
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25  HyFlex CM Introkit Compact (Organizer, 25 mm S  HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper) O GP Greater Taper) O GP Greater Taper)		6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper) O GP Greater Taper) O GP Greater Taper)		6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25  HyFlex CM Introkit Compact (Organizer, 25 mm S  HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini  Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef  HyFlex EDM NiTi File Sequences	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper) O GP Greater Taper) O GP Greater Taper)		6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper) D GP Greater Taper) O GP Greater Taper) O GP Greater Taper) KO GuttaFlow bioseal 2.5 m	);	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25  HyFlex CM Introkit Compact (Organizer, 25 mm S  HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fin  Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef  HyFlex EDM NiTi File Sequences  Content / Sizes	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm	l; 25 mm	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Finiend Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onel HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267	25 mm 60019363	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 45 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Finitendo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM One HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  KO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277	25 mm 60019363 60022278	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Finiendo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs	5 mm Sequence, Sequence, ROEK( isher File, ROEK( , 25/12); 1 × ROEK	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  KO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277	25 mm 60019363 60022278	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper	5 mm Sequence, Sequence, ROEKO isher File, ROEKO 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268	25 mm 60019363 60022278 60019364	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 40.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268	25 mm 60019363 60022278 60019364	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 40/04 Finitend Highlight Kit)  1 x HyFlex EDM Shaping Set Simple (10/05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022268  21 mm	25 mm 60019363 60022278 60019364 25 mm	600195: 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 40.04 Finitender) (Organizer, HyFlex EDM Shaping Set + 40.04 Finitender) Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022275  60022270	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595	600195: 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  60  10/.05 Glidepath File  20/.05 Preparation File  25/~ HyFlex OneFile  40/.04 Finishing File	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper) D GP Greater Taper) O GP Greater Taper) CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022269  60022275  60022270  60022271	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597	600195: 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 40.04 Finitend Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes Shaping Set Simple 10.05; 25/~; 25./12 15 mm Shaping Set Medium 10.05; 20.05; 25/~ Finishing Set 40.04; 50.03; 60.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25./12 Orifice Opener  10.05 Glidepath File 20.05 Preparation File 25/~ HyFlex OneFile  40.04 Finishing File 50.03 Finishing File	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022275  60022270  60022271	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	600195: 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onel HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File  25/~ HyFlex OneFile  40/.04 Finishing File  50/.03 Finishing File  60/.02 Finishing File	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GP Greater Taper)  21 mm  60022267  60022277  60022275  60022270  60022271  60022272	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597	600195: 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Finisendo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM One HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File 20/.05 Preparation File 25/~ HyFlex OneFile 40/.04 Finishing File 50/.03 Finishing File 60/.02 Finishing File 60/.02 Finishing File	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GUITAFIOW bioseal 2.5 m  21 mm  60022267  60022277  60022275  60022270  60022271  60022272  60022274	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	6001953 6001878 6001966
HyFlex Introkits  HyFlex CM Introkit Premium  (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 440.04 Finisende Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File  25/- HyFlex OneFile  40/.04 Finishing File  50/.03 Finishing File  60/.02 Finishing File  HyFlex Accessories  Item	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022279  60022270  60022271  60022272  60022274  Content	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	6001953
HyFlex Introkits  HyFlex CM Introkit Premium  (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 440.04 Finisendo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File  20/.05 Preparation File  40/.04 Finishing File  60/.02 Finishing File  60/.02 Finishing File  HyFlex Accessories  Item  Endo Procedure Block	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GUITAFIOW bioseal 2.5 m  21 mm  60022267  60022277  60022275  60022270  60022271  60022272  60022274	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	600196
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 440.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes Shaping Set Simple 10.05; 25/~; 25./12 15 mm Shaping Set Medium 10.05; 20.05; 25/~ Finishing Set 40.04; 50.03; 60.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25./12 Orifice Opener  10.05 Glidepath File 20.05 Preparation File 25/~ HyFlex OneFile 40.04 Finishing File 60.02 Finishing File 60.02 Finishing File HyFlex Accessories  Item Endo Procedure Block Endo Organizer	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  O GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022276  60022270  60022271  60022274  Content  1 pc	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	6001966 6001203 6001108 6001108
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 45 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Finisendo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onel HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  26/.12 Orifice Opener  27/.12 Orifice Opener  28/.13 Orifice Opener  29/.03 Finishing File  40/.04 Finishing File  40/.03 Finishing File  40/.03 Finishing File  40/.03 Finishing File  40/.04 Finishing File  40/.05 Procedure Block  Endo Procedure Block  Endo Practice Block Pack	5 mm Sequence, Sequence, ROEK( isher File, ROEK( 25/12); 1 × ROEI File 25	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022270  60022271  60022272  60022274  Content  1 pc  1 pc	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	6001966 6001203 6001108 6001108
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 45 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onel HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  60/.05 Glidepath File  20/.05 Preparation File  25/~ HyFlex OneFile  40/.04 Finishing File  50/.03 Finishing File  60/.02 Finishing File  HyFlex Accessories  Item  Endo Procedure Block  Endo Organizer  Endo Practice Block Pack	5mm Sequence, Sequence, ROEK isher File, ROEK 25/.12); 1 × ROEI File 25	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022270  60022271  60022272  60022274  Content  1 pc  1 pc	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598	6001953 6001878 6001966 6002203 6001108 6001108
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM One HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File  25/— HyFlex OneFile  40/.04 Finishing File  50/.03 Finishing File  60/.02 Finishing File  HyFlex Accessories  Item  Endo Procedure Block  Endo Organizer  Endo Practice Block Pack  CanalPro Cordless Handpiece	5 mm Sequence, Sequence, ROEK isher File, ROEK 25/.12); 1 × ROEI File 25	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022270  60022271  60022272  60022274  Content  1 pc  1 pc	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598 60019599	6001108 6001108 6001108
HyFlex Introkits  HyFlex CM Introkit Premium  (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM Onef HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File  25/~ HyFlex OneFile  40/.04 Finishing File  60/.02 Finishing File  60/.02 Finishing File  HyFlex Accessories  Item  Endo Procedure Block Endo Organizer  Endo Practice Block Pack  CanalPro Cordless Handpiece  CanalPro CL Complete Set (without Apex Locato (Motor Handpiece, Charger, Contral Angle)	55mm Sequence, 65equence, ROEK isher File, ROEK 25/.12); 1 × ROEI File 25  15 mm 660019596	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022270  60022271  60022272  60022274  Content  1 pc  1 pc	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019595 60019597 60019598 60019599	6001953 6001878 6001966 6002203 6001108 6001108
HyFlex Introkits  HyFlex CM Introkit Premium (2 pcs Organizer, 2 pcs 21 mm Sequence, 2 pcs 25 HyFlex CM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, 25 mm S HyFlex EDM Introkit Compact (Organizer, HyFlex EDM Shaping Set + 40/.04 Fini Endo Highlight Kit  1 x HyFlex EDM Shaping Set Simple (10/.05, 25/~, 1 x ROEKO Guttapercha Points HyFlex EDM One HyFlex EDM NiTi File Sequences  Content / Sizes  Shaping Set Simple 10/.05; 25/~; 25/.12 15 mm  Shaping Set Medium 10/.05; 20/.05; 25/~ Finishing Set 40/.04; 50/.03; 60/.02  HyFlex EDM NiTi File Refills, 3 pcs  Size / Taper  25/.12 Orifice Opener  10/.05 Glidepath File  20/.05 Preparation File  25/~ HyFlex OneFile  40/.04 Finishing File  60/.02 Finishing File  60/.02 Finishing File  60/.03 Finishing File  Endo Procedure Block Endo Organizer Endo Practice Block Pack  CanalPro Cordless Handpiece	55mm Sequence, 65equence, ROEK isher File, ROEK 25/.12); 1 × ROEI File 25  15 mm 660019596	ROEKO GP Greater Taper)  D GP Greater Taper)  O GP Greater Taper)  CO GuttaFlow bioseal 2.5 m  21 mm  60022267  60022277  60022268  21 mm  60022269  60022270  60022271  60022272  60022274  Content  1 pc  1 pc	25 mm 60019363 60022278 60019364 25 mm 60019594 60022276 60019597 60019598 60019599	6001953 6001878 6001966 6002203 6001108 6001108