### Planmeca Compact<sup>™</sup> i Classic **Quick guide**

## Patient chair control


Control panel		
Chair to autoposition A Backrest down	Short activation Long activation	A *
Chair to autoposition B Backrest up	Short activation Long activation	Л. В
Chair to autoposition C Chair up	Short activation Long activation	°.↑
Chair to autoposition D  Chair down	Short activation Long activation	D V
Chair to rinsing position Chair to previous position	Short activation when LED OFF Short activation when LED ON	S
Chair to rinsing position only	Long activation	
Flexy button		
Chair to rinsing position		
Foot control		
Short activation Long activation Chair to autoposition A Long activation Backrest down Short activation Long activation	Chair to autoposition C Chair up B Chair to autoposition C Long activation Backrest up Chair to autoposition D Chair down	B Chair to rinsing position




# Dental unit control

Control panel			
Bowl rinse	Short activation		
Cup fill and bowl rinse		2 x Short activation Or 0.5-1 sec. activation	
Cup fill as long as pressed		Long activation	
Door open / assistant call function for preprogrammed time Short activation			
Door open / assistant call function as long as pressed   Long activation			
Operating light On ► Off			
Intensity of operating light		Long activation	
Maintenance procedure	Short flushing	3 sec. activation	
	Long flushing	6 sec. activation	
Foot control	Dot control         No instrument activated		
Preprogrammed unit function Short activation Door open / assistant call function for preprogrammed time Long activation Door open / assistant call function as long as pressed Short activation Door activation Bowl rinse 2 x Short activation Cup fill and bowl rinse Long activation Cup fill and bowl rinse Long activation Cup fill and bowl rinse			

#### **Instrument control**

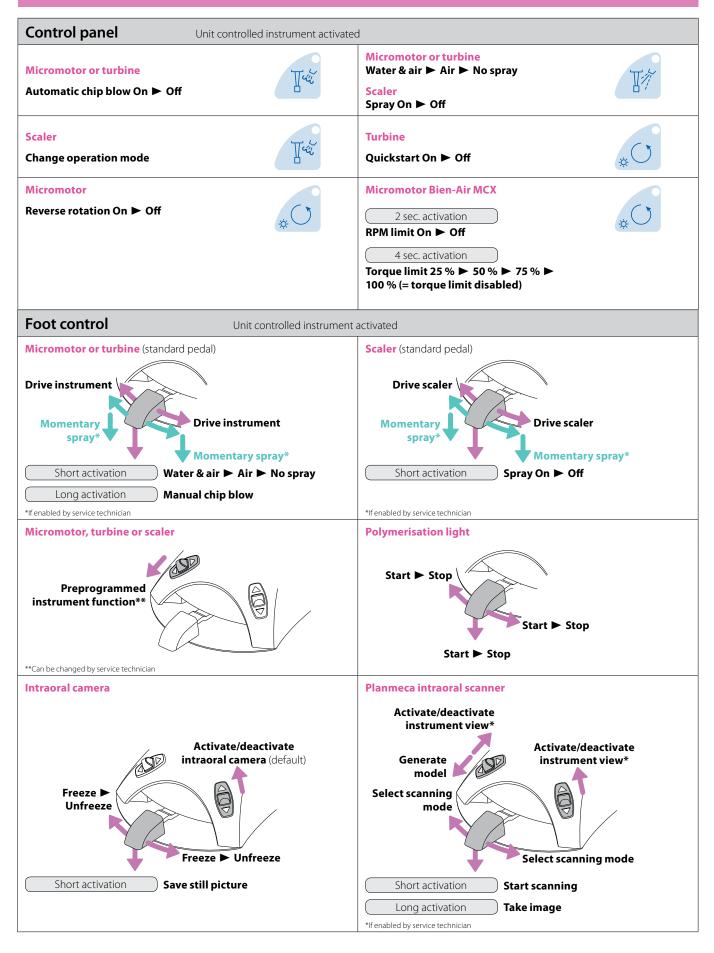

## Programming

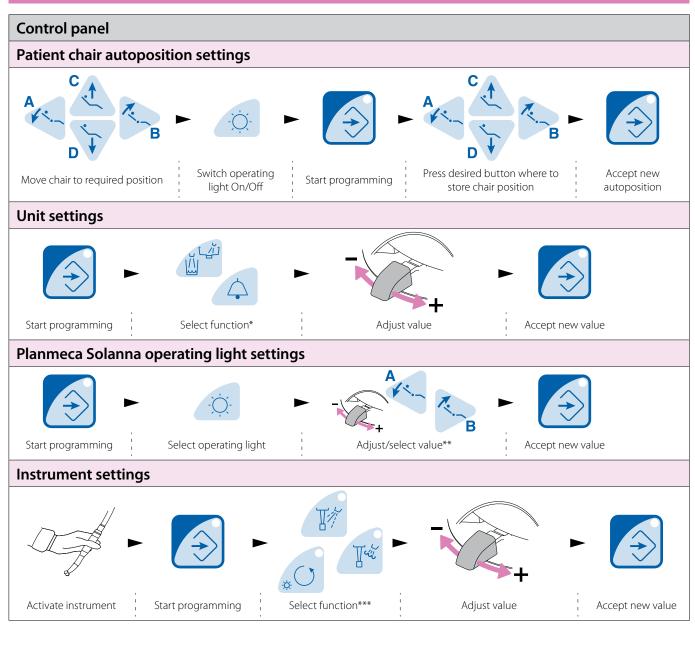

\*Unit functions



**Duration of cup filling** 2 - 10 sec.



**Duration of bowl rinsing** 5 - 240 sec.



Duration of door open/ assistant call signal

**Maximum brightness** 

Mild > Strong > Glaring

0 - 250 sec.

\*\*Operating light functions

30-

**Intensity** 30 - 100%







·...

Til

White light tone Warm > Neutral > Cool



**Chip blow water** 0 (off) - F=100 (full)

CR Chip blow air 0 (off) - F=100 (full)



Intensity of instrument light L. oFF (off), 70 - 100%

¢) ۲.

Duration of polymerisation cycle 5 - 100 sec.

For detailed instructions, please see the user's manual. Copyright Planmeca. Rights for changes reserved.