
Planmeca Compact[™] i3 Maintenance and infection control

After every patient

Surfaces

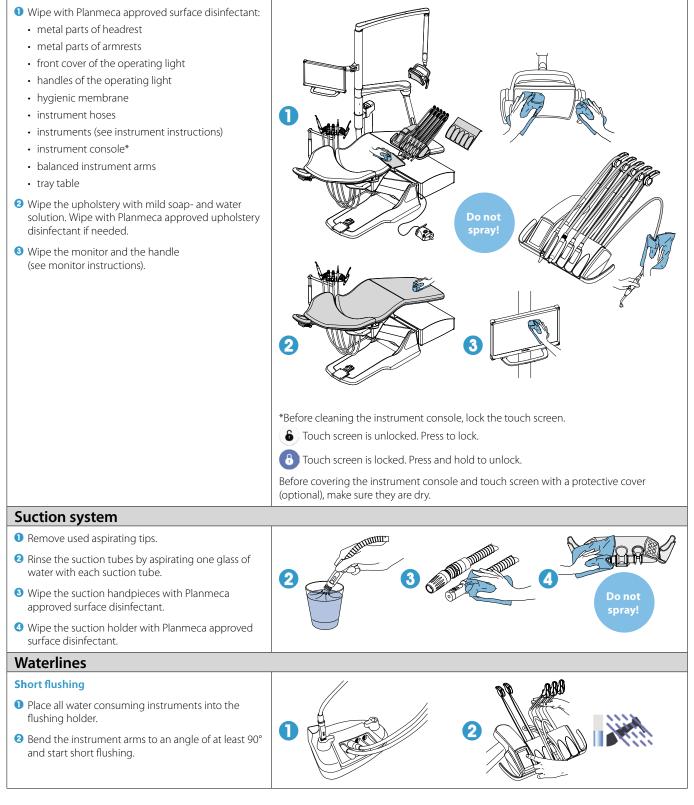



In the morning

Suction system

- Rinse the suction tubes by aspirating 0.5 l of water with each suction tube.
- Wipe the suction handpieces with Planmeca approved surface disinfectant.



O




ABB

Waterlines


Long flushing

- Place all water consuming instruments into the flushing holder.
- 2 Bend the instrument arms to an angle of at least 90° and start long flushing.


After working day

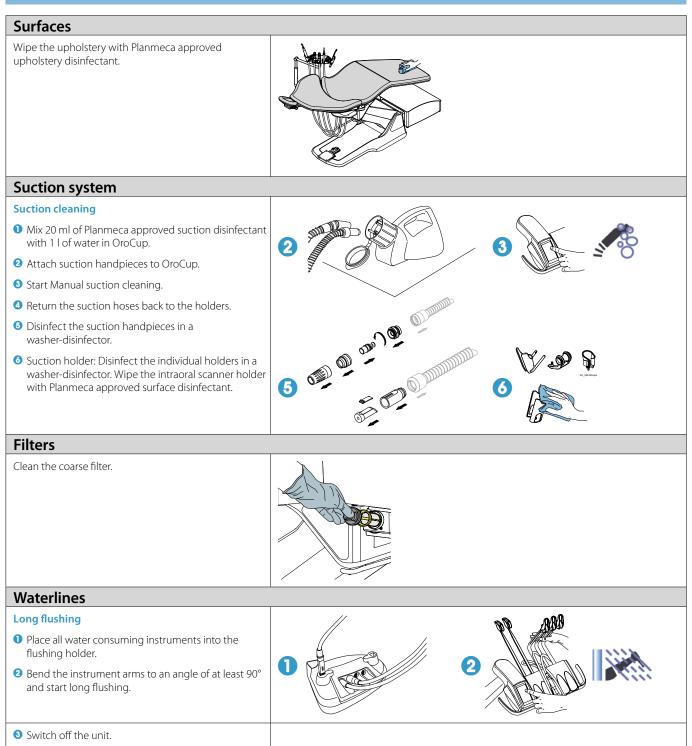

Weekly

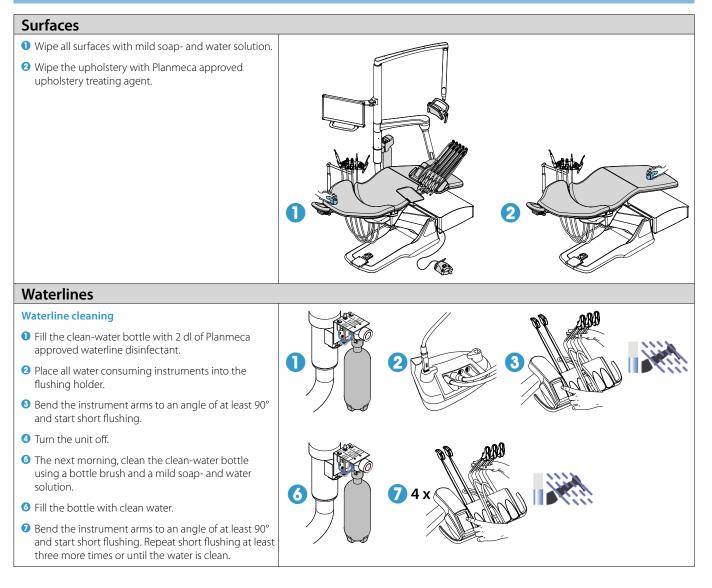

Monthly

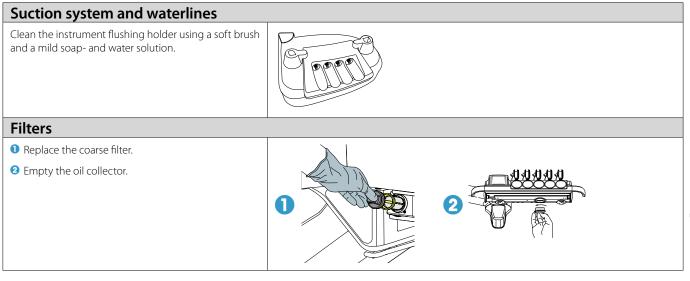

30022014_1/10.2019/en