

Patient chair control

Control panel



Touch screen is unlocked. Press to lock.



Touch screen is locked. Press and hold to unlock.

Backrest down as long as pressed



Backrest up as long as pressed




Chair down as long as pressed



Chair up as long as pressed



Chair to autoposition

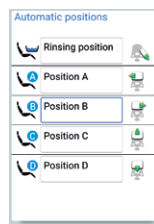


Flexy button

Chair to rinsing position



Foot control

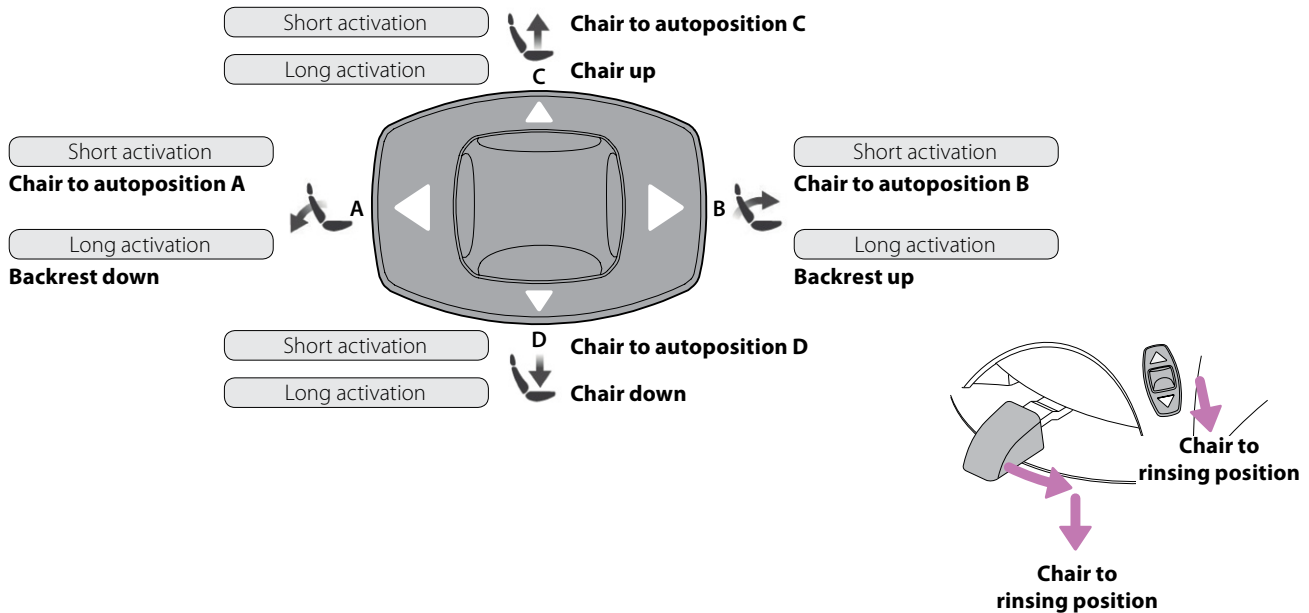

Dental unit control

Control panel



Touch screen is unlocked. Press to lock.



Touch screen is locked. Press and hold to unlock.

Cup fill and bowl rinse

Short activation



Cup fill as long as pressed

Long activation

Bowl rinse

Short activation



Door open / assistant call function for preprogrammed time

Short activation



Door open / assistant call function as long as pressed

Long activation

Operating light On ► Off

Short activation



Intensity of operating light

Long activation

Operating light's composite mode On ► Off

Short activation



Intensity of operating light in composite mode

Long activation

Start required timer



Select timer number

Maintenance procedure



Select procedure

Foot control


No instrument activated

Short activation

Door open / assistant call function for preprogrammed time

Long activation

Door open / assistant call function as long as pressed

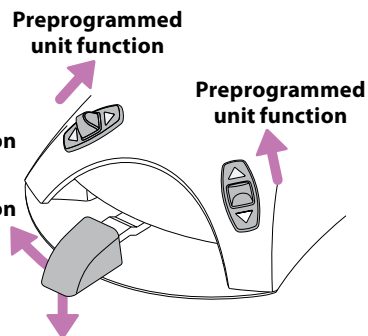

Short activation

Cup fill and bowl rinse

Long activation

Cup fill as long as pressed

Instrument control

Control panel



Touch screen is unlocked. Press to lock.




Touch screen is locked. Press and hold to unlock.

<p>Micromotor or turbine Automatic chip blow On ► Off</p>	<p>Micromotor, turbine or scaler Instrument light On ► Off</p>	<p>Micromotor or turbine Water & air ► Air ► No spray</p> <p>Scaler Spray 1 ► Spray 2 ► No spray</p>
<p>Scaler Change operation mode</p>	<p>Turbine Power reduction On ► Off</p>	<p>Turbine Quickstart On ► Off</p>
<p>Micromotor Reverse rotation On ► Off</p>	<p>Micromotor RPM limit On ► Off</p>	<p>Micromotor Bien-Air MX2 Preset selection</p>
<p>Micromotors Bien-Air MX2 and Bien-Air MCX Torque limit On ► Off</p>	<p>Micromotor Bien-Air MX2 Toggle drive mode when torque limit On</p>	<p>Micromotor Bien-Air MX2 Toggle drive mode when torque limit On</p>
		<p>Micromotor Bien-Air MX2 Toggle drive mode when torque limit On</p>

Foot control


Unit controlled instrument activated

Micromotor or turbine (standard pedal)

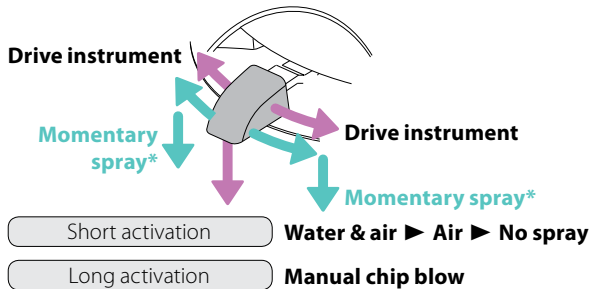


*If enabled by service technician

Scaler (standard pedal)

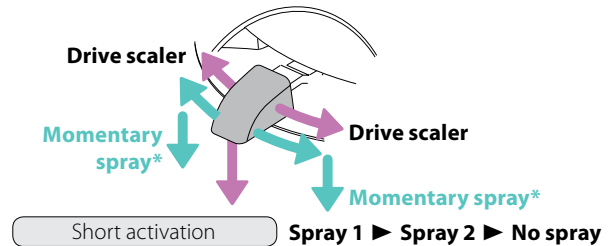


*If enabled by service technician

Micromotor, turbine or scaler

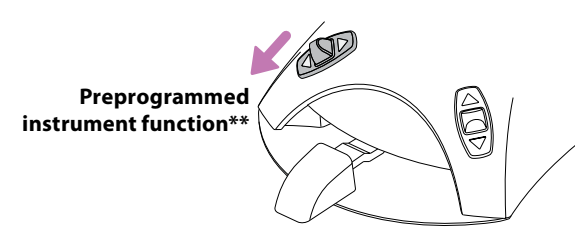


**Can be changed by service technician


Polymerisation light

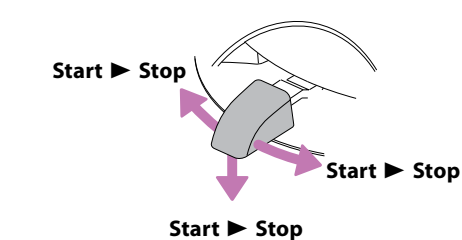

Intraoral camera

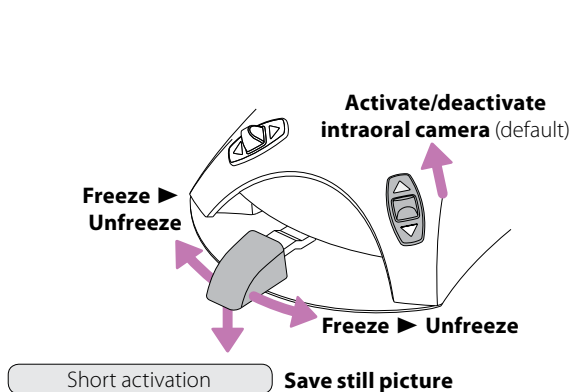

Planmeca intraoral scanner

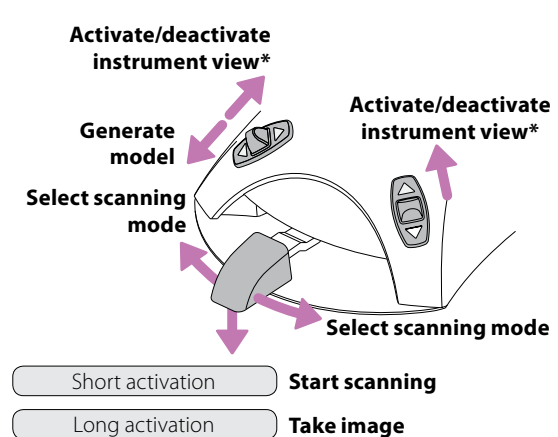

*If enabled by service technician

Motorised headrest control

Foot control



Short activation

Normal mode
On ► Off



Long activation

Tilt mode
On ► Off

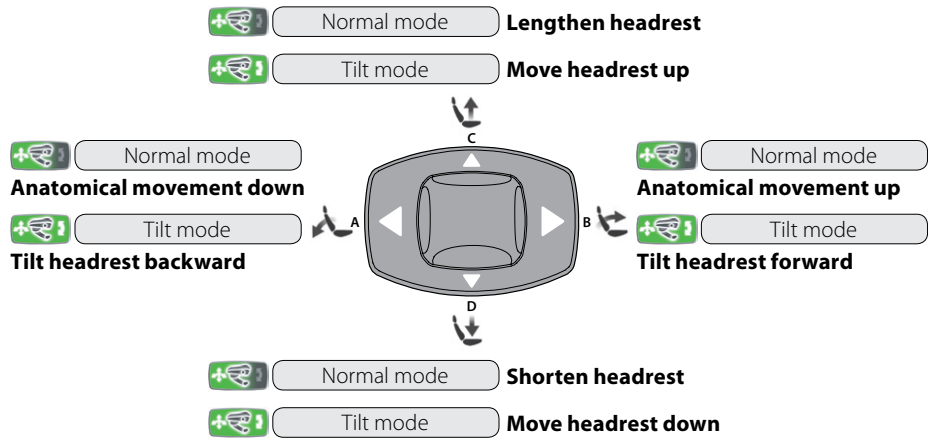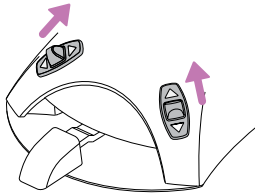

Programming

Control panel



Touch screen is unlocked. Press to lock.



Touch screen is locked. Press and hold to unlock.

Patient chair autoposition settings



The function is enabled.



The function is disabled.



Unit settings




The function is enabled.



The function is disabled.

Most programming follows this simple sequence:

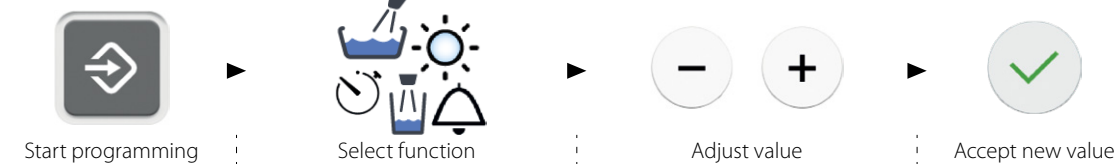

Instrument settings




The function is enabled.



The function is disabled.

Most programming follows this simple sequence:

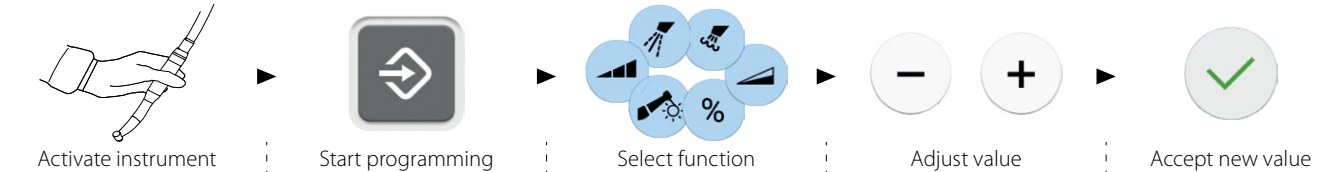


Time and date settings



The function is enabled.



The function is disabled.

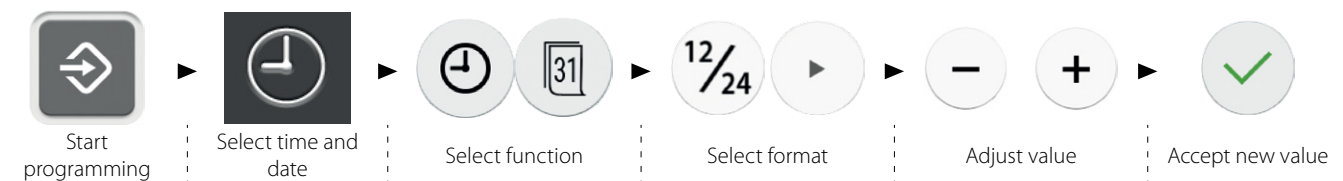

30011780_2/01.2019/en