Professional Dental Manufacturer Since 2003

Features:

- 1- Preset speed mode(high-mid-low speed)
- 2- A gentle, pulsed stimulus begins to increase at a rate specifically designed for patient comfort.
- 3- If the patient indicates perception, simply release the button the stimulus stops immediately, but the numbers remain frozen on the screen for about 3 minutes.
- 4- Specifically designed for patient comfort.
- 5- Convenient to operate.
- 6- Maximum stimulus 80.
- 7. Easy diagnosis:
- a) <u>Between O-40</u>, the patient feels ache and anesthesia: alive teeth nerve.
- b)<u>Between 4O-80,</u> the patient feels ache and anesthesia, part of teeth nerve dead;
- c) 80, no above-mentioned reaction, dead teeth nerve.
- 8-The device turns itself off in three minutes after operation (No wasted batteries)

This product has been applied layout-design patent, Counterfeit must be investigated; NO ZL2007303339259

CO-PULPTESTER





