

dentisan

INNOVATION. EDUCATION. COMPLIANCE.

DentiChlor[®] tablets

Disinfectant



DentiChlor tablets are suitable for all aspects of surface and environmental disinfection.

With broad spectrum efficacy, they are suitable for a wide variety of applications, including disinfection of water storage containers, dental impressions and blood spills.

A convenient, reliable and consistent means of producing chlorine solutions for disinfectant operations.

- DentiChlor Disinfectant Tablets are used to deliver the correct and exact strength chlorine solutions for disinfection of blood spills and surfaces in high-risk areas.
- Because the chlorine-based solution is made up on-the-spot as required there are no transportation, storage or contamination issues normally associated with hazardous and naturally unstable hypochlorite solutions.
- Chlorine tablets are stable, compact and easy to store for longer periods until required.
- Chlorine is the chemical recommended by healthcare organisations around the world to disinfect surfaces after blood spills have occurred and for general environment disinfection.
- Enables users to create a disinfectant solution with a specific concentration of chlorine.
- The tablet format is stable and easier to store and transport than liquid disinfectant solutions.
- Effective against a wide range of pathogens and other infectious agents.

Instructions:

- For surfaces dilute 1 tablet in 1L of water.
- For impressions dilute 1 tablet in 1L of water. Contact time is 5 mins.
- For disinfection of water storage containers dilute 1 tablet in 5L of water. Leave overnight.

Pack Size : 200 tablets.
Each tablet = 3g

Item code : QD-DS290