



#### Health & Safety

**Warning.** H319 Causes serious eye irritation. H335 May cause respiratory irritation. H410 Very toxic to aquatic life with long lasting effects. EUH031 Contact with acids liberates toxic gas. RCH002b For professional users only. Contains TROCLOSENE SODIUM. P271 Use in a well-ventilated area. P273 Avoid release to the environment. P280 Wear eye protection. P337+P313 If eye irritation persists: Get medical advice/attention. P391 Collect spillage. P402+P404 Store in a dry place. Store in a closed container. P501 Dispose of contents/ container in accordance with local regulations.

**dentisan**<sup>®</sup>  
INNOVATION, EDUCATION, COMPLIANCE

Dentisan, Bateman Street,  
Derby DE23 8JL  
t: +44(0)1332 268526  
w: [www.dentisan.co.uk](http://www.dentisan.co.uk)  
e: [info@dentisan.co.uk](mailto:info@dentisan.co.uk)

200

| DS290 |

# DentiChlor tablets

**dentisan**<sup>®</sup>

INNOVATION, EDUCATION, COMPLIANCE

Disinfectant tablets for  
impressions and devices

- measured dose
- rapid action
- easy to use

#### Description

A convenient, reliable and consistent means of producing chlorine solutions for disinfection operations.

#### Directions

For disinfecting impressions and devices, dissolve 1 tablet in 1 litre of water (1000ppm Chlorine), using a Dentisan Impression Bath for convenience with a contact time of 5 minutes. Refer to HTM 01:05 guidance, rinse well before use. Always consult Manufacturer's directions for the material/ item to be disinfected before use to ensure suitability of product solution.

For water storage containers, use 1 tablet per 5 litres of water with a minimum contact time of 10 minutes. Rinse well before use.

For blood/ body fluid spills, use 1 tablet per 100ml of water (10,000ppm Chlorine).

Avoid contact with fabrics. Please refer to product safety datasheet before use. Use only as directed.