Firing paste

REF 1922 0300

Firing paste

REF 1922 0300

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Indikation:

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ENGLISH | INSTRUCTIONS FOR USE

Indication:

The firing paste is a highly viscous, creamy and slightly sticky paste for the gentle holding and stabilization of prostheses (dental restorations like crowns, bridges, veneers etc.) in the ceramic furnace during the glazing process. The product is designed for professional use by dental technicians in the dental lab.

How to use:

Fill the restoration with firing paste. Use the right sized pin for the mesh tray. Apply the pin into the paste inside the restoration. Completely remove small amounts of paste in case it gets in contact to the outer surface of the restoration. Now, position the pin with the restoration as usual in the center of the tray and put the tray into the furnace. Carry out the glazing process according to the Instructions for Use and firing table of the ceramic glazing material and furnace used. The firing paste can withstand temperatures up to 1500 °C (2730 °F). After the firing process, the paste can be removed very easily with a strong stream of air or carefully with a suitable instrument.

Renfert GmbH
Untere Giesswiesen 2
78247 Hilzingen/Germany
Tel. +49 7731 8208-0
www.renfert.com
Made in Germany

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