

Hu-Friedy INSTRUMENT **MANAGEMENT** & INFECTION **PREVENTION**

PRODUCT CATALOG & REFERENCE GUIDE



How the best perform

THE INSTRUMENT MANAGEMENT SYSTEM

It may seem like a small step, but instrument management can have a big impact on your practice. Keeping instruments organized and managing them properly improves the flow of your practice. The sterilization area can run more efficiently, and staff members can remain focused on the patient and providing the best level of care. The INSTRUMENT MANAGEMENT SYSTEM (IMS®) BY HU-FRIEDY keeps instruments organized and protected in cassettes. This allows clinicians to efficiently move instruments from chairside through instrument reprocessing to storage in a safer, more compliant process.

1. Chairside: Patient Preview & Presentation

- Systematically organizes instruments according to procedure type, allowing for more focus on patients and less time spent looking for missing instruments.
- · Neatly organized instruments offer a more professional appearance and can help enhance referrals.

2. Cleaning: Processed & Prepped

- Instruments are kept together throughout cleaning, rinsing and drying, reducing the potential for breakage or loss.
- · Minimizes handling of instruments, reducing the chance for sharps injuries.

3. Sterilization: Wrapped & Ready

- · Less likely to overload the sterilizer, which ensures instruments are properly sterilized.
- Reduced handling of instruments provides safer protocol and minimizes liability.


4. Storage: Neat & Complete

- Ready for use or storage until needed for increased time savings.
- Complete procedural set-ups for easy set-up and teardown.

TABLE OF CONTENTS

The Instrument Management System	2
Why IMS?	4
Points of Performance	5
IMS Infinity Series™ Cassettes	6
Cassette Accessories	11
IMS Collections	13
ADA CDT Coding Collections	21
IMS Signature Series® Cassettes	24
IMS Resin Cassettes	28
Signature Series™ Tubs	29
Organization	31
GreenLight Dental Compliance Center™ by Hu-Friedy	32
Cleaning and Care	34
Surface Disinfection	38
Dental Unit Waterlines	40
Sterilization	42
Hand Hygiene	47
Infection Control Starter Packages	49
Reference	51



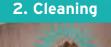











6 I recommend the use of Cassettes to save time, increase efficiency, protect instruments from damage and reduce handling of contaminated instruments

by dental personnel. Cassettes create a well organized system for transporting, cleaning, sterilizing and storing instruments. -LESLIE CANHAM, CDA, RDA

2



HOW IMS SAVES TIME



MORE EFFICIENT

At every dental office, the sterilization center is the central hub. Everything flows in and then flows back out. When reprocessing moves slowly, it creates a bottleneck that impacts the entire office.

It can be easy to overlook the importance of the sterilization center. Though it's often the busiest area of a practice, it doesn't generate any revenue, and the required labor is a large expense.

With IMS, practices can **STREAMLINE REPROCESSING** by eliminating the time-consuming steps that slow down staff. Instruments don't need to be scrubbed by hand and they don't need to be sorted and pouched. That time saved adds up quickly. On average, offices that use IMS see a time **SAVINGS OF 5 TO 10 MINUTES** per procedure.* Over the course of a day, that saves offices an hour or more.

Your practice doesn't have extra time to waste on more timeconsuming processes. With IMS, staff is able to work faster and get more done during the day, instead of working later to wrap up everything that didn't get done.



When presenting at our design courses, we ask attending dentists "Do you know how to sterilize your own instruments?" The response is either a blank stare or a definitive "No I can't." And each time it highlights the

disconnect for dentists in developing an efficient and effective sterilization area within their facility. This disconnect often results in undersized, improperly sequenced and ill-equipped outcomes.

We as designers rely upon and highly recommend that our clients resource Hu-Friedy's expertise in all aspects of instrument processing. In our experience, all dentists can benefit from Hu-Friedy's sterilization observation consult. It is an excellent entry point to identify sterilization issues and provide solutions.

-JEFF & PAT CARTER

*Based on Hu-Friedy market survey results when compared to single instrument reprocessing.



RETURN ON INVESTMENT

When practices invest in IMS, they often see an immediate change, most noticeably in their bottom line. That's because the time saved with IMS adds up throughout the day.

Even a conservative estimate of 5 minutes saved per procedure – most practices **SAVE 5 TO 10 MINUTES PER PROCEDURE**[•] with IMS – means more time to see new patients. That can lead to **TENS OF THOUSANDS OF DOLLARS IN NEW REVENUE**.

Refer to the chart on the right to see the potential growth your practice can see with IMS based on the your current number of procedures and revenue per day.


You can also calculate your potential savings at **Hu-Friedy.com/IMS-calculators**.



As the former CEO of T.H.E. Design, I was involved with the design of thousands of elegant offices,

but few items enhanced the efficiency of the practice like the IMS cassettes by Hu-Friedy. We could create an extremely efficient sterilization area which was smaller by a third when the practice employed IMS cassettes...as opposed to the practice employing trays with bagged instrumentation. ... By utilizing IMS cassettes and an instrument washer, the average practice was saving two hours of labor every day in sterilization. Two big reasons I strongly recommended, and continue to recommend, the use of IMS cassettes.

- MARK THOLEN, DDS, MBA

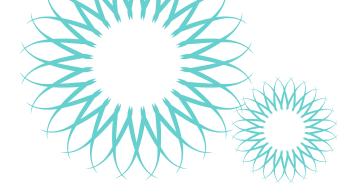

HOW CAN IMS GROW YOUR BUSINESS?

CURRENT		POTENTIAL NEW REVENUE	
PROCEDURES PER DAY	REVENUE PER DAY	PER DAY	PER 6 MONTHS
	\$1,000	\$104	\$11,440
	\$2,000	\$208	\$22,880
10	\$3,000	\$313	\$34,430
	\$4,000	\$417	\$45,870
	\$5,000	\$521	\$57,310
	\$1,000	\$208	\$22,880
	\$2,000	\$417	\$45,870
20	\$3,000	\$625	\$68,750
	\$4,000	\$833	\$91,630
	\$5,000	\$1,042	\$114,620
	\$1,000	\$313	\$34,430
	\$2,000	\$625	\$68,750
30	\$3,000	\$938	\$103,180
	\$4,000	\$1,250	\$137,500
	\$5,000	\$1,563	\$171,930
	\$1,000	\$417	\$45,870
	\$2,000	\$833	\$91,630
40	\$3,000	\$1,250	\$137,500
	\$4,000	\$1,667	\$183,370
	\$5,000	\$2,083	\$229,130
	\$1,000	\$521	\$57,310
	\$2,000	\$1,042	\$114,620
50	\$3,000	\$1,563	\$171,930
	\$4,000	\$2,083	\$229,130
	\$5,000	\$2,604	\$286,440

Example: A practice bills \$2,000 of revenue per day, doing 20 procedures. \$2,000 \div 480 minutes (8 hours) = \$4.17 revenue per minute. 20 procedures x 5 minutes saved per procedure x \$4.17 per minute = \$417.00 of new revenue per day. \$417.00 new revenue per day x 110 days = \$45,870 of new revenue per 6 months.

*Based on Hu-Friedy market survey results when compared to single instrument reprocessing.





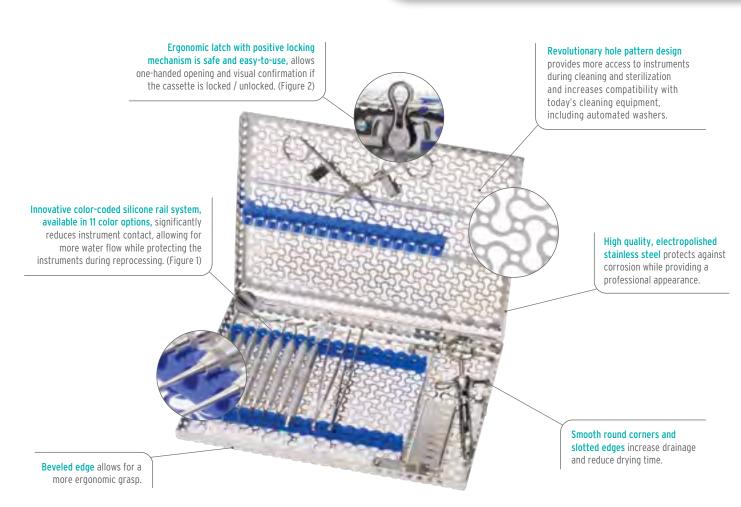




Figure 1



Figure 2



6 The IMS cassette system significantly aids efficiency at chairside and during processing - it is also important for the protection of my staff and our delicate instruments. We've found longevity

of expensive instruments is increased and handling time is decreased.



IMS Infinity Series[™] Cassettes are made of long-lasting, low-maintenance stainless steel. Soft, colored silicone rails securely hold instruments throughout the processing cycle. Available in over 100 different sizes, configurations and color options, cassettes can be specialized to meet your needs.



Exam Cassette - 5 Instruments

Exterior Dimensions 3" x 8" x 1.25" (76 mm x 203 mm x 32 mm) Holds 5 instruments.

Red		IMN6050
Gray		IMN6051
White	\square	IMN6052
Orange		IMN6053
Purple		IMN6054
Yellow		IMN6055
Lavender		IMN6056
Black		IMN6057
Blue		IMN6058
Green		IMN6059
Ocean Blue		IMN60511

Small Cassette - 8 Instruments

Exterior Dimensions 5.5" x 8" x 1.25" (140 mm x 203 mm x 32 mm) Holds 8 instruments with an accessory area.

Red		IMN5080
Gray		IMN5081
White	\square	IMN5082
Orange		IMN5083
Purple		IMN5084
Yellow		IMN5085
Lavender		IMN5086
Black		IMN5087
Blue		IMN5088
Green		IMN5089
Ocean Blue		IMN50811

Small Cassette - 10 Instruments

Exterior Dimensions 5.5" x 8" x 1.25"

(140 mm x 203 mm x 32 mm) Holds 10 instruments.

Red	IMN5100
Gray	IMN5101
White	D IMN5102
Orange	IMN5103
Purple	IMN5104
Yellow	IMN5105
Lavender	IMN5106
Black	IMN5107
Blue	IMN5108
Green	IMN5109
Ocean Blue	IMN51011







IMN416x, 16 Instrument Cassette shown left. Also available in 12 instrument cassette with larger accessory area (IMN412x).



Signa-Stat Cassette - 12 Instruments

Exterior Dimensions 6.5" x 10.5" x 1.25" (165 mm x 267 mm x 32 mm) Holds 12 instruments with an accessory area. Fits in SciCan® STATIM® or Midmark M3 autoclave.

Red		IMN8120
Gray		IMN8121
White	\square	IMN8122
Orange		IMN8123
Purple		IMN8124
Yellow		IMN8125
Lavender		IMN8126
Black		IMN8127
Blue		IMN8128
Green		IMN8129
Ocean Blue		IMN81211

Large Cassette with Accessory Area – 12 or 16 Instruments

Exterior Dimensions 8" x 11" x 1.25" (203 mm x 280 mm x 32 mm) Also available in 8" x 10.5" x 1.25" size (IMN416xQ) to fit unique ultrasonic tanks.

		12 Instrument	16 Instrument
Red		IMN4120	IMN4160
Gray		IMN4121	IMN4161
White	\square	IMN4122	IMN4162
Orange		IMN4123	IMN4163
Purple		IMN4124	IMN4164
Yellow		IMN4125	IMN4165
Lavender		IMN4126	IMN4166
Black		IMN4127	IMN4167
Blue		IMN4128	IMN4168
Green		IMN4129	IMN4169
Ocean Blue		IMN41211	IMN41611

Large Cassette - 20 Instruments

Exterior Dimensions 8" x 11" x 1.25"

(203 mm x 280 mm x 32 mm) Holds 20 instruments.

Red	IMN4200
Gray	IMN4201
White	□ IMN4202
Orange	IMN4203
Purple	IMN4204
Yellow	IMN4205
Lavender	IMN4206
Black	IMN4207
Blue	IMN4208
Green	IMN4209
Ocean Blue	IMN42011







Double-Decker[®] Cassette - 8 Instruments

Exterior Dimensions 3" x 8" x 1.5" (76 mm x 203 mm x 38 mm) Holds 8 instruments.

Red		IMN9080
Gray		IMN9081
White	\square	IMN9082
Orange		IMN9083
Purple		IMN9084
Yellow		IMN9085
Lavender		IMN9086
Black		IMN9087
Blue		IMN9088
Green		IMN9089
Ocean Blue		IMN90811

Double-Decker[®] Cassette - 14 Instruments

Exterior Dimensions 4.5" x 8" x 1.5" (114 mm x 203 mm x 38mm) Holds 14 instruments.

Red	IMN9140
Gray	IMN9141
White	□ IMN9142
Orange	IMN9143
Purple	IMN9144
Yellow	IMN9145
Lavender	IMN9146
Black	IMN9147
Blue	IMN9148
Green	IMN9149
Ocean Blue	IMN91411



Double-Decker® Cassette - 20 Instruments

Exterior Dimensions 6" x 9.7" x 1.5" (152 mm x 246 mm x 38 mm) Holds 20 instruments with an accessory area. Has a built-in needle cap holder and slot for a winged syringe.

Red	IMN9200
Gray	IMN9201
White	□ IMN9202
Orange	IMN9203
Purple	IMN9204
Yellow	IMN9205
Lavender	IMN9206
Black	IMN9207
Blue	IMN9208
Green	IMN9209
Ocean Blue	IMN92011

With Double-Decker Cassettes, instruments are placed in both the lid and base of the cassette and held in place by arms. This compact arrangement allows for better fit in some sterilizers.











Small Ultrasonic Insert/Handpiece Cassette



Large Ultrasonic Insert/Handpiece Cassette



Oral Surgery Cassette

Exterior Dimensions 8" x 14.5" x 1.25" (203 mm x 368 mm x 32 mm) Accommodates 12 instruments, 4 elevators, 3 hinged instruments in the lid and a separate accessory area.

Fits in most 10" chambers or larger sterilizers, most instrument washers and larger ultrasonics.

Red		IMNOSO
Gray		IMNOS1
White	\square	IMNOS2
Orange		IMNOS3
Purple		IMNOS4
Yellow		IMNOS5
Lavender		IMNOS6
Black		IMNOS7
Blue		IMNOS8
Green		IMNOS9
Ocean Blue		IMNOS11

Ultrasonic Insert/Handpiece Cassette

Small: Exterior Dimensions 1.5" x 8" x 1.25" (38 mm x 203 mm x 32 mm)

Large: Exterior Dimensions 3" x 8" x 1.25" (76 mm x 203 mm x 32 mm)

		Small Cassette	Large Cassette
Red		і імизонио	ІМИ60НО
Gray		IMN30HU1	IMN60H1
White	\square	IMN30HU2	IMN60H2
Orange		ІМИЗОНИЗ	IMN60H3
Purple		IMN30HU4	IMN60H4
Yellow		IMN30HU5	IMN60H5
Lavender		IMN30HU6	IMN60H6
Black		IMN30HU7	IMN60H7
Blue		IMN30HU8	IMN60H8
Green		і імизония	ІМN60Н9
Ocean Blue		IMN30HU11	IMN60H11

Small holds 2 Ultrasonic Inserts or 1 handpiece.

Large holds up to 3 high-speed handpieces or 5 ultrasonic inserts.

Radiography Cassette

Exterior Dimensions 5.5" x 8" x 1.25" (140 mm x 203 mm x 32 mm)

Holds heat sterilizable x-ray accessories

Always follow manufacturer's instructions for use for reprocessing of sensor holders.

| IMN7000



IMS INFINITY SERIES[™] **CASSETTES**



66 As an orthodontic clinical consultant, I work with offices to increase their efficiency. Implementing IMS into their practice has been a huge benefit to many successful practices across the country. The clinicians love the system and doctors feel the stress level decrease and production increase. The patients' perception of the high quality care they receive is of great internal marketing value as well.

-ANDREA COOK







Large Thin Orthodontic Cassette

Exterior Dimensions 8" x 11" x 0.75" (203 mm x 280 mm x 19 mm) Holds up to 7 pliers and 4 hand instruments

Purple	IMNORTHL4
Blue	IMNORTHL8
Green	IMNORTHL9

Small Thin Orthodontic Cassette

Exterior Dimensions 7" x 8" x 0.75" (178 mm x 203 mm x 19 mm) Holds up to 4 pliers and 3 hand instruments

Purple	IMNORTHS4
Blue	IMNORTHS8
Green	IMNORTHS9

Double-Decker[®] Small Orthodontic Cassette

Exterior Dimensions 4.5" x 8" x 1.5" (114 mm x 203 mm x 38 mm) Holds up to 3 pliers, a Mathieu needle holder and 2 hand instruments

Red	IMN9140-0
Gray	IMN9141-0
White	□ IMN9142-0
Orange	IMN9143-0
Purple	IMN9144-0
Yellow	IMN9145-0
Lavender	IMN9146-0
Black	IMN9147-0
Blue	IMN9148-0
Green	MN9149-0
Ocean Blue	IMN91411-0

CASSETTE ACCESSORIES



IMS PARTS BOX

Designed to keep small items such as rubber dam clamps and matrix band retainers organized during the sterilization process.

IMS Large Parts Box

| IMS-1273 Exterior Dimensions: 3.5" x 1.12" x 1" (89 mm x 29 mm x 25 mm)

IMS Small Parts Box

| IMS-1271 Exterior Dimensions: 1.56" x 1.25" x 1" (40 mm x 32 mm x 25 mm)

IMS CASSETTE BARCODE CLIP

Snap-on Barcode for outside of Cassettes

IMS Cassette Barcode Clip

IMS-CABL

Dimensions: .75" x 2.26" x .067" (19 mm x 57.4 mm x 1.7 mm)

Contains lasermarked barcode to be used for identification and tracking purposes. Compatible with standard barcode reader. Follows HIBC barcode standards. Lasermark tested to last more than 800 sterilization cycles.

Signature Series Barcode Plate

Metal plate can have barcode label affixed or can be lasermarked with text. Barcodes cannot be lasermarked onto the metal.

IMS-1378

Dimensions: .88" x 1.88" x .047" (22 mm x 48 mm x 1.2 mm) For Signature Series cassettes only. Installed by Hu-Friedy.

Infinity Series Barcode Plate | IMS-1379

Dimensions: .88" x 1.75" x .047" (22 mm x 44 mm x 1.2mm) For Infinity Series cassettes only. Snaps into place.

AIR WATER SYRINGE CLIPS - RESIN

Designed to fit large or small Resin

tip securely in place throughout cleaning and sterilization.

For small two-tier resin cassettes

IMS Resin Cassette Clip

| IMS-1278 For large resin cassettes IMS Resin Cassette Clip

| IMS-1279

CASSETTES







IMS-1278

IMS ENDODONTIC STAND

Hinged, double-lid stand holds endodontic files and reamers. Lid is etched with a ruler to measure stop positions easily from either direction.

IMS Endodontic Stand, Holds 24

| IMS-1275

Exterior Dimensions: 2.6" x 2.25" x 1.2" (67 mm x 57 mm x 30 mm)

IMS Endodontic Stand, Holds 48 IMS-1275D

Exterior Dimensions: 2.6" x 2.25" x 1.2" (67 mm x 57 mm x 30 mm)



IMS-1275

AIR AND WATER CLIPS

Designed for IMS Cassettes. Air and Water clips can be permanently installed in your cassettes.

Infinity	Series	Air/Water	Clip	IM1005N

Signature Series Air/Water Clip | IM1005

Infinity and Signature Series I IM1006 Double-Decker Air/Water Clip



Complete list of cassette replacement parts can be found on Hu-Friedy.com/IMS-Replacement-Parts





IMS BUR CUSHIONS

A hollow, rectangular, silicone base with a stainless steel lid that is designed to hold burs securely in place during cleaning and sterilization. Hollow design allows drainage and proper cleaning.



IMS 1/2 Bur Cushion, Short Lid IMS-1372SH

Holds 6 burs. Exterior Dimensions: 1.75" x 1.3" x 0.9" (44 mm x 33 mm x 22 mm)

IMS 1/2 Bur Cushion, Tall Lid IMS-1372TH

Holds 6 burs. Exterior Dimensions: 1.75" x 2.44" x 0.9" (44 mm x 62 mm x 22 mm)



IMS Bur Cushion, Short Lid

Holds 12 burs. Exterior Dimensions: 3.3" x 1.3" x 0.9" (84 mm x 33 mm x 22 mm

IMS Bur Cushion, Tall Lid

Holds 12 burs. Exterior Dimensions: 3.3" x 2.44" x 0.9" (84 mm x 62 mm x 22 mm)

IMS RUBBER DAM CLAMP ORGANIZING BOARD

RDC Organizing Board | RDCOB Dimensions: 4.6" x 2.4" x .375"

(117 mm x 61 mm x 10 mm)

Holds up to 8 clamps during sterilization. Fits into cassette accessory area of most cassettes.



not actual size

HINGED INSTRUMENT CLIPS

Secure scissors, hemostats and other hinged instruments in the open position for optimal cleaning and sterilization.

Hinged Instrument Clips, 2 Pack

Hinged Instrument Clips, 6 Pack



NEEDLE CAP HOLDER

Needle Cap Holders, 2 Pack

Needle Cap Holders, 6 Pack

Combination Pack**

**6 snap-in needle cap holders and 6 snap-in hinged instrument clips. Maintains cap in upright position for one-handed needle recapping.



IMS COLLECTIONS



The IMS Infinity Series Collection and IMS Signature Series Collection feature a compilation of clinicians' most popular procedural set-ups. Order a Collection today and receive high quality Hu-Friedy instruments and IMS cassettes together at significantly reduced prices. Create a professional, efficient working environment with the IMS Infinity Series Collection.



Perio – Double-Decker®

	IMNPERIO	 IMPERIO Signature Series
14 Instrument Cassette, Blue	IMN9148	IM9148
Satin Steel® Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
11/12 Old Dominion University After Five Explorer, Satin Steel Handle	EXD11/12A6	EXD11/12A6
CP-12 Color-Coded Probe, Satin Steel Handle	PCP126	PCP126
EverEdge 2.0 H6/H7 Hygienist Scaler	SH6/79E2	SH6/79E2
EverEdge 2.0 204S Sickle Scaler	S204S9E2	S204S9E2
EverEdge 2.0 Mini Five Gracey Curette	SAS1/29E2	SAS1/29E2
EverEdge 2.0 #7/8 After5 Gracey Curette, EE2 #9 handle	SRPG7/89E2	SRPG7/89E2
EverEdge 2.0 11/12 Gracey Curette	SRPG11/129E2	SRPG11/129E2
EverEdge 2.0 13/14 After Five Gracey Curette	SRPG13/149E2	SRPG13/149E2
EverEdge 2.0 4R/4L Columbia University Curette	SC4R/4L9E2	SC4R/4L9E2
#3 Ceramic Stone	SS3C	SS3C
A/W Syringe Tip Clip	IM1006	IM1006

Prophy

	IMNPROPHY Infinity Series	 IMPROPHY Signature Series
Small Infinity Series 10 Instrument Cassette, Lavender	IMN5106	IM5106
Satin Steel Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
11/12 Old Dominion University Explorer, Satin Steel Handle	EXD11/126	EXD11/126
CP-12 Color-Coded Probe, Satin Steel Handle	PCP126	PCP126
EverEdge 2.0 H6/H7 Hygienist Scaler	SH6/79E2	SH6/79E2
EverEdge 2.0 11/12 Gracey Curette	SG11/129E2	SG11/129E2
EverEdge 2.0 13/14 Gracey Curette	SG13/149E2	SG13/149E2
EverEdge 2.0 13/14 Columbia University Curette	SC13/149E2	SC13/149E2
3C Ceramic Sharpening Stone	SS3C	SS3C
A/W Syringe Tip Clip	IM1005N	IM1005

Signature Series

These collections may be ordered using the collection part code bolded at the top of the list (ex. IMNPERIO).



IMS COLLECTIONS



6 Hu-Friedy Cassettes keep your set-ups together as a unit, so instead of processing hundreds of single instruments, you process fewer, larger items. The economy of time and effort is undeniable, but wait until you see the increased safety to go along with

the efficiency! Sharp instruments are organized for safer, easier use, and contained for safer handling. Cassettes with needle cap holders also become an engineered safety device for re-capping.

-NANCY DEWHIRST, RDH, BS





Exam	IMNEXAM	 IMEXAM Signature Series
Exam Infinity Series 5 Instrument Cassette, Yellow	IMN6055	IM6055
Satin Steel® Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
5 Explorer, Satin Steel Handle	EXD56	EXD56
CP-12 Color-Coded Probe, Satin Steel Handle	PCP126	PCP126
U17 Utility Pick-Up Dressing Pliers	DPU17	DPU17



Infinity Series



Signature Series

Sinus Lift	IMNSINUS Infinity Series	 IMSINUS Signature Series
Large Infinity Series 12 Instrument Cassette, Gray	IMN4121	IM4121
2-Pack Signature Series Hinged Instrument Clips	IM1000	IM1000
Buser Periosteal, Satin Steel Handle	PPBUSER6	PPBUSER6
Palti Bone Graft Scoop	IMP6522	IMP6522
Bone Tamper 7 mm	BONT	BONT
Bone Rasp	RASP	RASP
Sinus Retractor Misch 10	SRM10	SRM10
Sinus Curette 1	SINC1	SINC1
Sinus Curette 2	SINC2	SINC2
Bone Grafting Syringe, 7 mm	SYBG	SYBG
#1 Mead Mallet	MAL1	MAL1
Crile-Wood Perma Sharp Needle Holder, 15 cm (6'')	NH5038	NH5038
#2/4 Molt Curette	CM2/4	CM2/4
#UNC 12 Colorvue Kit	PCVNCKIT6	PCVNCKIT6
#23 Seldin Periosteal Retractor	P23	P23
Bone Packer 6 mm/8 mm	BPACK	BPACK





Infinity Series





Infinity Series



Suture Removal	IMNSUTURE Infinity Series	 IMSUTURE Signature Series
Small Infinity Series 10 Instrument Cassette, Orange	IMN5103	IM5103
Satin Steel Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
5 Explorer, Satin Steel Handle	EXD56	EXD56
EverEdge 2.0 13/14 Columbia University Curette	SC13/149E2	SC13/149E2
2 College Dressing Pliers	DP2	DP2
13S Suture Scissors	135	135
A/W Syringe Tip Clip	IM1005N	IM1005
Hinged Instrument Clips, 2	IM1000	ІМ1000

Total Restorative – Composite, Crown & Bridge, Amalgam

	l a briago, rana	
	IMNRESTOR	IMRESTOR Signature Series
	I IMN4165	I IM4165
Large Infinity Series 16 Instrument Cassette, Yellow	IMIN4105	IM4105
Satin Steel Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
23/CP-12 Color-Coded Expro, Satin Steel Handle	XP23/126	XP23/126
18 Excavator, Satin Steel Handle	EXC186	EXC186
113 Serrated Gingival Cord Packer, Satin Steel Handle	GCP1136	GCP1136
Calcium Hydroxide Placement Instrument, Satin Steel Handle	PICH6	PICH6
0/1 Marquette Condenser, Satin Steel Handle	PLG0/16	PLG0/16
1/2 Blk Condenser, Satin Steel Handle	PLG1/26	PLG1/26
Regular/Large CFII Amalgam Carrier	AC5202	AC5202
3/6 Discoid-Cleoid Carver, Satin Steel Handle	CD3/66	CD3/66
8 Wiland Carver, Satin Steel Handle	CVWI86	CVWI86
1/2-3 Hollenback Carver, Satin Steel Handle	CVHL1/26	CVHL1/26
IPC Interproximal XTS Carver	TNCVIPC	TNCVIPC
2 Goldstein Flexi-Thin XTS® Composite Instrument	TNCIGFT2	TNCIGFT2
3 Goldstein Flexi-Thin XTS Composite Instrument	TNCIGFT3	TNCIGFT3
27/29 Burnisher, Satin Steel Handle	BB27/296	BB27/296
24 Spatula, Satin Steel Handle	CS246	CS246
2 College Dressing Pliers	DP2	DP2
Miller Articulating Paper Forceps	APF2	APF2
Curved Iris Scissors	S18	S18
CW Aspirating Anesthetic Syringe	SYRCW	SYRCW
Amalgam Well	WA	WA
Bur Cushion Short Lid, Holds 12	IMS-1372S	IMS-1372S
Hinged Instrument Clips, 2	IM1000	IM1000
A/W Syringe Tip Clip	IM1005N	IM1005





Signature Series

Cement (Crown Seat)	IMNCEMENT	 IMCEMENT Signature Series
Small Infinity Series 10 Instrument Cassette, Red	IMN5100	IM5100
Satin Steel® Mirror Handle	МН6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
5 Explorer, Satin Steel Handle	EXD56	EXD56
8 Wiland Carver, Satin Steel Handle	CVWI86	CVW186
24 Spatula, Satin Steel Handle	CS246	CS246
Miller Articulating Paper Forceps	APF2	APF2
CW Aspirating Anesthetic Syringe	SYRCW	SYRCW
2 College Dressing Pliers	DP2	DP2
Bur Cusion Short Lid, Holds 6	IMS-1372SH	IMS-1372SH
A/W Syringe Tip Clip	IM1005N	IM1005



Infinity Series



Crown and Bridge	IMNCROWN	•
	Infinity Series	Signature Series
Large Infinity Series 16 Instrument Cassette, Green	IMN4169	IM4169
Satin Steel [®] Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
5 Explorer, Satin Steel Handle	EXD56	EXD56
CP-12 Color-Coded Probe, Satin Steel Handle	PCP126	PCP126
8 Wiland Carver, Satin Steel Handle	CVWI86	CVWI86
18 Excavator, Satin Steel Handle	EXC186	EXC186
113 Serrated Gingival Cord Packer, Satin Steel Handle	GCP1136	GCP1136
2 Goldstein Flexi-Thin XTS® Composite Instrument	TNCIGFT2	TNCIGFT2
24 Spatula, Satin Steel Handle	CS246	CS246
2 College Dressing Pliers	DP2	DP2
Miller Articulating Paper Forceps	APF2	APF2
CW Aspirating Anesthetic Syringe	SYRCW	SYRCW
18 Curved Iris Scissors	S18	S18
Bur Cusion Short Lid, Holds 12	IMS-1372S	IMS-1372S
Hinge Instrument Clips, 2	IM1000	IM1000
A/W Syringe Tip Clip	IM1005N	IM1005






6 Using an instrument management system like Hu-Friedy's IMS facilitates processing and can substantially enhance instrument organization. It is essential to keep tasks as organized and safe for clinical personnel. Because direct handling of contaminated instruments is significantly

reduced before sterilization, personnel responsible for instrument cleaning sterilization are at much less risk for sharps injuries.

-JOHN MOLINARI, PhD

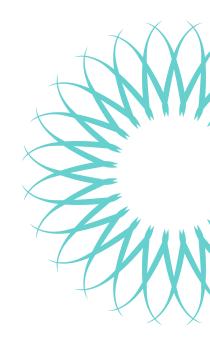



Infinity Series



Signature Series

Amalgam		IMAMALGAM
	Infinity Series	Signature Series
Large Infinity Series 16 Instrument Cassette, White	IMN4162	IM4162
Satin Steel Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
5 Explorer, Satin Steel Handle	EXD56	EXD56
18 Excavator, Satin Steel Handle	EXC186	EXC186
Calcium Hydroxide Placement Instrument, Satin Steel Handle	PICH6	PICH6
Regular/Large CFII Amalgam Carrier	AC5202	AC5202
0/1 Marquette Condenser, Satin Steel Handle	PLG0/16	PLG0/16
1/2 Black Condenser, Satin Steel Handle	PLG1/26	PLG1/26
3/6 Discoid-Cleoid Carver, Satin Steel Handle	CD3/66	CD3/66
89/92 Discoid-Cleoid Carver, Satin Steel Handle	CD89/926	CD89/926
1/2-3 Hollenback Carver, Satin Steel Handle	CVHL1/26	CVHL1/26
21B Burnisher, Satin Steel Handle	BB21B6	BB21B6
27/29 Burnisher, Satin Steel Handle	BB27/296	BB27/296
2 College Dressing Pliers	DP2	DP2
Miller Articulating Paper Forceps	APF2	APF2
CW Aspirating Anesthetic Syringe	SYRCW	SYRCW
Amalgam Well	WA	WA
Bur Cushion Short Lid, holds 12	IMS-1372S	IMS-1372S
A/W Syringe Tip Clip	IM1005N	IM1005





Infinity Series




Composite/Crown Prep	 IMNCMPOSIT Infinity Series	IMCOMPOSIT Signature Series
Large Infinity Series 16 Instrument Cassette, Blue	IMN4168	IM4168
Satin Steel Mirror Handle	MH6	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3	MIR5/3
5 Explorer, Satin Steel Handle	EXD56	EXD56
18 Excavator, Satin Steel Handle	EXC186	EXC186
113 Serrated Gingival Cord Packer, Satin Steel Handle	GCP1136	GCP1136
IPC Interproximal XTS® Carver	TNCVIPC	
2 Goldstein Flexi-Thin XTS Composite Instrument	TNCIGFT2	TNCIGFT2
3 Goldstein Flexi-Thin XTS Composite Instrument	TNCIGFT3	TNCIGFT3
24 Spatula, Satin Steel Handle	CS246	CS246
2 College Dressing Pliers	DP2	DP2
Miller Articulating Paper Forceps	APF2	APF2
CW Aspirating Anesthetic Syringe	SYRCW	SYRCW
18 Curved Iris Scissors	S18	S18
Bur Cushion Short Lid, holds 12	IMS-1372S	IMS-1372S
Hinge Instrument Clips, 2	IM1000	IM1000
A/W Syringe Tip Clip	IM1005N	IM1005



66 Worker and patient safety are my passion! During presentations I am often asked for practical advice on how to reduce the risk of sharps injury for workers and also how to save time while safely processing instruments. With the Hu-Friedy IMS cassette system both issues are easily addressed. Securing contaminated instruments immediately

reduces the risk of sharps exposure creating a safer work environment. Efficiencies are gained through utilization of the IMS system which organizes instruments based on the procedure to be performed, eliminating the need to hand organize instrument. This action alone reduces operatory turn over time.

-KAREN GREGORY, RN

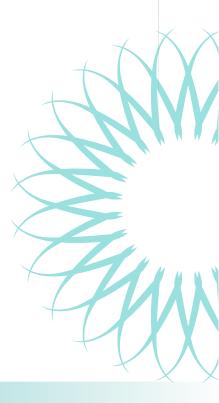






66 As a former practicing dentist, my favorite part of IMS was not searching for missing instruments! The complete instrument setup is always in the cassette so we were more organized and efficient throughout the day. Not only did this save the practice money, but it also allowed team members to spend more time with patients. Most

importantly, I felt confident knowing team members involved in instrument processing had a safer work environment with less risk of injury.

– Dr. Marie Fluent, DDS

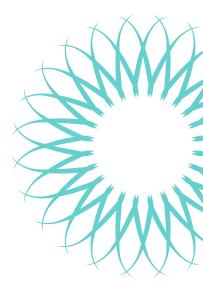



Infinity Series



Signature Series

Endo	 IMNENDO Infinity Series	 IMENDO Signature Series
Large Infinity Series 12 Instrument Cassette, Gray	IMN4121	IM4121
Satin Steel Mirror Handle	MH6	MH6
4 Front Surface Mouth Mirror, 3 pack	MIR4/3	MIR4/3
DG16 Endo Explorer, Satin Steel Handle	EXDG166	EXDG166
1 Locking Endo Pliers	EPL1	EPL1
1 Locking Endo Pliers	EPL1	EPL1
31L Excavator, Satin Steel Handle	EXC31L6	EXC31L6
D11 Nickel Titanium Endo Spreader	RCSD11NT	RCSD11NT
1/3 Endo Plugger	RCP1/3	RCP1/3
5/7 Endo Plugger	RCP5/7	RCP5/7
9/11 Endo Plugger	RCP9/11	RCP9/11
1 Glick Blade/Endo Plugger	RCPGL1	RCPGL1
Miller Articulating Paper Forceps	APF2	APF2
CW Aspirating Anesthetic Syringe	SYRCW	SYRCW
Rubber Dam Forceps	RDF	RDF
1 Satin Steel Rubber Dam Clamp	RDCM1	RDCM1
2 Satin Steel Rubber Dam Clamp	RDCM2	RDCM2
9 Satin Steel Rubber Dam Clamp	RDCM9	RDCM9
26N Satin Steel Rubber Dam Clamp	RDCM26N	RDCM26N
Endo Stand, Holds 24 Files and Reamers	IMS-1275	IMS-1275
Bur Cushion Short Lid, Holds 12	IMS-1372S	IMS-1372S
IMS Small Parts Box	IMS-1271	IMS-1271



IMS COLLECTIONS




6 Orthodontic practices clean and sterilize 600 to 1,000 instruments everyday! Cleaning and sterilizing this many instruments while treating patients can create a lot of stress for busy orthodontic teams. Hu-Friedy cassettes have

helped my orthodontic clients improve clinical efficiency in addition to protecting orthodontic pliers during the cleaning and sterilization process. Improved efficiency gives the orthodontic team more time to focus on patient care and stay on schedule! Happy Staff + Happy Patients = Happy Doctor!





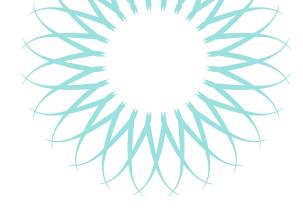

Ortho Banding/Bonding

Double-Decker 20 Instrument Orthodontic Cassette, Orange	IM5203-0R
Satin Steel Mirror Handle	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3
LD1/BPS2 Ligature Band Pusher	678-912
1/2-3 Hollenback Carver, Satin Steel Handle	CVHL1/26
#18 Dressing Pliers	DP18
Universal Cut and Hold Distal End Cutter	678-101L
Micro-Mini Pin and Ligature Cutter	678-107
Weingart Pliers	678-201L
Direct Bond and Bracket Holder	678-212
Mathieu Pliers	678-330

Available in Signature Series only

Ortho Archwire Adjustment

Double Decker 20 Instrument Orthodontic Cassette, Green	IM5209-0R
Satin Steel Mirror Handle	MH6
5 Front Surface Mouth Mirror, 3 pack	MIR5/3
Ligature Director/Scaler	678-905
Universal Cut and Hold Distal End Cutter	678-101L
15º Hard Wire Cutter	678-103
Micro-Mini Pin and Ligature Cutter	678-107
Weingart Pliers	678-201L
Direct Bond and Bracket Holder	678-212
Light Wire Bird Beak Pliers	678-304
Three Jaw Pliers	678-302
Mathieu Pliers	678-330
Available in Signature Series only	



Hu-Friedy has created a collection of procedural instrument kits that corresponds to **American Dental Association (ADA) – Current Dental Terminology (CDT)** Coding. Start pairing patients and insurance codes more effectively to move from standardized care to personalized care.

Begin each preventative appointment with an Evaluation Kit and add the supplemental treatment kit based on the patient assessment. Each kit, uniquely color-coded, offers the highest quality, technology, and ergonomic instrumentation within a small, space-efficient cassette for ensured equipment compatibility and safety. Plastic Test Sticks are included to test for sharpness prior to utilizing instrumentation to provide ergonomic, efficient and quality care to your patients. ADA CDT Coding kits can be ordered easily using a single part code at a significantly reduced bundled cost.







Evaluation Kit

Blue Small 5 Instrument Exam Cassette	IM6058
#6 Satin Steel Mirror Handle	MH6
5 Mirror Head, Double Sided High Definition	MIR5DSHD
#5 DE Explorer in Satin Steel Handle	EXD56
UNC Probe in Satin Steel Handle	PCPUNC126
#11/12 ODU DE Explorer in Satin Steel Handle	EXD11/12A6
Nabors Probe 3-6-9-12	PQ2N6

Available in Signature Series only

Prophylaxis Kit

Exam 5 Instrument Cassette, Green	IM6059
Plastic Test Stick – Individual	PTS-IND
Nevi 2 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI29E2
Nevi 4 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI49E2
5/6 Barnhart Universal Curette, DE EverEdge 2.0 #9 Handle	SBH5/69E2
15/16 Gracey Curette, DE EverEdge 2.0 #9 Handle	SG15/169E2
13/14 Gracey Curette, DE EverEdge 2.0 #9 Handle	SG13/149E2
Available in Signature Series only	

Available in Signature Series only

Pedo Prophy Kit

•	
IMS Exam Cassette, 5 Instruments, Lavender	IM6056
Mirror Handle, SE Satin Steel #6 Handle	MH6
5 Mirror Head, Double Sided High Definition	MIR5DSHD
#5 DE Explorer in Satin Steel Handle	EXD56
13/14 Columbia Universal Curette, DE EverEdge 2.0 #9 Handle	SC13/149E2
Nevi 3 Posterior Sickle Scaler, DE Ever Edge 2.0 #9 Handle	SCNEVI39E2
Nevi 1 Anterior Sickle Scaler, DE Ever Edge 2.0 #9 Handle	SCNEVI19E2
Plastic Test Stick – Individual	PTS-IND

Available in Signature Series only





Scaling and Root Planing Kit

Double Decker Cassette, 8 Instruments, Red	IM9080
Plastic Test Stick – Individual	PTS-IND
3/4 After 5 Langer Universal Curette, DE EverEdge 2.0 #9 Handle	SL3/4AF9E2
Nevi 2 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI29E2
Nevi 4 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI49E2
1/2 Mini 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SAS1/29E2
7/8 After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRPG7/89E2
15/16 After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRPG15/169E2
13/14 After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRPG13/149E2
17/18 Gracey Curette, DE Ever Edge 2.0 #9 Handle	SG17/189E2

Presence of Gingival Inflammation Kit

Exam 5 Instrument Cassette, Yellow	IM6055
Plastic Test Stick – Individual	PTS-IND
Nevi 3 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI39E2
Nevi 4 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI49E2
1/2 Mini 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SAS1/29E2
15/16 After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRPG15/169E2
13/14 After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRPG13/149E2
Available in Signature Series only	



Debridement for Comprehensive Evaluation Kit

Exam 5 Instruments Cassette, Orange	IM6053
Plastic Test Stick – Individual	PTS-IND
Nevi 4 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI49E2
17/18 McCalls Universal Curette, DE EverEdge 2.0 #9 Handle	SM17/189E2
1/2 Rigid Mini 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SAS1/2R9E2
15/16 Rigid After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRP15/16R9E2
13/14 Rigid After 5 Gracey Curette, DE EverEdge 2.0 #9 Handle	SRP13/149RE2

Available in Signature Series only





66 Hu-Friedy IMS instrument cassettes have made a major difference in my mobile practice of Dental Hygiene and my infection control practices. As a mobile dental hygiene business owner and infection control specialist, I have invested a lot into my instruments. These simple, durable

devices are so easy to use and made a complex job become trouble-free. This product has been one of the greatest investments I have made into my practice. They simplify my life, save time and frustration. Every practice should be using this time and life saving device!

- NOEL BRANDON KELSCH, RDH



Periodontal Maintenance Kit

Double Decker Cassette, 8 Instruments, Gray	IM9081
Plastic Test Stick – Individual	PTS-IND
3/4 Langer Universal Curette, DE EverEdge 2.0 #9 Handle	SL3/49E2
Nevi 2 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI29E2
Nevi 4 Posterior Sickle Scaler, DE EverEdge 2.0 #9 Handle	SCNEVI49E2
1/2 Pattison Gracey LITE Curette, DE EverEdge 2.0 #9 Handle	SG1/29APE2
7/8 Pattison Gracey LITE Curette, DE EverEdge 2.0 #9 Handle	SG7/89APE2
11/12 Pattison Gracey LITE Curette, DE EverEdge 2.0 #9 Handle	SG11/129APE2
13/14 Pattison Gracey LITE Curette, DE EverEdge 2.0 #9 Handle	SG13/149APE2
#15/16 Gracey Curette, EverEdge 2.0 #9 Handle	SG15/169E2
Available in Signature Series only	

Implant Maintenance Kit

Exam 5 Instrument Cassette, White	IM6052
Colorvue Probe, UNC 12, SE Satin Steel Handle	PCVPROMONC
Titanium Implant Scaler, 204SD Sickle Scaler	IMP204SDT
Titanium Implant Scaler, Langer 1/2 Universal Curette	IMPLG1/2T
Titanium Implant Scaler, 1/2 Gracey Mini Five Curette	IMPM1/2T
Implacare II Implant Scaler, H6/7 Sickle Scaler	IMPLH6/76
Plastic Test Stick – Individual	PTS-IND

Available in Signature Series only





16 instrument cassette shown above. Also available in 12 and 20 instrument cassettes.



Small Ultrasonic Insert/Handpiece Cassette



Large Ultrasonic Insert/Handpiece Cassette



Double Symmetry IQ® Handpiece Cassette



Large Cassette-12, 16 and 20 Instruments

Exterior Dimensions 8" x 11" x 1.25" (203 mm x 280 mm x 32 mm) Holds 16 instruments with an accessory area.

Also available in 8" x 10.5" x 1.25" size (IM416xQ) to fit unique ultrasonic tanks.

		12 Instrument	16 Instrument	20 Instrument
Red		IM4120	IM4160	IM4200
Gray		IM4121	IM4161	IM4201
White	\square	IM4122	IM4162	IM4202
Orange		IM4123	IM4163	IM4203
Yellow		IM4125	IM4165	IM4205
Lavender		IM4126	IM4166	IM4206
Blue		IM4128	IM4168	IM4208
Green		IM4129	IM4169	IM4209

Ultrasonic Insert/Handpiece Cassette

Small: Exterior Dimensions 1.5" x 8" x 1.25" (38 mm x 203 mm x 32 mm)

Large: Exterior Dimensions 3" x 8" x 1.25" (76 mm x 203 mm x 32 mm)

		Small Cassette	Large Cassette
Red		ІМЗОНИО	ІМ60Н0
Gray		IM30HU1	IM60H1
White	\square	IM30HU2	IM60H2
Orange		IM30HU3	IM60H3
Yellow		IM30HU5	IM60H5
Lavender		IM30HU6	ІМ60Н6
Blue		IM30HU8	IM60H8
Green		IM30HU9	ІМ60Н9

Small holds 2 Ultrasonic Inserts or 1 handpiece.

Large holds up to 3 high-speed handpieces or 5 ultrasonic inserts.

Piezo Handpiece Cassette

Small: Exterior Dimensions 1.5" x 8" x 1.25" (38 mm x 203 mm x 32 mm)

Large: Exterior Dimensions 3" x 8" x 1.25" (76 mm x 203 mm x 32 mm)

Single	IM30IQ8
Double (pictured)	IM60IQ8

IMIQ38

Piezo Wrench Cassette

Exterior Dimensions 5.63" x 1.95" x 1.77" (143 mm x 50 mm x 45 mm)

Blue

Wrench not included.











Oral Surgery Cassette

Exterior Dimensions 8" x 14.5" x 1.25" (203 mm x 368 mm x 32 mm) Accommodates 12 instruments, 4 elevators, 3 hinged instruments in the lid and a separate accessory area.

Fits in most 10" chambers or larger sterilizers, most instrument washers and larger ultrasonics.

Red		IMOSO
Gray		IMOS1
White	\square	IMOS2
Orange		IMOS3
Yellow		IMOS5
Lavender		IMOS6
Blue		IMOS8
Green		IMOS9

Double-Decker[®] Cassette - 8 Instruments

Exterior Dimensions 3" x 8" x 1.5" (76 mm x 203 mm x 38 mm) Holds 8 instruments.

Red		IM9080
Gray		IM9081
White	\square	IM9082
Orange		IM9083
Yellow		IM9085
Lavender		IM9086
Blue		IM9088
Green		IM9089

Double-Decker[®] Cassette - 14 Instruments

Exterior Dimensions 4.5" x 8" x 1.5" (114 mm x 203 mm x 38 mm) Holds 14 instruments.

		14 Instrument	Small Ortho Cassette
Red		IM9140	IM9140-0R
Gray		IM9141	IM9141-0R
White	\square	IM9142	IM9142-0R
Orange		IM9143	IM9143-0R
Yellow		IM9145	IM9145-0R
Lavender		IM9146	IM9146-0R
Blue		IM9148	IM9148-0R
Green		IM9149	IM9149-OR

Double-Decker® Cassette - 20 Instruments

Exterior Dimensions 6" x 9.7" x 1.5" (152 mm x 246 mm x 38 mm) Holds 20 instruments with an accessory area. Has a built-in needle cap holder and slot for a winged syringe.

Red	■ IM9200
Gray	IM9201
White	D IM9202
Orange	IM9203
Yellow	📃 🛛 ім9205
Lavender	IM9206
Blue	IM9208
Green	IM9209



IMS SIGNATURE SERIES[®] CASSETTES



Large Thin Orthodontic Cassette

Small Thin Orthodontic Cassette





Large Thin Orthodontic Cassette

Exterior Dimensions $8" \times 11" \times 0.75"$ (203mm x 280mm x 19mm) Holds up to 7 pliers and 4 hand instruments.



Small Thin Orthodontic Cassette

Exterior Dimensions 7" x 8" x 0.75" (178mm x 203mm x 19mm) Holds up to 4 pliers and 3 hand instruments. Holds up to 7 pliers and 4 hand instruments.

Blue

Blue

Double-Decker[®] Small Orthodontic Cassette

Exterior Dimensions 4.5" x 8" x 1.5" (114mm x 203mm x 38mm) Holds up to 3 pliers, 1 Mathieu needle holder and 2 hand instruments.

Red		IM9140-0R
Gray		IM9141-OR
White	\square	IM9142-OR
Orange		IM9143-0R
Yellow		IM9145-OR
Purple		IM9146-0R
Blue		IM9148-0R
Green		IM9149-OR

Double-Decker® Large Orthodontic Cassette

Exterior Dimensions 6" x 8" x 1.5" (152mm x 203mm x 38mm) Holds up to 6 pliers, 1 Mathieu needle holder and 3 hand instruments without accessory area.

Red	IM5200-OR
Gray	IM5201-OR
White	□ IM5202-OR
Orange	IM5203-OR
Yellow	IM5205-OR
Purple	IM5206-OR
Blue	IM5208-OR
Green	IM5209-OR

*Double-Decker Large Ortho Cassette with Accessory Area - IM920x-OR



66 IMS Cassettes are, in my opinion, the most important efficiency tool for each dental procedure. Handling one cassette verses all the individual instruments for each procedure saves so much time during set-up, clean-up and reprocessing. If dental practices want to enhance productivity and efficiency,

they simply must implement the IMS cassette system. 🌒 🌒

-MARY GOVONI, CDA, RDA, RDH, MBA





Exam Cassette - 5 Instruments

Exterior Dimensions 3" x 8" x 1.25" (76 mm x 203 mm x 32 mm) Holds 5 instruments.

Red	■ IM6050
Gray	IM6051
White	D IM6052
Orange	IM6053
Yellow	🗩 IM6055
Lavender	IM6056
Blue	IM6058
Green	■ IM6059



8 instrument cassette shown above. Also available in 10 instrument cassette with no accessory area.



Small Cassette - 8 and 10 Instruments

Exterior Dimensions 5.5" x 8" x 1.25" (140 mm x 203 mm x 32 mm) Holds 8 instruments with an accessory area.

		8 Instrument	10 Instrument
Red		IM5080	IM5100
Gray		IM5081	IM5101
White	\square	IM5082	IM5102
Orange		IM5083	IM5103
Yellow		IM5085	IM5105
Lavender		IM5086	IM5106
Blue		IM5088	IM5108
Green		IM5089	IM5109

Signa-Stat Cassette - 12 Instruments

Exterior Dimensions 6.5" x 10.5" x 1.25" (165 mm x 267 mm x 32 mm) Holds 12 instruments with an accessory area. Fits in SciCan $^{\circ}$ STATIM $^{\circ}$ or the Midmark M3 autoclave.

Red		IM8120
Gray		IM8121
White	\square	IM8122
Orange		IM8123
Yellow		IM8125
Lavender		IM8126
Blue		IM8128
Green		IM8129

Hu-Friedy cassettes can be modified to fit the unique needs of your practice. Call Hu-Friedy for more information.



Hu-Friedy Resin Cassettes are constructed of high-quality resin. Soft silicone instrument rails gently hold instruments in place. Cools down quickly after sterilization and are lightweight.







Small Cassettes Exterior Dimensions 4.5" x 8" x 1.5" (114 mm x 203 mm x 38 mm) Small Resin 7 Instruments IMS-3107 Holds up to 7 instruments. The cassette is perfect for exam and prophylactic procedures. Small Resin 12 Instruments | IMS-3112 Holds up to 12 instruments Small Resin 16 Instruments | IMS-3116 Holds up to 16 instruments in a compact space (pictured) Large Cassettes - No Accessary Area Exterior Dimensions 8" x 11" x 1.5" (203 mm x 279 mm x 38 mm) | IMS-1118 Large Resin 18 Instruments Holds up to 18 instruments (pictured) Large Resin 24 Instruments IMS-1124 Holds up to 24 instruments Large Cassettes - with Accessary Area Exterior Dimensions 8" x 11" x 1.5" (203 mm x 279 mm x 38 mm) | IMS-1112 Large Resin 12 Instruments Holds up to 12 instruments (pictured) | IMS-1116 Large Resin 16 Instruments Holds up to 16 instruments

BLEACH CLEANING RECOMMENDATIONS FOR RESIN CASSETTES

The resin cassettes should be bleach cleaned on a six month basis. They may be submerged in a solution of two cups bleach to one gallon of water (0.5 liters bleach to 4 liters of water) for 5 to 30 minutes. If staining has occurred, use less water and more bleach. If bleach does not break through the stain, wipe cassettes with isopropyl alcohol prior to bleaching. Be sure to remove instruments before placing cassettes in bleach solution as bleach is destructive to metal. This cleaning procedure is only recommended with resin cassettes. If staining does occur on stainless steel cassettes, wipe clean with soap and water. DO NOT use bleach process described for resin cassettes on stainless steel cassettes.

SIGNATURE SERIES[™] TUBS

Together, procedure tubs and cassettes create an efficient and organized system for managing instruments and consumable products. The time savings achieved by using IMS (Instrument Management System) cassettes and procedure tubs will allow the staff to spend time on more value added activities in the dental office.

Signature Series Tubs are designed to match IMS cassette rail colors and are treated with [Sanafor® P0-5] antimicrobial* to protect the product by helping prevent the growth of stain and odor causing bacteria.

Signature Series[™] Complete Tub 12-3/4" x 11-3/16" x 4" | IMS-1400 Includes: Red • Signature Series Tub | IMS-1401 Gray White | IMS-1402 • Tub Liner⁺ • Tub Divider (package of 8) | IMS-1403 Orange Single Tub Cup w/Cover⁺ Purple | IMS-1404 • Double Tub Cup w/Cover⁺ Yellow | IMS-1405 Divided Slide Tray⁺ Lavender | IMS-1406 Black Locking Tub Cover⁺ | IMS-1407 Blue IMS-1408 Green | IMS-1409 Ocean Blue | IMS-14011



Signature Series Tub 12-1/4" x 10-7/8" x 2-3/4"

Red		IMS-1410
Gray		IMS-1411
White	\square	IMS-1412
Orange		IMS-1413
Purple		IMS-1414
Yellow		IMS-1415
Lavender		IMS-1416
Black		IMS-1417
Blue		IMS-1418
Green		IMS-1419
Ocean Blue		IMS-14111

Locking Tub Cover⁺

12-3/4" x 11-3/16" x 1-1/2"

Protects items from airborne materials during transportation and storage

IMS-1420



Tub Insert 9-3/4" x 10-3/4" x 11/32"

The insert features 10

compartments for easy organization of tub materials Sits on rim of tub allowing for

sits on rim of tub allowing for storage underneath

| IMS-1423



Tub Dividers

Tub Liner

Divided Slide Tray⁺ 9-3/4" x 5-3/8" x 11/16"

Stores cotton rolls, cotton squares, clamps, syringe tips, etc.

| IMS-1421

Flat Slide Tray[†] 9-3/4" x 5-3/16" x 1- 3/8"

Can be used to store discs, strips, toffelmeil, articulating paper, etc.

IMS-1422

Dividers

12-3/4" x 11-3/16" x 1-3/8"

| IMS-1427

Includes 8 dividers

Tub Liner⁺

9-5/8" x 8-7/8" x 1/8"

| IMS-1428

Non-slip liner prevents small items from slipping under divider sections

* These antimicrobial properties are built-in to protect the product. This product does not protect users or others against bacteria, viruses, germs or other disease organisms.

†Clear components and tub liner do not contain antimicrobial product projection.





SIGNATURE SERIES TUB CUPS & COVERS

Signature Series Tub cups and covers can be used to store brushes, clamps, cotton pellets, syringe tips, prophy paste, prophy angle tips, pumice, implant tools, etc.



Single Tub Cup with Cover ⁺	
2-1/4" x 1-5/8" x 1-1/8"	IMS-1424
Double Tub Cup with Cover ⁺	
3-3/8" x 2-1/4" x 1-1/8"	IMS-1425
Long Tub Cup with Cover ⁺	
8-1/3" x 2-1/4" x 1-1/8"	IMS-1426
 Provides 5 storage compartments Dividers are removable for customization	

• Perfect for storing capsules, discs, abrasive strips, wedges, matrix bands, etc.

SIGNATURE SERIES RACKS

	7[
1	13"
	$\left \right $
	¥ 1
- Information	10.1

Adjustable 3 Tub Rack

B=3/4"

Adjustable 4 Tub Rack

| IMS-1433

| IMS-1434

Adjustable 3 Tub Rack

MIN 7-1/2" - MAX 15-1/2" x 10-1/2" x 13"

Fits most standard storage cabinets
Holds a maximum of 3 tubs


with lids or 6 cassettes

- Non-slip feet
- Sturdy metal construction
- No screws, tools, nuts or bolts needed
- Adjustable 4 Tub Rack

MIN 7-1/2" - MAX 15-1/2" x 10-1/2" x 18-3/4"

- Holds a maximum of 4 tubs with lids or 8 cassettes
- Non-slip feet
- Sturdy metal construction

SIGNATURE SERIES CLIP-ON HOLDERS

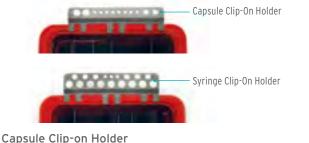

| IMS-1429

I IMS-1430

| IMS-1432

Capsule Clip-on Holde

- 8" x 2-1/2" x 3/4"
- · Allows quick access to composite material during procedure
- · Conveniently clips on any side of tub
- Holds up to 8 capsules, 4 syringe tubes and capsule/gun dispenser

Syringe Clip-on Holder

8" x 2-5/8" x 3/4"

- · Allows quick access to composite material during procedure
- Conveniently clips on any side of tub
- Holds up to 15 syringe tubes and 3 applicator tips

SIGNATURE SERIES SYRINGE STANDS



- Stands up for easy access to syringes
- Lays flat to fit inside a tub or drawer
- Holds up to 20 syringe tubes

10-Unit Syringe Add On

8-1/2" x 2-3/8" x 3/4"

- An add-on for the Syringe 20 Unit Stand
- Provides additional space for syringe storage
- Holds up to 10 syringe tubes

ANTIMICROBIAL PROTECTION

*Antimicrobial protection is integrated into the tub and tub components during the manufacturing process. This technology works by interrupting key cell functions that allow the microorganisms to function, grow and reproduce. Antimicrobial protection begins to work as soon as the microorganism comes into contact with the product surface and works continuously to inhibit the growth of microbes that can cause stain, odors and product degradation - for the lifetime of the product.

*These antimicrobial properties are built-in to protect the product. This product does not protect users or others against bacteria, viruses, germs or other disease organisms.



ORGANIZATION



IMS COLOR CODE INSTRUMENT RINGS

IMS Color Code Instrument Rings are silicone rubber rings used for identifying and organizing instruments. Useful for maintaining proper instrument order in cassettes and identifying instruments by procedure; can withstand all heat sterilization methods.



		Regular Use on #2 and 4 Handles	Large Use on #6, 7, 8 and 9 Handles
Assorted Color	rs	IMS-1280	IMS-1280L
Gray		IMS-1281	IMS-1281L
White	\square	IMS-1282	IMS-1282L
Orange		IMS-1283	IMS-1283L
Brown		IMS-1284	IMS-1284L
Yellow		IMS-1285	IMS-1285L
Red		IMS-1286	IMS-1286L
Green		IMS-1287	IMS-1287L
Blue		IMS-1288	IMS-1288L
Black		IMS-1289	IMS-1289L
Pink		IMS-12810	IMS-12810L
Lavender		IMS-12811	IMS-12811L
Ocean Blue		IMS-12812	IMS-12812L
Purple		IMS-12813	IMS-12813L

All single color packages contain 50 pieces. The assorted color package contains 90 pieces.

SIGNATURE SERIES[™] GRIPS

Signature Series grips can only be used with Satin Steel Colours®, EverEdge® technology and ResinEight® instruments, and are used for identifying and organizing instruments. They're useful for maintaining proper instrument order in cassettes and identifying instruments by procedure. Grips will withstand all approved heat sterilization methods



Blue

Black

Pink

Lavender

Ocean Blue

16 Assorted

80 Assorted

All single color packages contain 16 pieces. The assorted color packages contain your choice of either 16 or 80 pieces.

IMS COLOR TABS

Silicone tabs that fit on cassette lid for identifying cassette procedure by color.



All single color packages contain 5 pieces. The assorted color package contains 13 pieces (Red, Gray, White, Orange, Purple, Yellow, Lavender, Black, Blue, Green, and Ocean Blue, Pink, and Brown).

Red		IMS-1290N
Gray		IMS-1291N
White	\square	IMS-1292N
Orange		IMS-1293N
Purple		IMS-1294N
Yellow		IMS-1295N
Lavender		IMS-1296N
Black		IMS-1297N
Blue		IMS-1298N
Green		IMS-1299N
Pink		IMS-12910N
Ocean Blue		IMS-12911N
Brown		IMS-12913N
Assorted		IMS-12912N

IMS CASSETTE I.D. LABELS AND I.D. MARKER

Specially designed polyester pressuresensitive labels for cassette identification.

SSG8

|SSG9

| SSG10

SSG11

SSG12

| SSGO

SGMULTI



RESIN 8 COLORS GRIPS

Red	SSGRED8
Orange	SSGORANGE8
Yellow	SSGYELLOW8
Purple	SSGPURPLE8
Lavender	SSGLAVNDR8
Green	SSGGREEN8
Blue	SSGBLUE8
Gray	SSGGRAY8

All single color packages contain 16 pieces.

IMS Cassette	IMS-1234
I.D. Labels	

50 per package

| IMS-1235 IMS I.D. Marker

Permanent indelible ink marker is ideal for identifying procedural set-ups. For use on I.D. Labels or Monitor Tape.



"Small violations can have **big consequences.**

What could I be missing?"



STAYING COMPLIANT IS MORE CRITICAL THAN EVER.

According to an industry research study,* 99% of dental professionals believe their infection control programs are effective. But 62% are still concerned with how current their programs are, when compared to published OSHA standards.

Compliance covers more than just OSHA; CDC and state board regulations also factor into your infection prevention protocols. The number and extent of published standards means practices are held to a high level of accountability. But with these standards, there is no "good enough" – only compliant or noncompliant.

NO FACILITY IS IMMUNE FROM THE RISK OF NONCOMPLIANCE.

DISCOVER GREENLIGHT

A BETTER WAY TO STAY COMPLIANT

Maintaining compliance doesn't have to be a burden. GreenLight Dental Compliance Center by Hu-Friedy is an exclusive program that keeps all your infection prevention guidelines and regulations conveniently housed in one portal.

HOW YOU BENEFIT

GreenLight Dental Compliance Center helps you create customized infection prevention and instrument reprocessing protocols and internal audits, so you are prepared. GreenLight also saves time on staff training and updates, with improved performance in your infection prevention practices. You'll find more than convenience with GreenLight – you'll also gain the confidence that comes from knowing your facility is in compliance.





More than ever dental practices are being held accountable-publicly and professionally for their infection prevention/control standards. The ideal approach to ensure your

infection prevention processes are as up-to-date as possible and that your team is standardized is to leverage Hu-Friedy's Greenlight Dental Compliance Center. Greenlight has all the infection prevention policies and procedures and resources you need to maintain optimum infection prevention compliance. And best of all, Greenlight is state specific! So no more guess work when it comes to being compliant with the Centers for Disease Control and Prevention (CDC) and state dental boards. Protect your patients and your practice with the Greenlight Compliance Center.

-LINDA HARVEY, RDH, MS, HRM Compliance/Risk Management Specialist rusted Confidential Protection for Your Practice

GREENLIGHT DELIVERS

- Exclusive access to a comprehensive members-only online infection prevention portal
- A one-stop resource for updated compliance information from regulatory bodies
- Easy access to state board web pages
- Customized infection prevention protocol templates compliant with your practice's specific requirements
- · Free consultation for new members to review protocols
- · Online infection prevention practice self assessment
- Online continuing education on OSHA and other topics
- Breach response plan with accompanying tools
- Training and resources specifically for infection control coordinators

READY TO PERFECT YOUR PROTOCOLS?

Your Hu-Friedy representative will explain program requirements and how you can get started on the path to increased compliance.

Learn more by calling your rep, or visit us online at

GreenLightComplianceCenter.com

"Hu-Friedy engaged The Key Group, an independent third-party market research company, to survey dental professionals on their infection control practices.

CLEANING AND CARE

Enzymax® Dual Enzyme Ultrasonic Detergent is specifically formulated for efficient ultrasonic cleaning and pre-soaking of cassettes and instruments. Contains Steelgard™ agents to extend instrument life and chelating agents to protect instruments from harmful minerals naturally found in water that contribute to buildup, spotting and corrosion. Multi-purpose applications including ultrasonic detergent, presoak, linen presoak and evacuation cleaner.

ENZYMAX









IMS-1222	
IMS-1222C	
IMS-1224	
IMS-1224C	
IMS-1226	
IMS-1226P	
	IMS-1222C IMS-1224 IMS-1224C IMS-1226

Enzymax Powder	
1.76 lb (800 g) Powder Makes 50 Gallons (189 liters)	IMS-1230
6 containers of 1.76 lb (800 g) Powder Makes 300 Gallons (1,134 liters)	IMS-1230C

Enzymax Pax® Single Use Packets

32 Dissolvable Single-Use Powder Packets Makes 32 Gallons (121 liters)	IMS-1232
96 Dissolvable Single-Use Powder Packets Makes 96 Gallons (363 liters)	IMS-1233

Enzymax Spray Gel

Dual enzyme gel with corrosion inhibitor acts as a ready-to-use instrument pre-cleaner. Maintains moisture to help prevent debris from drying on instruments before cleaning and helps dissolve blood, tissue, and other debris. Can also be used as a linen spot remover.

24 Fl oz (709 ml) 450 Sprays | IMS-1229

Hu-Friedy strongly recommends Enzymax with our instruments to maintain instrument longevity, integrity and appearance.



Enzymax Earth™ Dual Enzyme Ultrasonic Detergent offers the same superior cleaning attributes as Enzymax with an increased awareness of our environment. All Enzymax Earth products are biodegradable* and phosphate free.

ENZYMAX EARTH



Enzymax Earth Liquid		
Gallon Bottle Makes 128 Gallons (485 liters)	IMS-1336	
Enzymax Earth Pax Single Use Packets		
32 Dissolvable Single-Use Powder Packets Makes 32 Gallons (121 liters)	IMS-1332	
96 Dissolvable Single-Use Powder Packets Makes 96 Gallons (363 liters)	IMS-1333	

LILAC UTILITY GLOVES



Lilac Utility Gloves

Three pack of flocklined nitrile utility gloves for use when cleaning and processing instruments. Resist some chemicals better than latex gloves⁺ and superior resistance to puncture for improved protection.⁺ Steam autoclavable to 250°F (121°C) up to five times.

Small (Size 7)	40-060
Medium (Size 8)	40-062
Large (Size 9)	40-064
Extra Large (Size 10)	40-066

*Biodegradable formula according to the Organization for Economic Cooperation and Development (OECD) testing + CDC - https://safety.fsu.edu/safety_manual/supporting_docs/Glove%20selection%20chart%20CDC.pdf + Please refer to packaging instructions for the penetration times of chemicals tested..







SHINE RENEW PRODUCTS



Shine reNEW[™] Products

Effectively remove stain and rust, restoring the luster to stainless steel and tungsten carbide instruments.

Liquid, 1 Gallon (3.8 liters)		IMS-1453
Instrument Wipes (20 wipe	canister)	IMS-1455
• Keeps hinged instruments	• Quick and s	imple to use
moving freely by dissolving	 Shine reNEV 	V Instrument Wipes

- built-up residueEliminates the need for
- Eliminates the need for abrasive scrubs
- Shine reNEW Instrument Wipes can also be used on copper, brass and anodized aluminum

IMS® DAILY CLEAN GENERAL PURPOSE DETERGENT



IMS® Daily Clean General Purpose Detergent

Concentrated powder detergent specifically formulated for efficient ultrasonic cleaning or presoaking of cassettes and instruments.

1.1 lb (500 g) Makes 35 Gallons (132 liters)	IMS-1220
5 lb (2.27 kg) Makes 160 Gallons (606 liters)	IMS-1218

INSTRUMENT LUBRICANTS



Instrument Lubricants

The instrument lubricants come in a convenient non-aerosol pump bottle.

Penetrating Oil 8 oz (236 ml) Removes light rust and corrosion to loosen problem joints

loosen

I IPS

Instrument Lubricant Spray 8 oz (236 ml) | ILS Keeps hinged instruments moving freely


Instrument Lubricant Spray (ILS) and Penetrating Oil (IPS) do not contain silicone.



CLEANING MONITORS

Proper instrument cleaning is the most important step to instrument reprocessing. If an instrument is not properly cleaned, it cannot be effectively sterilized. Take the steps to assure that your cleaning process achieves the highest quality results by incorporating Hu-Friedy Cleaning Monitors into your instrument reprocessing protocols. Using multiparameter testing, Hu-Friedy Cleaning Monitors are designed to give you a clear interpretation of your cleaning process.



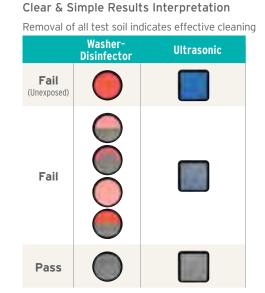

See Troubleshooting Charts (N1736 and N1692) for possible reasons for results and corrective actions.

Wash-Checks" OLU-

Washer-Disinfector Cleaning Monitor, 50 PCS



Ultrasonic Cleaning Monitor, 50 PCS | IMS-1200U



Cleaning Monitor Holder, 1 EA | IMS-1200H

	Washer-Disinfector	Ultrasonic
Ideal Cleaning Monitor Usage	Run a monitor test in the morning to release the equipment for use and minimum of one monitor per load for load release.	Monitor once daily with an empty load for a machine release and periodically throughout the day with instruments for a load release, if needed.
Parameters tested by the cleaning monitors	 Time Temperature Detergent concentration Spray arm function Enzyme soak 	 Cavitation Time Temperature Detergent
Common Cleaning Process Failures	 Inadequate water spray/impingement Clogged spray arms Overloading Instrument shadowing Inadequate detergent dosing Improper detergent dosing 	 Ineffective cavitation Overloading Insufficient time, temperature and/or detergent



SURFACE DISINFECTION

Hu-Friedy's AdvantaClear™ Surface Disinfectant is designed to clean and disinfect dental surfaces while working quickly and effectively killing a broad spectrum of disease-causing microorganisms in one minute.* With a low-alcohol, fragrance-free formula, AdvantaClear is safe on all hard, non-porous healthcare surfaces, including glass, plastics, stainless steel and vinyl.



ADVANTACLEAR SURFACE DISINFECTANT



AdvantaClear Wipes

Powerful scrubbing wipes that quickly and effectively clean and disinfect while still being safe on surfaces.

	US	Canada
Canister, 160 Wipes, 6" x 6.75" (15 cm x 17 cm)	IMS-2160	IMS-2160CA
Flat Pack, 10 Wipes 6'' x 6.75''	IMS-2010	



AdvantaClear Ready-to-Use Liquid

Convenient, ready-to-use packaging that is easy to use. No mixing or diluting required.

	US	Canada
Spray Bottle, 24 fl. oz (710 mL)	IMS-2024	IMS-2024CA
Gallon Bottle, 3.8 L	IMS-2128	

Available in the US and Canada

*This product kills the following organism in 2 minutes on pre-cleaned, hard, non-porous surfaces at room temperature: Mycobacterium bovis BCG (TB)





FAST-ACTING ONE MINUTE CONTACT TIME

A broad-spectrum disinfectant with one minute kill time⁺ – faster kill times mean faster turnover.



SAFE ON SURFACES

Designed to be compatible with a wide range of surfaces from acrylic, aluminum, brass, copper, stainless steel, chrome, plastics and more.



CLEANS & DISINFECTS

Quick and easy way to effectively clean and disinfect surfaces.



FRAGRANCE FREE

Helps eliminate odor-causing bacteria with no harsh fragrance.



LOW ALCOHOL FORMULA

Will not affix blood and soil to surfaces.

Minutes

	Methicillin-Resistant Staphylococcus aureus (MRSA)	1
KILL TIMES	Staphylococcus aureus	1
	Staphylococcus aureus Rosenbach	1
Gram-Positive Bacteria	Vancomycin-Resistant Enterococcus faecalis (VRE)	1
	Vancomycin Intermediate Staphylococcus aureus (VISA)	1
	Methicilin Resistant Staphylococcus epidermidis (MRSE)	1
	Enterococcus faecium	1
	Acinetobacter baumannii	1
	Multi-Drug Resistant (MDR) Acinetobacter baumannii	1
	Pseudomonas aeruginosa ATCC 15442	1
	Pseudomonas aeruginosa ATCC BAA-2108	1
	Salmonella enterica	1
Crem Negative Destante	Carbapenum Resistance Klebsiella pneumoniae	1
Gram-Negative Bacteria	Klebsiella pneumoniae	1
	Bordetella pertussis	1
	Extended-Spectrum β -lactamase <i>Escherichia coli</i> (ESBL)	1
	Enterobacter aerogenes	1
	Enterobacter cloacae	1
	Carbapenem Resistant Escherichia coli	1
	Hepatitis B Virus (HBV)	<1*
	Hepatitis C Virus (HCV)	<1*
	Herpes Simplex Virus Type 2	1
Enveloped Vinueest	Herpes Simplex Virus Type 1	1
Enveloped Viruses [†]	Human Immunodeficiency Virus (HIV-1) associated with AIDS	<1 *
	Influenza A Virus (H3N2)	1
	Influenza A Virus (H1N1)	1
	Human Coronavirus	1
Non-onvoloned Virusest	Rotavirus	1
Non-enveloped Viruses ⁺	Norovirus	1
Fungi	Trichophyton interdigitale (ringworm)	1
Mycobacteria	Mycobacterium bovis BCG tuberculosis (TB)	2

"HBV, HCV and HIV-1 Precautions: This product kills HBV, HCV and HIV-1 on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in health care or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of HBV, HCV and/or the human immunodeficiency virus Type 1 (HIV-1) associated with AIDS. **Special instructions for cleaning against HBV, HCV and HIV-1 of surfaces/ objects soiled with blood/body fluids: Personal Protection:** Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable latex gloves, gowns, masks and eye coverings. **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces/objects before disinfection. **Contact Time:** Allow hard, non-porous surfaces to remain wet for 30 seconds to kill HBV, HCV, and HIV-1 prior to wiping or allow surface to air dry. **Infectious Materials Disposai:** Cleaning materials used that may contain blood/body fluids must be autoclaved and/or disposed of in accordance with local regulations for infectious materials disposal.

+ This product kills the following organism in 2 minutes on pre-cleaned, hard, non-porous surfaces at room temperature: Mycobacterium bovis BCG (TB). See product label for details.









DENTAL UNIT WATERLINES

ATTRACTOR AND A

HU-FRIEDY WATERLINE FILTER

The Hu-Friedy Waterline Filter continuously works to deliver clean water to patients and staff for one year. The filter features a convenient, ready-to-use cartridge that is easy to install and inhibits microbial contamination in the waterline. With a universal design, it can be used with either a municipal waterline or an independent water reservoir. The filter is non-corrosive and does not contain silver or hazardous materials, allowing for quick and easy disposal.



One-Year Lifespan

No daily, weekly or monthly protocols



Cleans and Maintains Meets or exceeds <200 CFU/mL



Universal Design

Can be used with either municipal waterlines or independent water reservoirs



User Friendly

Easily installed in just minutes



Anti-Microbial Cleaner

Inhibits growth of microbial contamination

	US	Canada
Hu-Friedy Waterline Filter	IMS-1440	IMS-1440CA




CDC RECOMMENDATIONS

The Centers of Disease Control (CDC) recommends that dental unit waterlines be tested on a regular basis to maintain its established standard of less than 500 colony-forming units per milliliter.*

If dental unit waterlines are not maintained on a regular basis, bacteria will develop on the inner surfaces of the tubing, blocks and other system components.

Simply maintaining the waterlines is not enough; it is a two part process of cleaning and maintenance. Over time, bacteria can overwhelm the waterline environment and begin to create microbial deposits. Using a daily irrigant, like VistaClean, and cleaning the waterlines regularly with an EPA approved antimicrobial, like VistaTab, will clean and control microbial contaminants to provide improved water quality while helping to protect patients, staff and equipment.

*Reference Centers for Disease Control and Prevention. Guidelines for Infection Control in Dental Health-Care Settings - 2003. MMWR 2003; 52 (No. RR-17): [pg. 31].

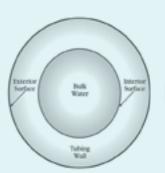



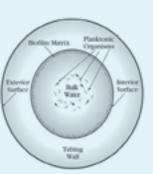


Figure 2. Contaminated dental waterline tubing. Biofilm on interior surface with abundant free-floating organisms.





DENTAL UNIT WATERLINE PRODUCTS



VistaTab[™] Dental Waterline Cleaner Tablets

VistaTab is an antimicrobial tablet used for the cleaning and control of microbial contaminants in dental unit waterlines. When placed in water, the solid tablet quickly dissolves to create a non-corrosive solution that is **seven times more effective than bleach**.

CONTAINS:

than bleach

15 - VistaTab Cleaner Tablets

• Quickly dissolves to create a non-corrosive solution seven times more effective

microbial contaminants

- Antimicrobial tablet used periodically for the cleaning and control of
- | IMS-1451
- EPA registered in all 50 states
- Available in the U.S. only

VistaClean[™] Irrigant Solution Concentrate

VistaClean is a daily irrigant solution derived from organic citrus botanicals and is safe for use with patients. It helps protect waterlines and equipment from deposits and scale and works to emulsify organic and inorganic contaminants that can build up in waterlines. VistaClean leaves no taste or odor and is non-toxic when used as directed. This **environmentally friendly solution is non-corrosive** and has no negative effect on bond strength.

CONTAINS:

- 1 One Fl oz bottle of VistaClean Irrigant Solution
- Derived from organic citrus botanicals
- Listed with the FDA as a Class I Medical Device
- Helps protect waterlines and equipment from deposits and scale
- Works to emulsify organic and inorganic contaminants that build up in waterlines
- · Safe for use with patients

IMS-1452

- No odor or taste
- Non-toxic, non-corrosive
- No effect on bond strength
- Available in the U.S. and Canada







IMS CASSETTE WRAP

Bonded Double Wrap

Hu-Friedy's Bonded Double Wrap allows dental practitioners to be in compliance with the most stringent sterilization standards for healthcare settings set forth by ANSI/AAMI ST79 and CDC

Guidelines. Our Bonded Double Wrap is comprised of two sheets of wrap, ultrasonically bonded together to form a single, two-ply application providing enhanced safety and increased




efficiency and productivity during wrapping and unwrapping. The two sheets are different colors which aids in identifying breaches in the packaging post sterilization and allows package differentiation by color. Compatible with the most common sterilization cycles, the material is also recyclable for reduced waste. Manufactured for single use only.


24" x 24" (610 mm x 610 mm) 240 Pieces	IMS-2224
20" x 20" (508 mm x 508 mm) 480 Pieces	IMS-2220
15" x 15" (380 mm x 380 mm) 480 Pieces	IMS-2215

WRAPPING TECHNIQUES

Method 1: Standard Gift

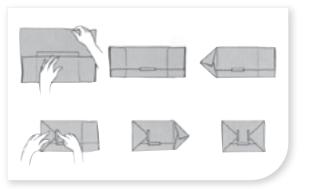

Method 2: Hospital style

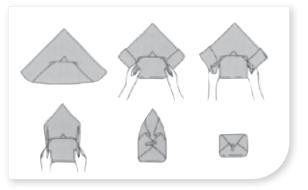

Autoclave and Universal Wraps

IMS Cassette Wrap is engineered to withstand the harsh conditions of sterilization. Our unique compilation of textiles is designed to be strong and soft for easy wrapping and durability. This lowlint, memory free wrap is available in two grades to meet your needs. Manufactured for single use only.

Autoclave Wrap

Autoclave Wrap offers the best combination of softness and added strength.

24" x 24" (610 mm x 610 mm) 500 Sheets	IMS-1210
20" x 20" (508 mm x 508 mm) 500 Sheets	IMS-1212
15" x 15" (380 mm x 380 mm) 1,000 Sheets	IMS-1211
12" x 12" (305 mm x 305 mm) 1,000 Sheets	IMS-1213

Universal Wrap

Universal Wrap provides great performance at a lower cost.

24" x 24" (610 mm x 610 mm) 500 Sheets	IMS-1214
20" x 20" (508 mm x 508 mm) 500 Sheets	IMS-1216
15" x 15" (380 mm x 380 mm) 1,000 Sheets	IMS-1215
12" x 12" (305 mm x 305 mm) 1,000 Sheets	IMS-1217

Note: Hu-Friedy Autoclave and Universal wrap is also available in packs of 100 sheets. Add "H" to the end of the part code.

WRAP SIZE RECOMMENDATIONS

	1/2"	151	2	
IMS Cassettes	Ĩ,	S.	ő	2 A
Exam, Double-Decker® 8-Instrument, Ultrasonic Insert, Symmetry IQ & Handpiece Cassette	٠	•		
Space Saver Seven™ Cassette	•	•		
Double-Decker 14-Instrument Cassette \triangle		٠	•	
Small 8- or 10-Instrument Cassette \triangle		•	•	
Signa-Stat or Double-Decker 20-Instrument Cassette Large 16- or 20-Instrument Cassette			•	
Large 12- or 20-Instrument Cassette			•	
Oral Surgery Cassette				•
Orthodontic Cassette Box				•
Large Thin Orthodontic Cassette			•	
Small Thin Orthodontic Cassette			•	
Double-Decker Large Ortho Cassette			•	

△ The Double-Decker 14-instrument cassette and Small cassettes are best with the 15" x 15" sterilization wrap. However, if an office is trying to purchase only one size, the 20" x 20" size is recommended.



IMS® TAPE

IMS Monitor Tape

Strong 3/4" (19 mm) tape can be written on and visually differentiates processed from non-processed cassettes. Pre-printed tape is available with specific procedural set-ups, color-coded or blank. 60 yards (55 m) per roll

PROCEDU	RE
---------	----

Ortho	IMS-1243	Extraction	IMS-1257
Archwire	IMS-1244	Perio	IMS-1258
Banding	IMS-1245	Prophy	IMS-1259
Bonding	IMS-1246	Pedo	IMS-1260
Restorative	IMS-1247	Gray	IMS-1262
Surgery	IMS-1248	Orange	IMS-1263
X-Ray	IMS-1249	Purple	IMS-1264
Amalgam	IMS-1251	Yellow	IMS-1265
C&B	IMS-1252	Red	IMS-1266
Cement	IMS-1253	Green	IMS-1267
Composite	IMS-1254	Blue	IMS-1268
Endo	IMS-1255	Black	IMS-1269
Exam	IMS-1256		

IMS Lead and Latex Free Monitor Tape*

Safe for the environment, your staff and patients. Not made with lead or natural rubber latex. Does not require special hazardous disposal. Easy to use and read results. For steam autoclaves. 3/4" (19 mm) 60 yards (55 m) per roll.

Blank

| IMS-1460



IMS Tape Dispensers

Rubber suction cups on the base secure dispenser to table or counter. Dispenser can also be mounted under cabinet.

IMS Tape Dispenser 10.75" x 6.25" x 7.6" (273 mm x 159 mm x 194 mm) Holds up to 9 rolls of IMS Monitor Tape	IMS-1270
IMS Mini Tape Dispenser 4.4" x 6.25" x 7.6" (111 mm x 159 mm x 194 mm) Holds up to 4 rolls of IMS Monitor Tape	IMS-1270H

BAGETTE® SELF-SEALING STERILIZATION POUCHES

Bagette[®] Self-Sealing Sterilization Pouches are constructed with durable materials and thick seals. Built-in external and internal quality control indicators relay accurate time and temperature verification, allowing you peace of mind. The full line of pouches include those custom fit especially for our cassettes and smaller sizes for all other sterilization needs.



Size	QTY	Suitable Contents	Part Code
2-1/4" x 4"	200	Rubber Dam Clamps or Burs	IMS-1346
2-3/4" x 9"	200	1 Handpiece or 1-3 Hand Instruments	IMS-1347
3-1/2" x 9"	200	2-5 Hand Instruments	IMS-1236
5-1/4" x 10"	200	Handpiece or Multiple Instruments	IMS-1348
7-1/2" x 13"	100	Exam Cassette⁺, Double-Decker 8 Instrument	IMS-1237
10" x 14"	100	Small Cassette†, Double Decker 14 Instrument, Small Thin Ortho	IMS-1238
12" x 17"	100	Large Cassette⁺, Double Decker 20 Instrument, Large Thin Ortho	IMS-1239
13" x 20"	100	Oral Surgery Cassette ⁺	IMS-1345

*Not made with natural rubber latex

+ Please see Hu-Friedy.com for complete list of suitable cassettes for this pouch.



IMS STEAM STERILIZATION MONITORS

IMS Steam Sterilization Indicators



IMS Steam Sterilization Indicators

Indicator strip verifies that steam sterilization parameters of time and temperature have been met. Use one in each cassette or pouch and retain for office records.

250 per box

| IMS-1240

IMS Steam Sterilization Integrators



IMS Steam Sterilization Integrators

Sterilization Integrators are a Type 5, multi-variable, sterilization monitor designed to react to all critical variables in the sterilization cycle: time, temperature and presence of steam. The integrators utilize a moving front indicator technology, which is an advancing color bar moving from reject to accept, eliminating the need to interpret or match colors.

100 per pouch

| IMS-1241

SPORECHECK BIOLOGICAL MONITORING SYSTEM

The SporeCheck 24 Hour In-Office Test[®] biological monitoring system provides reliable results in 24 hours, while being in compliance with local, state and federal guidelines. The SporeCheck system can be used with all types of steam sterilizers – including gravity, pre-vacuum and flash sterilizers.




SporeCheck Starter Kit Contains: Dry Block Incubator with built-in vial crusher, record notebook and 1 box SporeCheck Biological Indicators, 25 count	IMS-1376
SporeCheck Biological Indicator Vials, 25 count	IMS-1373
SporeCheck Biological Indicator Vials, 100 count (pictured at left)	IMS-1373H
55-60° Dry Block Incubator	IMS-1374
Sterilizer Record Notebook	IMS-1375
Replacement Vial Crusher	IMS-1377

The Center for Disease Control (CDC) recommends that an internal chemical indicator is placed in every package to be processed.* *Reference: Centers for Disease Control and Prevention. Guidelines for Infection Control in Dental Health-Care Settings – 2003. MMWR 2003; 52 (No. RR-17): [pg. 24].



STERILIZATION CONTAINER SYSTEM

Hu-Friedy's Sterilization Containers are a reusable, rigid system used for the packaging of cassettes and instruments during sterilization. An alternative to the wrap and pouch materials commonly used to package cassettes and instruments, Sterilization Containers provide significant economic and environmental benefits along with enhanced safety.

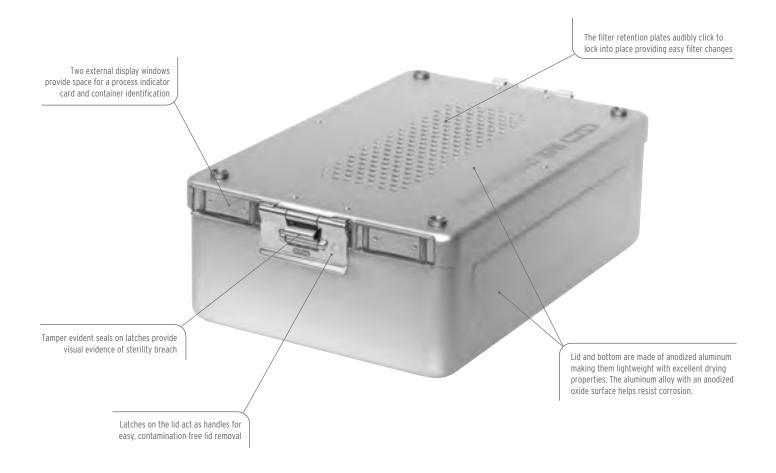







Land	Sterilization Container 312 x 190 x 50 mm 12-1/4" x 7-1/2" x 2"	IMDINCOOM
L'SL.	Sterilization Container 312 x 190 x 65 mm 12-1/4" x 7-1/2" x 2-1/2"	IMDINC01M
	Top and bottom with filter. Fits one container cassette.	
in a line in	Sterilization Container 312 x 190 x 100 mm 12-1/4" x 7-1/2" x 4"	IMDINCO2M
	Top and bottom with filter. Fits two container cassettes.	
C'SLIT	Sterilization Container 312 x 190 x 135 mm	IMDINCO3M
	12-1/4" x 7-1/2" x 5-1/3" Top and bottom with filter.	
	Fits three container cassettes.	

CONTAINER ACCESSORIES



Container Indicator Card 250 pieces, single use	IMCO-INDNA
Tamper Evident Seal for Lock 100 pieces, single use	IMCOM-SEAL
Disposable Paper Filter 100 pieces, blank, single use	IMCO-FIL3M

Container Indicator Card

HAND HYGIENE



HAND ESSENTIALS

Hu-Friedy Hand Essentials is a line of effective hand moisturizers and sanitizers designed for frequent use to help with program compliance. All of Hu-Friedy's hand hygiene products are skin friendly and cost effective.





Hand Essentials[™] Skin Repair Cream

Helps restore the skin's natural moisture balance

	US	Canada
32 Fl oz (946 mL) Bottle w/Pump	IMS-1500	IMS-1500CA
4 Fl oz (118 mL) Tube	IMS-1501	N/A
2 Fl oz (59 mL) Tube	IMS-1502	IMS-1502CA

• Aids in delivering moisture and nutrients to skin

- Contains Olivamine® to to help nourish skin
- CHG and latex compatible






| IMS-1505





INCREASED SKIN HYDRATION

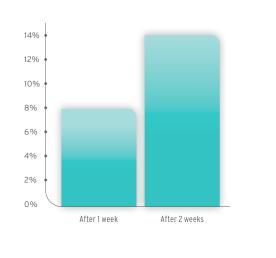

Sterillium Comfort Gel® significantly increases skin hydration after only one week of product usage compared to untreated skin.¹

Sterillium Comfort Gel is a registered trademark of Bode Chemie GmbH Co. *Data on file. Available upon request.

Hand Essentials Instant Hand Antiseptic

An essential aid to disinfect hands

16 Fl oz (473 mL) Bottle w/Pump

- · Contains Vitamin E and aloe vera to condition skin
- 70% v/v ethyl alcohol gel dries quickly on hands
- Scent-free formula
- CHG and latex compatible

Sterillium Comfort Gel® Hand Antiseptic


Scientifically proven, easy-to-use hand sanitizing gel

16.06 Fl oz (475 mL) Bottle w/Pump	IMS-1504
Sterillium Wall Bracket (for IMS-1504) 2/pk	974730
1.69 Fl oz (50 mL) Bottle	IMS-1504S

- 85% alcohol kills 99.999% of a broad range of nosocomial pathogens within 15 seconds
- Proven at only 2.4 mL per application
- Proven to increase skin hydration within one week
- CHG and latex compatible

TRADING ECONOMY FOR EFFICACY - IN VIVO TEST

Our test comparing Sterillium Comfort Gel® against two competitors products showed that for Sterillium, only 2.4 mL was required to entirely cover the hands and nails.

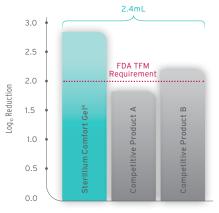

Test method follows FDA TFM for Health-Care Antiseptic Drug Products² and compliant with ASTM E1174, Standard Test Method for Evaluation of Healthcare Personnel Handwash Formulations. Sterillium Comfort Gel® in comparison to two other alcohol-based hand sanitizers available on the market.³ In vivo testing shows, use of only 2.4 mL of Sterillium Comfort Gel® exceeds the FDA TFM efficacy requirements.*

48

Hu-Friedy offers complete Infection Control Starter Packages that include everything you need to ensure a safer and smarter practice for you and your patients from day one. Simply choose the appropriate package based on whether or not you are a cassette user and whether you practice in the U.S. or Canada.

INFECTION CONTROL STARTER PACKAGES



UNITED STATES PACKAGES



Cassette User | ICSPCASS Cassette Infection Control Starter Package

IMS Universal Wrap 20" x 20", 100 Sheets	IMS-1216H
IMS Universal Wrap 15" x 15", 100 Sheets	IMS-1215H
IMS Lead/Latex Free Autoclave Mon. Tape	IMS-1460
IMS Integrator Strips, 100/Pkg	IMS-1241
SporeCheck Starter Kit	IMS-1376
Enzymax Earth, 1 Gallon Bottle	IMS-1336
Pump for Enzymax Gallon Bottle	IMS-1226P
Enzymax Earth Powder Sample Pack (Qty 2) IMS-1332SP
Enzymax Spray Gel, 24 oz	IMS-1229
AdvantaClear Surface Disinfectant Wipes, 160/Canister (Qty 3)	IMS-2160
Hu-Friedy Waterline Filter	IMS-1440
Hand Essentials Skin Repair Cream, 2 oz	IMS-1502
Sterillium Comfort Gel, 16.06 oz	IMS-1504
Hand Essentials Instant Hand Antiseptic, 16	oz IMS-1505



Cassette User Expanded | ICSP Cassette Infection Control Starter Package, Expanded

IMS Universal Wrap 20" x 20", 500 Sheets	IMS-1216
IMS Universal Wrap 15" x 15", 1000 Sheets	IMS-1215
IMS Lead/Latex Free Autoclave Mon. Tape (Qty 2)	IMS-1460
IMS Monitor Tape - Exam	IMS-1256
IMS Monitor Tape - Prophy	IMS-1259
IMS Monitor Tape - Restorative	IMS-1247
IMS Monitor Tape - Endo	IMS-1255
IMS Mulitple Tape Dispenser, 9 Roll Cap.	IMS-1270
IMS Integrator Strips, 100/Pkg	IMS-1241
SporeCheck Starter Kit	IMS-1376
Enzymax Earth, 1 Gallon	IMS-1336
Pump for Enzymax Gallon Bottle	IMS-1226P
Enzymax Earth Powder Sample Pack	IMS-1332SP
Enzymax Spray Gel, 24 oz	IMS-1229
AdvantaClear Surface Disinfectant Wipes, 160/Canister (Qty 6)	IMS-2160
Hu-Friedy Waterline Filter	IMS-1440
Cleaning Monitor Holder	IMS-1200H
Cleaning Monitors - Ultrasonic, 50/Pkg	IMS-1200U
Hand Essentials Skin Repair Cream, 4 oz	IMS-1501
Sterillium Comfort Gel, 16.06 oz	IMS-1504
Hand Essentials Instant Hand Antiseptic, 16 oz	IMS-1505



Non-Cassette User | ICSPCONSBL Infection Control Starter Package

IMS Bagette 3-1/2" x 9", 200 Pouches		IMS-1236
IMS Integrator Strips, 100/Pkg		IMS-1241
SporeCheck Starter Kit		IMS-1376
Enzymax Earth, 1 Gallon Bottle		IMS-1336
Pump for Enzymax Gallon Bottle		IMS-1226P
Enzymax Earth Powder Sample Pack (Qty 2)	IMS-1332SP
Enzymax Spray Gel, 24 oz		IMS-1229

AdvantaClear Surface Disinfectant Wipes, 160/Canister (Qty 3)	IMS-2160
Hu-Friedy Waterline Filter	IMS-1440
Hand Essentials Skin Repair Cream, 2 oz	IMS-1502
Sterillium Comfort Gel, 16.06 oz	IMS-1504
Hand Essentials Instant Hand Antiseptic, 16 oz (Qty 2)	IMS-1505

Please note that contents of starter packages are subject to change. To verify the most current configurations, please visit hu-friedy.com.



For a discounted price off of retail, you will receive Hu-Friedy IMS wrap or Bagettes, SporeCheck, Enzymax, Hand Essentials products and more.

CANADIAN PACKAGES



Cassette User | ICSPCASSCA

Cassette Infection Control Starter Package

IMS Universal Wrap 20" x 20", 100 Sheets	IMS-1216H
IMS Universal Wrap 15" x 15", 100 Sheets	IMS-1215H
IMS Lead/Latex Free Autoclave Mon. Tape	IMS-1460
IMS Integrator Strips, 100/Pkg	IMS-1241
SporeCheck Starter Kit	IMS-1376
Enzymax Earth, 1 Gallon Bottle	IMS-1336
Pump for Enzymax Gallon Bottle	IMS-1226P
Enzymax Earth Powder Sample Pack	IMS-1332SP
Enzymax Spray Gel, 24 oz	IMS-1229
AdvantaClear Surface Disinfectant Wipes, 160/Canister (Qty 3)	IMS-2160CA
Hu-Friedy Waterline Filter	IMS-1440CA
Hand Essentials Skin Repair Cream, 2 oz (Qty	(2) IMS-1502CA





Cassette User Expanded | ICSPCA Cassette Infection Control Starter Package, Expanded

IMS Universal Wrap 20" x 20", 500 Sheets	IMS-1216
IMS Universal Wrap 15" x 15", 1000 Sheets	IMS-1215
IMS Lead/Latex Free Autoclave Mon. Tape (Qty 2)	IMS-1460
IMS Monitor Tape - Exam	IMS-1256
IMS Monitor Tape - Prophy	IMS-1259
IMS Monitor Tape - Restorative	IMS-1247
IMS Monitor Tape - Endo	IMS-1255
IMS Mulitple Tape Dispenser, 9 Roll Cap.	IMS-1270
IMS Integrator Strips, 100/Pkg	IMS-1241
SporeCheck Starter Kit	IMS-1376
Enzymax Earth, 1 Gallon	IMS-1336
Pump for Enzymax Gallon Bottle	IMS-1226P
Enzymax Spray Gel, 24 oz	IMS-1229
Cleaning Monitor Holder	IMS-1200H
Cleaning Monitors - Ultrasonic, 50/Pkg	IMS-1200U
AdvantaClear Surface Disinfectant Wipes, 160/Canister (Qty 6)	IMS-2160CA
Hu-Friedy Waterline Filter	IMS-1440CA
Hand Essentials Skin Repair Cream, 2 oz (Qty 2)	IMS-1502CA



Non-Cassette User | ICSPCONSCA Infection Control Starter Package

IMS Bagette 3-1/2" x 9", 200/Bx	IMS-1236
IMS Bagette 7-1/2" x 13", 100/Bx	IMS-1237
IMS Integrator Strips, 100/Pkg	IMS-1241
SporeCheck, Starter Kit	IMS-1376
Enzymax Earth, 1 Gallon Bottle	IMS-1336
Pump for Enzymax Gallon Bottle	IMS-1226P

Sample Pack, Enzymax Earth Powder	IMS-1332SP
Enzymax Spray Gel, 24 oz	IMS-1229
AdvantaClear Surface Disinfectant Wipes, 160/Canister (Qty 3)	IMS-2160CA
Hu-Friedy Waterline Filter	IMS-1440CA
Hand Essentials Skin Repair Cream, 2 oz (Qty 2)	IMS-1502CA


Please note that contents of starter packages are subject to change. To verify the most current configurations, please visit hu-friedy.com.

REFERENCE SECTION



SUGGESTED STERICENTER LAYOUT

The Center for Disease Control (CDC) recommends the processing area should be divided into four sections to control quality and ensure safety:

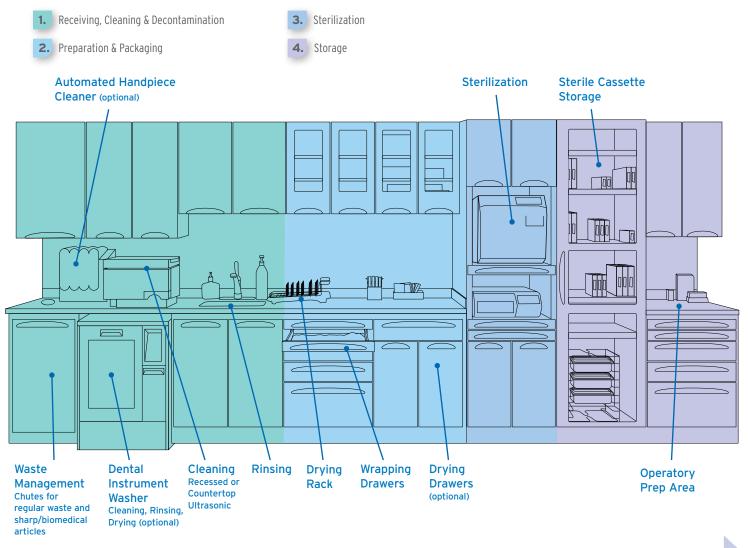

SOILED

STERILE

Space Requirements: Cassettes vs. Trays Counter Space Cassettes vs. Trays

	Cassettes	Trays
Instrument Break Down Area	12"	36"
Ultrasonic Cleaner Area	24"	24"
Sink	24"	24"
Drying and Packaging	18"	36"
Hand-piece Maintenance	12"	12"
Sterilizer	24"	24"
Tray Assembly and Storage	18"	36"
Total Space	11 feet	16 feet

Counter Space Required

	Cassettes	Trays
4 - 6 Operatories (Stericenter)	11 feet	16 feet
7 - 8 Operatories (Stericenter)	13 feet	19 feet
9 or more	15 feet	22 feet

Adapted with permission from Dr. Mark Tholen's book "A Guide to Designing an Elegant Dental or Medical Office"



CASSETTE AND EQUIPMENT "RULES OF THUMB"

In order to determine your specific instrument and cassette needs, review these "Rules of Thumb." They offer good insight into your decisions about cassettes and equipment.

How many cassettes do I need?

To estimate the minimum number of cassettes needed for your office:

- 1. Take the average number of procedures performed per day 20
- 2. Divide that number in half
- 3. Add one additional cassette for backup and emergency 10+1=11
- Total number if cassettes needed per day

*Note: Guidelines only! This example processes instruments through sterilization 2 times per day. High production offices will need enough cassettes for 3-4 sterilizations cycles per day. To determine compatibility of Hu-Friedy IMS cassettes with cleaning and sterilization equipment, please refer to Hu-Friedy's Equipment Capacity Chart at Hu-Friedy. com/IMS-get-started. Learn more about your potential savings from switching to IMS by visiting Hu-Friedy.com/IMS-calculators.

HUSAVESM CONSUMABLES SAVINGS PROGRAM*

Experience huge savings and earn great benefits as a member of Hu-Friedy's HUSAVE program!

HUSAVE offers a variety of products direct from Hu-Friedy at significantly reduced prices. No more worrying about re-ordering or stocking bulky products. The HUSAVE program is designed to dramatically reduce the time you spend purchasing while saving you money in the process!

What kind of ultrasonic equipment do I need? Offices with:

- 1 Doctor / 1 Hygienist = 3 gallon tank minimum
- 1 Doctor / 2 Hygienist = 3 gallon tank or 6 gallon tank (if a fast-pace office)
- 2 Doctors / 1 Hygienist = 3-6 gallon tank
- Group Practices = Two 6 gallon tanks or instrument washer Note: 1 Gallon = 3.785 liters

Sterilization Equipment?

Offices with:

10

11*

- 1 Doctor / 1 Hygienist = 10" + chamber
- 1 Doctor / 2 Hygienist = 10" + chamber
- 2 Doctors / 1 Hygienist or users of Dental Automated washer/ disinfector = One-Two 10" + chambers or one very large sterilizer

Simply choose the product you would like to order, and when you would like them to ship, and we will do the rest. We will ship the product as you indicated and bill you through your participating dealer of choice.

To enroll in HUSAVE or to learn more about the savings program, please call 1-866-HUSAVE3 or go to Hu-Friedy.com/HUSAVE

SAVE MONEY: 10-50% price discounts on all Consumable Product purchases. SAVE TIME:

Less time ordering. Place one order for the entire year or call us when you need product. SAVE **SPACE:** Regular shipments when you need them. No need to store excess supplies.

*Available in the contiguous US only


The HUSAVE Program includes the following product categories:



Cleaning & Care Products



Stainless Steel Crowns

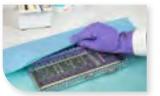

Sterilization Products



Preventive Products



Surface Disinfection



Prophy Powders




Sutures & Scalpel Blades



Symmetry Piezo tips




PROCEDURE TYPE	SET-UPS BY PROCEDURE	QUANTITY CASSETTES BY PROCEDURE	NUMBER OF INSTRUMENTS IN SET-UP	ACCESSORY RECOMMENDATION	CASSETTE RECOMMENDATION	CASSETTE RAIL COLOR
Directions	Total procedures for BUSIEST 1/2 day	Add 1 or 2 to the previous column	Count the number of straight, hinged and other instruments in the standard set-up	Use IMS Catalog to identify appropriate accessories (page 13)	Based on the number of instruments in the setup and referencing the IMS Catalog	0=Red; 1=Gray; 2=White; 3=Orange; 4=Purple, 5=Yellow; 6=Lavender; 7=Black, 8=Blue; 9=Green, 11=Ocean Blue
COMPOSITE RESTORATION						
AMALGAM RESTORATION						
UNIVERSAL RESTORATION						
CROWN & BRIDGE						
ENDO						
PROPHY						
EXAM						
SURGERY						
OTHER						
OTHER						
OTHER						
OTHER						
TOTAL CASSETTES						

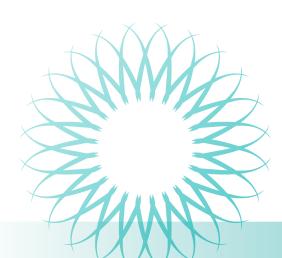



ADDITIONAL ITEMS

	IMS WRAP / BAGETTES	IMS TAPE / DISPENSER	STERILIZATION MONITORS	ENZYMAX / DAILY CLEAN	CLEANING MONITORS	ADVANTACLEAR	WATERLINE	HAND ESSENTIALS	ACCESSORIES	OTHER
PART CODE										
QUANTITY										
PART CODE										
QUANTITY										
PART CODE										
QUANTITY										
PART CODE										
QUANTITY										
PART CODE										
QUANTITY										
PART CODE										
QUANTITY										
PART CODE										
QUANTITY										

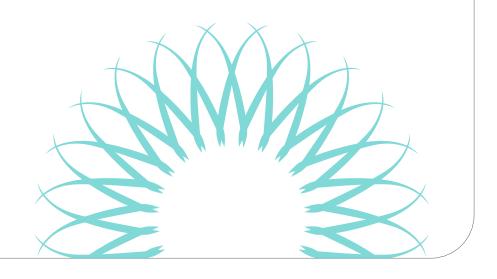



TROUBLESHOOTING

Problem	Cause	Prevention
• Spotting	 Insufficient rinsing after ultrasonic cleaning 	• Rinse thoroughly under steady stream of water for 30 seconds. Use distilled water if possible
	 Insufficient drying after ultrasonic cleaning 	 Rinse with hot water Optional: Dip cassettes in alcohol after rinsing
	 Not changing ultrasonic solution 	 Solution should be changed at least once a day
	 Sterilizer has not been cleaned 	 Follow manufacturer's guidelines for maintenance and cleaning Use only distilled water for reservoir
• Rust	Corrosion from carbon steel instruments spreads to stainless steel instruments	 Separate stainless and carbon instruments For carbon steel instruments: Dip in pre-sterilization solution as suggested by sterilizer manufacturer
	 Abrasion can destroy the chromium/oxide surface 	• Lubricate joints and lock area
	• Sterilizer not clean	 Follow manufacturer's guidelines for maintenance and cleaning
• Pitting	Chemical attack on instruments	 Rinse and dry instruments thoroughly. Use approved cleaning and sterilization solutions only, never use household bleach or stain and tartar remover
	 Chlorides (water leakage from water softener or additives containing chlorides 	 Use demineralized or distilled water in final rinse
	 Organic residues (blood, saliva, sweat) 	 Use approved cleaning and sterilization solutions only. Never use bleach, tartar or cement remover

Problem	Cause	Prevention
Staining (R	esin cassettes)	
• Black/ Purple	• Amalgam left in carrier	 Thoroughly remove amalgam or other compounds from the instrument prior to processing If carrier is plugged, sterilize separately and unplug while carrier is warm
	 Contact with ammonia 	• Use only proper and approved cleaning agents and rinse thoroughly
• Green	Chrome breakdown of instruments	 Inspect instruments and replace those with cracked handles or peeled plating
• Yellow/ Brown	 Sterilizer has not been cleaned 	 Change reservoir water once a week Regularly clean chamber and filters according to manufacturer's instructions
	Normal use discoloration	• Every 6 months soak resin cassettes (without instruments and not in the ultrasonic) in 2 cups bleach to 1 gallon of water for 5 to 30 minutes
• Brown/ Blue	 Sterilizer has not been cleaned 	 Regularly clean chamber and filters according to manufacturer's instructions
• Turning Matte Gray	• Excessive lime in water	• Use demineralized or distilled water for softened water in cleaning stage
	• Cement remover has been in contact with instruments	• Wipe off cement removers prior to placing in an automatic washer
Cassette Is	sues	
• Broken Hinges	 Overloading/ improperly placed instruments 	 Instruments should not protrude from cassette Only light force is needed to close the cassette
	 Improperly placed rails 	• Do not reposition resin cassette rails
• Wet Packs	 Insufficient drying before or during sterilization 	 Thoroughly dry cassette after cleaning, before wrapping Crack open autoclave sterilizer door during dry cycle Optional: After sterilization cycle, leave cassettes in warm sterilizer for 10 minutes
	 Improper loading of cassettes in sterilizer 	 Do not overpack sterilizer Keep cassettes slightly separated within the chamber Always use sterilizer's cassette rack



SIGNATURE SERIES REPLACEMENT RAILS



Signature Series Small Handpiece Cassette

Cassette Part Code	Color		Rail Replacement Set
IM30HU0	Red		IM30HUORL
IM30HU1	Gray		IM30HU1RL
IM30HU2	White	\square	IM30HU2RL
IM30HU3	Orange		IM30HU3RL
IM30HU5	Yellow		IM30HU5RL
IM30HU6	Lavender		IM30HU6RL
IM30HU8	Blue		IM30HU8RL
IM30HU9	Green		IM30HU9RL



Signature Series Large 16 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set
IM4160	Red		IM4160RAIL
IM4161	Gray		IM4161RAIL
IM4162	White	\square	IM4162RAIL
IM4163	Orange		IM4163RAIL
IM4165	Yellow		IM4165RAIL
IM4166	Lavender		IM4166RAIL
IM4168	Blue		IM4168RAIL
IM4169	Green		IM4169RAIL



Signature Series Large 20 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set
IM4200	Red		IM4200RAIL
IM4201	Gray		IM4201RAIL
IM4202	White	\square	IM4202RAIL
IM4203	Orange		IM4203RAIL
IM4205	Yellow		IM4205RAIL
IM4206	Lavender		IM4206RAIL
IM4208	Blue		IM4208RAIL
IM4209	Green		IM4209RAIL



Signature Series Small 10 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set
IM5100	Red		IM5100RAIL
IM5101	Gray		IM5101RAIL
IM5102	White	\square	IM5102RAIL
IM5103	Orange		IM5103RAIL
IM5105	Yellow		IM5105RAIL
IM5106	Lavender		IM5106RAIL
IM5108	Blue		IM5108RAIL
IM5109	Green		IM5109RAIL

Signature Series Small 8 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set
IM5080	Red		IM5080RAIL
IM5081	Gray		IM5081RAIL
IM5082	White	\square	IM5082RAIL
IM5083	Orange		IM5083RAIL
IM5085	Yellow		IM5085RAIL
IM5086	Lavender		IM5086RAIL
IM5088	Blue		IM5088RAIL
IM5089	Green		IM5089RAIL



Signature Series Large Double Decker - Ortho Cassette

-	-		
Cassette Part Code	Color		Rail Replacement Set
IM5200-OR	Red		IM5200-ORR
IM5201-OR	Gray		IM5201-0RR
IM5202-OR	White	\square	IM5202-ORR
IM5203-OR	Orange		IM5203-0RR
IM5205-0R	Yellow		IM5205-ORR
IM5206-0R	Lavender		IM5206-ORR
IM5208-0R	Blue		IM5208-ORR
IM5209-0R	Green		IM5209-0RR





Signature Series Exam Cassette - 5 Instrument

Cassette Part Code	Color		Rail Replacement Set
IM6050	Red		IM6050RAIL
IM6051	Gray		IM6051RAIL
IM6052	White	\square	IM6052RAIL
IM6053	Orange		IM6053RAIL
IM6055	Yellow		IM6055RAIL
IM6056	Lavender		IM6056RAIL
IM6058	Blue		IM6058RAIL
IM6059	Green		IM6059RAIL



Signature Series Signa Stat Cassette - 12 Instrument

Cassette Part Code	Color		Rail Replacement Set
IM8120	Red		IM8120RAIL
IM8121	Gray		IM8121RAIL
IM8122	White	\square	IM8122RAIL
IM8123	Orange		IM8123RAIL
IM8125	Yellow		IM8125RAIL
IM8126	Lavender		IM8126RAIL
IM8128	Blue		IM8128RAIL
IM8129	Green		IM8129RAIL



Signature Series Large Handpiece Cassette

Cassette Part Code	Color		Rail Replacement Set
ІМ60Н0	Red		IM60H0RAIL
IM60H1	Gray		IM60H1RAIL
IM60H2	White	\square	IM60H2RAIL
IM60H3	Orange		IM60H3RAIL
IM60H5	Yellow		IM60H5RAIL
IM60H6	Lavender		IM60H6RAIL
IM60H8	Blue		IM60H8RAIL
IM60H9	Green		IM60H9RAIL



Signature Series 8 Instrument Double Decker Cassette

Cassette Part Code	Color		Rail Replacement Set
IM9080	Red		IM9080RAIL
IM9081	Gray		IM9081RAIL
IM9082	White	\square	IM9082RAIL
IM9083	Orange		IM9083RAIL
IM9085	Yellow		IM9085RAIL
IM9086	Lavender		IM9086RAIL
IM9088	Blue		IM9088RAIL
IM9089	Green		IM9089RAIL



Signature Series – 14 Instrument Double Decker Cassette

Cassette Part Code	Color		Rail Replacement Set
IM9140	Red		IM9140RAIL
IM9141	Gray		IM9141RAIL
IM9142	White	\square	IM9142RAIL
IM9143	Orange		IM9143RAIL
IM9145	Yellow		IM9145RAIL
IM9146	Lavender		IM9146RAIL
IM9148	Blue		IM9148RAIL
IM9149	Green		IM9149RAIL

Signature Series 20 Instrument Double Decker Cassette

-			
Cassette Part Code	Color		Rail Replacement Set
IM9200	Red		IM9200RAIL
IM9201	Gray		IM9201RAIL
IM9202	White	\square	IM9202RAIL
IM9203	Orange		IM9203RAIL
IM9205	Yellow		IM9205RAIL
IM9206	Lavender		IM9206RAIL
IM9208	Blue		IM9208RAIL
IM9209	Green		IM9209RAIL



SIGNATURE SERIES REPLACEMENT RAILS



Signature Series Oral Surgery Cassette

Cassette Part Code	Color		Rail Replacement Set
IMOSO	Red		IMOSORAIL
IMOS1	Gray		IMOS1RAIL
IMOS2	White	\square	IMOS2RAIL
IMOS3	Orange		IMOS3RAIL
IMOS5	Yellow		IMOS5RAIL
IMOS6	Lavender		IMOS6RAIL
IMOS8	Blue		IMOS8RAIL
IMOS9	Green		IMOS9RAIL



Signature Series Space Saver Seven Cassette

Cassette Part Code	Color		Rail Replacement Set
IMSS70	Red		IMSS70RAIL
IMSS71	Gray		IMSS71RAIL
IMSS72	White	\square	IMSS72RAIL
IMSS73	Orange		IMSS73RAIL
IMSS75	Yellow		IMSS75RAIL
IMSS76	Lavender		IMSS76RAIL
IMSS78	Blue		IMSS78RAIL
IMSS79	Green		IMSS79RAIL



Signature Series Thin Ortho Cassette

Cassette Part Code	Color	Rail Replacement Set
IMORTHSM8	Blue	IMORTS8RL
IMORTHLG8	Blue	IMORTL8RL

INFINITY SERIES REPLACEMENT RAILS



Infinity Series Small Handpiece Cassette

Cassette Part Code	Color		Rail Replacement Set
IMN30HU0	Red		PK-30HUORL
IMN30HU1	Gray		PK-30HU1RL
IMN30HU2	White	\square	PK-30HU2RL
IMN30HU3	Orange		PK-30HU3RL
IMN30HU4	Purple		PK-30HU4RL
IMN30HU5	Yellow		PK-30HU5RL
IMN30HU6	Lavender		PK-30HU6RL
IMN30HU7	Black		PK-30HU7RL
IMN30HU8	Blue		PK-30HU8RL
IMN30HU9	Green		PK-30HU9RL
IMN30HU11	Ocean Blue		PK-30HU11R



Infinity Series Large 12 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set – Plain Lid	Rail Replacement Set – Embossed Lid
IMN4120	Red		PK-4120RL	PKE-4120R
IMN4121	Gray		PK-4121RL	PKE-4121R
IMN4122	White	\square	PK-4122RL	PKE-4122R
IMN4123	Orange		PK-4123RL	PKE-4123R
IMN4124	Purple		PK-4124RL	PKE-4124R
IMN4125	Yellow		PK-4125RL	PKE-4125R
IMN4126	Lavender		PK-4126RL	PKE-4126R
IMN4127	Black		PK-4127RL	PKE-4127R
IMN4128	Blue		PK-4128RL	PKE-4128R
IMN4129	Green		PK-4129RL	PKE-4129R
IMN41211	Ocean Blue		PK-41211RL	PKE-41211R

DO I HAVE A PLAIN OR EMBOSSED CASSETTE?

To identify an embossed style cassette, look at the lid. If it has two embossed stripes across the infinity pattern, you have an embossed cassette.

Embossed stripe detail on lid of Infinity Series Embossed Cassette





Infinity Series Large 16 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set – Plain Lid	Rail Replacement Set – Embossed Lid
IMN4160	Red		PK-4160RL	PKE-4160R
IMN4161	Gray		PK-4161RL	PKE-4161R
IMN4162	White	D	PK-4162RL	PKE-4162R
IMN4163	Orange		PK-4163RL	PKE-4163R
IMN4164	Purple		PK-4164RL	PKE-4164R
IMN4165	Yellow		PK-4165RL	PKE-4165R
IMN4166	Lavender		PK-4166RL	PKE-4166R
IMN4167	Black		PK-4167RL	PKE-4167R
IMN4168	Blue		PK-4168RL	PKE-4168R
IMN4169	Green		PK-4169RL	PKE-4169R
IMN41611	Ocean Blue		PK-41611RL	PKE-41611R



Infinity Series Large Thin Ortho Cassette

Cassette Part Code	Color	Rail Replacement Set
IMNORTHL4	Purple	PK-ORTL4L
IMNORTHL8	Blue	PK-ORTL8L
IMNORTHL9	Green	PK-ORTL9L



Infinity Series Small Thin Ortho Cassette

Cassette Part Code	Color	Rail Replacement Set
IMNORTHS4	Purple	PK-ORTS4L
IMNORTHS8	Blue	PK-ORTS8L
IMNORTHS9	Green	PK-ORTS9L



Infinity Series Large 20 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set – Plain Lid	Rail Replacement Set – Embossed Lid
IMN4200	Red		PK-4200RL	PKE-4200R
IMN4201	Gray		PK-4201RL	PKE-4201R
IMN4202	White	\square	PK-4202RL	PKE-4202R
IMN4203	Orange		PK-4203RL	PKE-4203R
IMN4204	Purple		PK-4204RL	PKE-4204R
IMN4205	Yellow		PK-4205RL	PKE-4205R
IMN4206	Lavender		PK-4206RL	PKE-4206R
IMN4207	Black		PK-4207RL	PKE-4207R
IMN4208	Blue		PK-4208RL	PKE-4208R
IMN4209	Green		PK-4209RL	PKE-4209R
IMN42011	Ocean Blue		PK-42011RL	PKE-42011R




Infinity Series Small 8 Instrument Cassette

Cassette Part Code	e Color		Rail Replacement Set
IMN5080	Red		PK2-5080RL
IMN5081	Gray		PK2-5081RL
IMN5082	White	\square	PK2-5082RL
IMN5083	Orange		PK2-5083RL
IMN5084	Purple		PK2-5084RL
IMN5085	Yellow		PK2-5085RL
IMN5086	Lavender		PK2-5086RL
IMN5087	Black		PK2-5087RL
IMN5088	Blue		PK2-5088RL
IMN5089	Green		PK2-5089RL
IMN50811	Ocean Blue	2	PK2-50811R



INFINITY SERIES REPLACEMENT RAILS

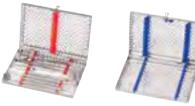


Infinity Series Small 10 Instrument Cassette

Cassette Part Code	Color		Rail Replacement Set	
IMN5100	Red		PK2-5100RL	
IMN5101	Gray		PK2-5101RL	
IMN5102	White	\square	PK2-5102RL	
IMN5103	Orange		PK2-5103RL	
IMN5104	Purple		PK2-5104RL	
IMN5105	Yellow		PK2-5105RL	
IMN5106	Lavender		PK2-5106RL	
IMN5107	Black		PK2-5107RL	
IMN5108	Blue		PK2-5108RL	
IMN5109	Green		PK2-5109RL	
IMN51011	Ocean Blue		PK2-51011R	



Infinity Series Exam Cassette - 5 Instrument

Cassette				
Part Code	Color		Rail Replacement Set	
IMN6050	Red		PK2-6050RL	
IMN6051	Gray		PK2-6051RL	
IMN6052	White	\square	PK2-6052RL	
IMN6053	Orange		PK2-6053RL	
IMN6054	Purple		PK2-6054RL	
IMN6055	Yellow		PK2-6055RL	
IMN6056	Lavender		PK2-6056RL	
IMN6057	Black		PK2-6057RL	
IMN6058	Blue		PK2-6058RL	
IMN6059	Green		PK2-6059RL	
IMN60511	Ocean Blue		PK2-60511R	

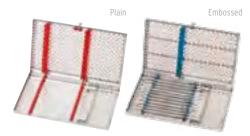

Infinity Series Signa Stat Cassette - 12 Instrument

1	· · · · · · · · · · · · · · · · · · ·			
Cassette Part Code	Color		Rail Replacement Set – Plain Lid	Rail Replacement Set – Embossed Lid
IMN8120	Red		PK-8120RL	PKE-8120R
IMN8121	Gray		PK-8121RL	PKE-8121R
IMN8122	White	\square	PK-8122RL	PKE-8122R
IMN8123	Orange		PK-8123RL	PKE-8123R
IMN8124	Purple		PK-8124RL	PKE-8124R
IMN8125	Yellow		PK-8125RL	PKE-8125R
IMN8126	Lavender		PK-8126RL	PKE-8126R
IMN8127	Black		PK-8127RL	PKE-8127R
IMN8128	Blue		PK-8128RL	PKE-8128R
IMN8129	Green		PK-8129RL	PKE-8129R
IMN81211	Ocean Blue		PK-81211RL	PKE-81211R



Infinity Series Large Handpiece Cassette

Cassette Part Code	Color		Rail Replacement Set	
IMN60H0	Red		PK-60HORL	
IMN60H1	Gray		PK-60H1RL	
IMN60H2	White	\square	PK-60H2RL	
IMN60H3	Orange		PK-60H3RL	
IMN60H4	Purple		PK-60H4RL	
IMN60H5	Yellow		PK-60H5RL	
IMN60H6	Lavender		PK-60H6RL	
IMN60H7	Black		PK-60H7RL	
IMN60H8	Blue		PK-60H8RL	
IMN60H9	Green		PK-60H9RL	
IMN60H11	Ocean Blue		PK-60H11RL	





A

Infinity Series 8 Instrument Double Decker Cassette

Cassette Part Code	Color		Rail Replacement Set	
IMN9080	Red		PK-9080RL	
IMN9081	Gray		PK-9081RL	
IMN9082	White	\square	PK-9082RL	
IMN9083	Orange		PK-9083RL	
IMN9084	Purple		PK-9084RL	
IMN9085	Yellow		PK-9085RL	
IMN9086	Lavender		PK-9086RL	
IMN9087	Black		PK-9087RL	
IMN9088	Blue		PK-9088RL	
IMN9089	Green		PK-9089RL	
IMN90811	Ocean Blue		PK-90811RL	

Infinity Series 14 Instrument Double Decker Cassette

Cassette Part Code	Color		Rail Replacement Set	
IMN9140	Red		PK-9140RL	
IMN9141	Gray		PK-9141RL	
IMN9142	White	\square	PK-9142RL	
IMN9143	Orange		PK-9143RL	
IMN9144	Purple		PK-9144RL	
IMN9145	Yellow		PK-9145RL	
IMN9146	Lavender		PK-9146RL	
IMN9147	Black		PK-9147RL	
IMN9148	Blue		PK-9148RL	
IMN9149	Green		PK-9149RL	
IMN91411	Ocean Blue		PK-91411RL	



Infinity Series 20 Instrument Double Decker Cassette

Cassette				
Part Code	Color		Rail Replacement Set	
IMN9200	Red		PK-9200RL	
IMN9201	Gray		PK-9201RL	
IMN9202	White	\square	PK-9202RL	
IMN9203	Orange		PK-9203RL	
IMN9204	Purple		PK-9204RL	
IMN9205	Yellow		PK-9205RL	
IMN9206	Lavender		PK-9206RL	
IMN9207	Black		PK-9207RL	
IMN9208	Blue		PK-9208RL	
IMN9209	Green		PK-9209RL	
IMN92011	Ocean Blue		PK-92011RL	





Infinity Series Oral Surgery Cassette

Cassette Part Code	Color		Rail Replacement Set – Plain Lid	Rail Replacement Set – Embossed Lid
IMNOSO	Red		PK-OSORAIL	PKE-OSOR
IMNOS1	Gray		PK-OS1RAIL	PKE-OS1R
IMNOS2	White	\square	PK-OS2RAIL	PKE-OS2R
IMNOS3	Orange		PK-OS3RAIL	PKE-OS3R
IMNOS4	Purple		PK-OS4RAIL	PKE-OS4R
IMNOS5	Yellow		PK-OS5RAIL	PKE-OS5R
IMNOS6	Lavender		PK-OS6RAIL	PKE-OS6R
IMNOS7	Black		PK-OS7RAIL	PKE-OS7R
IMNOS8	Blue		PK-OS8RAIL	PKE-OS8R
IMNOS9	Green		PK-OS9RAIL	PKE-OS9R
IMNOS11	Ocean Blue		PK-OS11RAI	PKE-OS11R



TECHNICAL SERVICE

Complete servicing for IMS Cassettes in North America is available at our Chicago facility. Please wrap and sterilize cassette prior to returning for repair. Items returned for servicing should be sent via UPS or registered insured mail:

ATTENTION: Technical Service Department Hu-Friedy Mfg. Co., LLC 3232 N. Rockwell St. Chicago, IL 60618

Price quotations on cost of repair are available upon request. If in need of additional service, please contact your local Hu-Friedy dealer. For IMS cassette service outside North America, please contact your local Hu-Friedy dealer.

REFERENCES FROM DATA ON PAGES 48

- 1. Independent Human Repeat Insult Patch Test. Data on file at Hu-Friedy.
- 2. Independent Skin Care Study/Skin Hydration. Data on file at Hu-Friedy.
- Topical antimicrobial drug products for over-the-counter human use; Tentative final monograph for health care antiseptic drug products; Federal Register 1994, 59 (116):31402-31452. To be codified at 21 CFR sec. 333.
- 4. Independent lab testing done on all products, April 13 2007.

IMS LIMITED WARRANTY

IMS® Cassette Limited Warranty

Hu-Friedy, at its sole option, will repair or replace, for a period of four years following the date of purchase, any IMS® cassette that is defective because of failure of materials (including corrosion) or workmanship of the cassette shell. Hu-Friedy, at its option, will also repair or replace, for a period of one year following the date of purchase, any, IMS® cassette that is defective because of failure of materials or workmanship of the internal and moving components. Purchaser must return the defective cassette (postage prepaid) and must have previously provided Hu-Friedy with the warranty registration card.

HU-FRIEDY DISCLAIMS LIABILITY, UNDER THIS WARRANTY OR OTHERWISE, FOR DAMAGES ARISING FROM (1) USER NEGLIGENCE, MISUSE, ABUSE OR ALTERATIONS TO THE CASSETTE; (2) THE USE OF COMMERCIAL/RESIDENTIAL GRADE WASHERS; (3) DENTAL AUTOMATED WASHER - DISINFECTORS WHERE MANUFACTURER'S PROCESSING GUIDELINES ARE NOT FOLLOWED; AND/OR (4) IMPROPER SET-UP AND INSTALLATION OF THE DEVICE.

Care Instructions for IMS cassettes can be found on the instrument processing insert included with each cassette and in the approved Instrument Reprocessing Guide on Hu-Friedy.com.

See website for complete warranty information. Hu-Friedy.com/IMSwarranty

TROUBLESHOOTING: REPROCESSING DO'S AND DON'TS

DO

- ...wear Hu-Friedy's Lilac Utility Gloves when handling contaminated cassettes and instruments.
- ...change the ultrasonic solution at least once a day or when visibly soiled. Dirty solution harbors particles that may inhibit efficient cleaning process.
- ...wipe off cement residue, glass ionomer and/or other heavy debris at chairside immediately after use.
- ...use Hu-Friedy's Enzymax products when cleaning Hu-Friedy cassettes and instruments for best results.
- ...sharpen instruments at chairside after sterilization and before a procedure. Hu-Friedy's sharpening stones can be processed along with instruments in the IMS cassette.
- ...autoclave utility gloves in seperate sterilization pouches.

DON'T

- ...let instruments protrude from cassettes.
- ...place cassettes in temperatures above 350°F/177°C.
- ...overload the cassette. If there is resistance when closing cassette, stop. Remove excess items and repeat the closing process. Forcing cassette to close under stress may inhibit future performance.
- ...put stainless steel and carbon steel instruments in the same cassette.
- ...reposition the location of the rails in resin cassettes. Improper rail location may cause undue stress on the cassette and can damage it.
- ... use IMS Autoclave Wrap in dry heat sterilization.
- ...run ultrasonic unit or steam autoclave without water.

HU-FRIEDY SOCIAL RESPONSIBILITY

At Hu-Friedy we believe in helping dental professionals perform their best. We strongly believe in protecting the health of our planet in addition to supporting the health of individuals. Part of this effort is looking toward the future and finding solutions such as printing this catalog on 30% recycled paper.

Hu-Friedy is proud to join a number of proactive, forward-thinking companies who are working together to ensure a better future for our planet with environmentally-friendly programs and products. Our internal effort includes a recycling program in which we recycled over 48,000 lbs of paper, 37,000 lbs of cardboard and over 1,000 lbs of batteries.

We are working toward finding ways to conserve energy, building our offering of "green" products and providing recycling options for our customers. Look for the leaf design throughout this catalog signifying our "green" offerings.

ENZYMAX EARTH™ DETERGENT

Enjoy all of the same great benefits of Enzymax detergents with an increased awareness for our environment. All Enzymax Earth products are biodegradable* and phosphate free (see page 31).

DENTAL WATERLINE CLEANER

VistaClean™ Irrigant Solution Concentrate utilizes an organic irrigant derived of citrus botanicals (see page 36).

IMS LEAD AND LATEX FREE MONITOR TAPE⁺


Making it safe for the environment, your staff and your patients and does not require special hazardous disposal (see page 38).

STERILIZATION CONTAINERS


Sterilization containers are a system of sterilization packaging in which reusable aluminum containers are used to sterilize instruments and cassettes. They provide a low cost of use, long service life and minimal use of disposable products (see pages 40-41).

ENVIRONDENT® INSTRUMENT RECYCLING PROGRAM

This program has been making a positive impact on the environment for over 20 years by recycling old, broken or retipped instruments in an environmentally responsible way. Visit **Environdent.com** for more information.

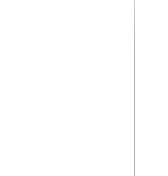



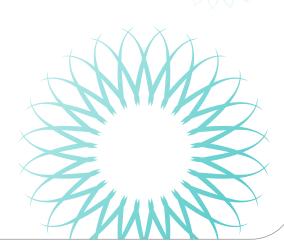



For more information about Hu-Friedy's Total Solutions visit **Hu-Friedy.com**



Hu-Friedy Mfg. Co., LLC: 3232 N. Rockwell St. | Chicago, IL 60618 | USA Telephone: 1-800-Hu-Friedy or 1-773-975-6100 | Website: Hu-Friedy.com

©2019 Hu-Friedy Mfg. Co., LLC. All rights reserved. All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. Sterillium Comfort Gei Is a registered trademark of Bode Chemie GmbH Co. Olivamine is a registered trademark of McCord Research Inc. The contents of this Product Catalog & Reference Guide including, but not limited to, all text, photographs and graphics, are the sole and exclusive property of Hu-Friedy Mfg. Co., LLC and may not be copied, reproduced or distributed without prior written permission. Some devices advertised may not be licensed for sale in accordance with Canadian Law. IMS-120/0620

