KaVo PROPHYflex 2 & 3

After Use

- 1. Remove glass/plastic jar from PROPHYflex
- 2. Screw on empty jar and run for 10 seconds to remove residue powder
- 3. Remove tip and wash/rinse tip through both ends with running water (use file if any clog is visible)
- 4. Wash/Rinse glass/plastic jar under running water, making sure no poweder remains
- Wash/Rinse PROPHYflex and run water from front to back through chamber to remove any remaining powder (screw may be removed and flushed if desired)
- 6. Dry with clean paper towel
- 7. STERILIZE (using steam or chemical)

After Sterilization

- Remove PROPHYflex from sterilizer and dry with clean towel, be sure powder chamber and screw are <u>VERY</u> dry
- 9. Dry top and glass/plastic jar with towel, again, very dry
- 10. Install tip and glass/plastic jar (without powder)
- 11. Run PROPHYflex for 30 seconds without powder (this will dry the inner chamber)
- 12. Fill with powder

Note

Compressor air must be absolutely free of any moisture

Do not let the PROPHYflex set too long after use without cleaning