

Mizzy PIP

Pressure Indicating Paste

Pressure Indicating Paste (PIP) is a pure white silicone paste, used for seating crowns, bridges and full, immediate and partial dentures. The brief procedure yields specific results that are easy to read and thoroughly reliable, surpassing results obtained with chemical set pressure indicating materials used to locate areas of denture discomfort. PIP is applied to the dry denture; companion PIP spray is applied to the tissue. The denture is then seated under pressure then removed. Areas where the paste has been displaced show points of excessive pressure.



ORDERING INFORMATION

1 oz Tube	#6120900
1-1/4 oz. Pip Jar w/ Remover	#6120200
2-1/4 oz. Pip Jar w/ Remover	#6120300
4 oz. PIP Pump	#6121000
PIP Jar – 8 oz	#6120400
PIP Brushes	#6120600
PIP Packet w/ Remover & Brush – 24/Pkg	#6120550
PIP Remover – 1 oz	#6000437
PIP Remover – 4 oz	#6120700