

Polishers | Set 4312A



for polishing composite.

This Kit is designed for polishing composite restorations and veneer acrylics. The threestep polishing system with coarse (blue), medium (pink) and fine grit (grey) guarantees an unequalled high-gloss finish on composite surfaces every time.

Recommendations for use:

- Recommended speed: Oopt. 5.000 - 6.000 rpm
- Use sufficient coolant (at least 50 ml/min.)
- Apply low contact pressure and work in a circular motion
- Work without polishing paste
- Use alcohol-free detergents and disinfectants suitable for polishers, e.g. Komet DC1 (manual reprocessing) or DCTherm (mechanical reprocessing)
- Suitable for sterilization in the autoclave only

Polishing is carried out in three subsequent steps:

1. The **light blue** polishers are designed for quick and effective pre-polishing of composite fillings.



9400.204.030



9403.204.055



2. The **light pink** polishers achieve a satin finish.





9404.204.055



3. A high-gloss finish is obtained with the fine, light grey polishers.





9405.204.055









