ProductInformation



Polishers | Kit 4313B









for polishing ceramics.

This Kit is designed for polishing ceramic restorations. The amazing surface quality obtained by these diamond interspersed polishers for pre-polishing, polishing and high-gloss polishing obviates the need for another final firing step after corrections have been made.

Recommendations for use:

- Recommended speed:
- O_{opt.} 5.000 6.000 rpm
- Use sufficient coolant
- (at least 50 ml/min.)
- Apply low contact pressure
- and work in a circular motion
- Work without polishing paste
- Use alcohol-free detergents and disinfectants suitable for polishers, e.g.
 Komet DC1 (manual reprocessing) or DCTherm (mechanical reprocessing)
- Suitable for sterilization in the
- autoclave only

Polishing is carried out in three subsequent steps:

1. The **dark blue** polishers are designed for quick and effective pre-polishing of ceramics.



 A high-gloss finish without need for an additional final firing step is obtained with the wear-resistant, high-performance grey polishers.

