


Locating & Installing Tap


(15L Drum)



When a drum arrives,
the tap is not visible

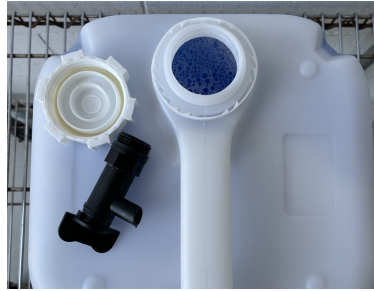

Unscrew the large cap at
the top to locate the tap
under the cap

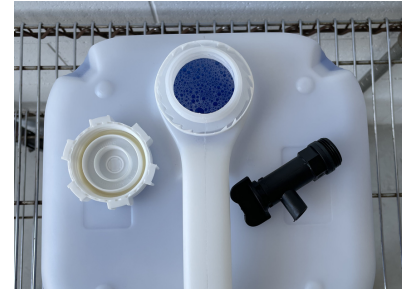

The tap is removed from
the cap by unscrewing in
an anti-clockwise direction



Replace the cap
and tighten



Lay the drum over so that
the small "bung" below the
label faces up



Unscrew the "bung" using
the spanner or similar tool,
by turning anti-clock wise



Screw the tap into the
"bung" thread

**ENSURE THE TAP IS
TURNED OFF**



Return the drum to the
upright position



**ENSURE THE TAP IS IN
THE OFF POSITION**

"Let's Get it Right"



Marketed by:
Majac Medical Products Pty. Ltd.
Tel: 1300 138 578 • Int: +61 7 3265 6355
Email: sales@majacmedical.com.au
www.majacmedical.com.au



FS 540017



**Drum
spanners
are
available
on request**