

Oxivir® Excel Wipes

Cleaning and disinfectant wipe for non-critical medical devices and surfaces (low level instrument grade disinfectant)

Oxivir Excel Wipes are a one step cleaning and disinfectant wipe for non-critical medical devices and surfaces (low level instrument grade disinfectant).

Based on proprietary AHP® technology to deliver fast, effective cleaning and disinfection.

Fast-acting

- · Bactericidal & Virucidal with 1 minute
- Kills SARS-CoV-2 (COVID-19 virus) in 15 seconds

Effective

- · Kills viruses, bacteria and yeast in 1 minute
- Virus: Herpes Simplex Type 1, Murine hepatitis virus, Adenovirus Type 5, Poliovirus, Norovirus, H1N1, Rotavirus, and Rhinovirus
- Bacteria: MRSA, VRE, CRE, S. aureus, E. hirae, P. aeruginosa, E. coli, P. vulgaris
- · Yeast: Candida
- · Virucidal, Bactericidal
- Proven to kill bacteria including Staphylococcus aureus and Pseudomonas aeruginosa
- · Effective against viruses including Norovirus

Easy to use

Ready-to-use pre-wetted wipes in a convenient soft pack

Environmentally Responsible

Active ingredient hydrogen peroxide, degrades into water and oxygen

Designed for

- Hospitals
- · Aged Care facilities
- · Child Care and Educational facilities
- Hospitality





ARTG No.: 396564



Oxivir® Excel Wipes

Cleaning and disinfectant wipe for non-critical medical devices and surfaces (low level instrument grade disinfectant)

Efficacy Data:

Oxivir Excel Wipes are proven to kill Gram-positive and Gram-negative bacteria, enveloped and non- enveloped viruses, fungi and yeast.

Microorganism	Test Method (dirty conditions)	Organism	Contact Time
Bacteria / *Yeast	EN13727:2013	Staphylococcus aureus ATCC 6538 Enterococcus hirae ATCC 10541 Pseudomonas aeruginosa ATCC 15442	30 secs 30 secs 30 secs
	EN 16615:2015	S. aureus MRSA E. faecalis VRE K. pneumoniae CRE Staphylococcus aureus ATCC 6538 Enterococcus hirae ATCC 10541 Pseudomonas aeruginosa ATCC 15442 Candida albicans ATCC 10231*	1 min 1 min 1 min 1 min 1 min 1 min 1 min
	TGA Option B Test	Escherichia coli NCTC 8196 Proteus vulgaris NCTC 4635 Pseudomonas aeruginosa ATCC 19582 Staphylococcus aureus NCTC 4163	10 mins 10 mins 10 mins 10 mins
Viruses	ASTM E1053	Herpes Simplex Type 1 ATCC/VR-733 Murine hepatitis virus (MHV) -1 ATCC/VR-261	1 min 1 min
	EN 14476:2013	Adenovirus Type 5 Poliovirus Type 1 Murine Norovirus (as surrogate of human norovirus) COVID-19 (SARS-CoV-2) Influenza A virus H1N1 (swine) Human Rotavirus Strain Wa Human Rhinovirus Type 14	30 secs 30 secs 30 secs 15 secs 1 min 1 min 1 min
Fungi	EN 16615:2015	Aspergillus brasiliensis	15 mins



Oxivir® Excel Wipes



Use instructions

All surfaces must remain visibly wet for 1 minute (contact time*).

*Contact time is the time the disinfectant needs to stay wet on a surface to ensure effectiveness.

Do not flush wipe in a toilet. Dispose of contents/container/wipe in accordance with local waste management guidelines. Do not mix with other detergents or other chemicals. Not recommended for use on copper, brass and marble. Always follow medical device manufacturer's recommendations. Not intended for use on any invasive surgical instruments.

Technical data			
Appearance	Pre-wetted wipes	Shelf Life	2 years
Odour	Characteristic odour		
Active Ingredient	Hydrogen peroxide 0.36%w/w		

The above data is typical of normal production and should not be taken as a specification.

Product	Pack size	Article code	Country
Oxivir Excel Wipes	24x100 wipes	70024385	ANZ

Accelerated Hydrogen Peroxide® and Design are registered trade marks of Diversey Inc. All rights reserved.

Safe handling

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet.

Storage information

Store in original container. Avoid extremes of temperature and humidity. Do not mix with other detergents or other chemicals. Store below 25°C.

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available online at www.diversey.com or by calling Toll free: Australia, 1800 647 779 or New Zealand, 0800 803 615.