



Professional Dental Manufacturer Since 2003

Maintenance

- 1. Keep the Pulp-Tester clean and store it somewhere dry and clean when the instrument is left unused for a long time.
- 2. Do not take part the instrument at radom. Please contact the manufacturers or the local dealers in case quality problem is discovered.
- 3. Parts like stainless electrode and test probe which have direct contact with the patient should be disinfected by autoclave each time after used. The crust of the instrument and the wire should be disinfected by the ethanol(75%) each month.



CO-PULPTESTER