

# points

DISPOSABLE BRUSH APPLICATORS

EINWEG PINSELAPPLIKATOR

BRUSH-APLICADORES DESCARTÁVEIS

MICRO-PINCELES APLICADORES DESECHABLES

PINCEAUX-APPLICATEURS JETABLES

APPLICATORI MONOUSO

DISPOSABLE BORSTEL APPLICATOREN

ENGANGS MIKROBØRSTER

APPLISERINGS BØRSTER

KERTAKÄYTTÖINEN ANNOSTELUTIKKU

ΜΙΑΣ ΧΡΗΣΗΣ ΒΟΥΡΤΣΑΚΙΑ ΕΠΑΛΕΙΨΗΣ

JEDNORAZOWE APLIKATORY

ELDOBHATÓ APPLICATOR PAMACS

ÜHEKORDSED APLIKATSIOONIHARJAKESED

ŠČETKASTI APLIKATORJI ZA ENKRATNO UPORABO

VIENREIZ LIETOJAMIE APLIKATORI



VIENKARINIS ŠEPETĒLIS -APLIKATORIUS

JEDNORÁZOVÉ APLIKAČNÍ ŠTĚTEČKY

JEDNORAZOVÉ APLIKAČNÉ KEFKY

ポイント ディスポーザブル・ブラシアプリケーター

拋棄式塗刷

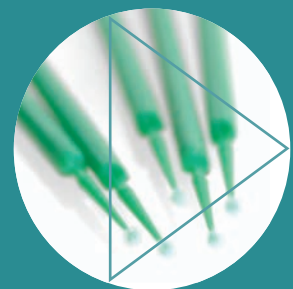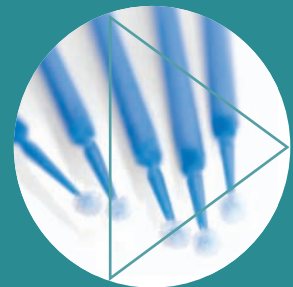

SDI

# points



Points disposable brush applicators are used to quickly transfer small amounts of materials into difficult to reach areas. Points are resistant to common solvents in dental materials

#### Solvent resistant fibers

The fiber tips are resistant to harsh environments and solvents.

#### Non-absorbent, non-linting fibers

The fibers will not dislodge during the procedure. The solution is suspended in the fibers until applied.

#### Three sizes

The three sizes of Points, superfine, fine and medium, enable one to pick up different drop sizes.

#### Choice of colors

The various colors are ideal for color coding multistep procedures.

#### Soft flexible material


Points can be bent for maximum accessibility.

#### Indications

- Sealant application
- Placing bonding agents
- Applying etchants
- Placing cavity liners
- Styptic and disclosing solutions
- Mop up excess resin
- Condensing calcium hydroxide in root canals

#### instructions:

- 1 Points can be used either straight or bent

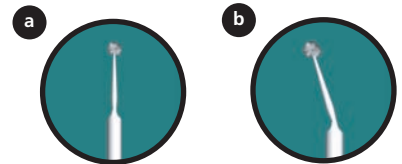

- 2 Dip the Point into the solution and place where required

- 3 Dispose of the Point after the application

**Suggestion:** When applying more than one solution, use a differently colored Point to avoid confusion

**SDI**

Made in Australia by SDI Limited  
Bayswater, Victoria 3153  
Australia 1 800 337 003  
Austria 00800 022 55 734  
Brazil 0800 770 1735  
France 00800 022 55 734  
Germany 0800 100 5759  
Ireland 01 886 9577  
Italy 800 780625  
New Zealand 0800 734 034  
Spain 00800 022 55 734  
United Kingdom 00800 022 55 734  
USA & Canada 1 800 228 5166  
[www.sdi.com.au](http://www.sdi.com.au)

#### Points purple - superfine

1 x 400 Points purple - superfine  
Reorder 8100122

#### Points dark green - fine

1 x 400 Points dark green - fine  
Reorder 8100116

#### Points dark blue - medium

1 x 400 Points dark blue - medium  
Reorder 8100118

#### Points Dispenser

Reorder 8100131

#### Points assortment - all sizes

1 x 100 Points purple - superfine  
1 x 100 Points dark green - fine  
1 x 100 Points light green - fine  
1 x 100 Points dark blue - medium  
Reorder 8100123

#### Points light green - fine

1 x 400 Points light green - fine  
Reorder 8100117

#### Points light blue - medium

1 x 400 Points light blue - medium  
Reorder 8100119



**CE**



M810301 C