

Instructions for use

SONICflex 2003 - REF 1.000.4246

SONICflex LUX 2003 L - REF 1.000.4243



KaVo. Dental Excellence.

Distributed by:

KaVo Dental GmbH

Bismarckring 39

D-88400 Biberach

Tel. +49 7351 56-0

Fax +49 7351 56-1488

Manufacturer:

Kaltenbach & Voigt GmbH

Bismarckring 39

D-88400 Biberach

www.kavo.com



Contents

1	User instructions	5
1.1	Warranty terms and conditions	7
2	Safety	9
2.1	Description of safety instructions	9
2.1.1	Description of safety instructions: Warning symbol	9
2.1.2	Description of safety instructions: Structure	9
2.1.3	Description of safety instructions: Description of danger levels	10
2.2	Purpose – Proper use	11
2.3	Safety instructions	13
2.3.1	Safety instructions: SONICflex tips	16
3	Product description	19
3.1	Technical data	20

3.2	Transport and storage conditions	22
4	First use	24
4.1	Connecting to devices	26
4.2	Attaching the MULTiflex and MULTiflex LUX coupling	27
4.3	Check O-rings	27
4.4	Check the pressure	28
5	Operation	30
5.1	Attaching SONICflex	30
5.2	Removing SONICflex	32
5.3	Inserting SONICflex tip	33
5.4	Removing SONICflex tip	35
5.5	Power setting	36
5.6	Regulating spray	36
5.7	Check the amount of water	37

6	Preparation methods according to ISO 17664	38
6.1	Preparations at the site of use	38
6.2	Cleaning	39
6.2.1	Cleaning: Manual cleaning - external	40
6.2.2	Cleaning: Automated external cleaning	41
6.2.3	Cleaning: Manual cleaning - internal	42
6.2.4	Cleaning: Automated internal cleaning	44
6.3	Disinfection	45
6.3.1	Disinfection: Manual disinfection - external	46
6.3.2	Disinfection: Manual disinfection - internal	47
6.3.3	Disinfection: Machine disinfection - external and internal	48
6.4	Drying	49
6.5	Care products and systems - Servicing	50
6.5.1	Care products and systems - Servicing: Torque wrench care	52

6.5.2	Care products and systems - Servicing: Care with KAVOspray	54
6.5.3	Care products and systems - Servicing: Care of the KaVo SPRAYrotor	55
6.5.4	Care products and systems - Servicing: Care with KaVo QUATTROcare	56
6.6	Packaging	57
6.7	Sterilisation	58
6.8	Storage	60
7	Tools	62



1 User instructions





Dear User

Congratulations on purchasing this KaVo quality product. By following the instructions below you will be able to work smoothly, economically and safely.

© Copyright by KaVo Dental GmbH

Symbols

	Refer to Safety/Warning symbol chapter
	Important information for users and service technicians

	Action request
	CE mark (Communauté Européenne). A product bearing this mark meets the requirements of the applicable EC directive.
	Can be steam-sterilised at 134 °C -1 °C / +4 °C (273 °F -1.6 °F / +7.4 °F)
	Thermodisinfected

Target group

This document is intended for dentists and their assistants. The section on starting up is also intended for service technicians.

1.1 Warranty terms and conditions

The following warranty conditions apply to this KaVo medical device:

KaVo provides the end customer with a warranty of proper function and guarantees zero defects in respect of material and processing for a period of 12 months from date of invoice, subject to the following conditions: In case of justified complaints, KaVo will honour its warranty with a free replacement or repair. Other claims of any nature whatsoever, in particular with respect to compensation, are excluded. In the event of default, gross negligence or intent, this shall only apply in the absence of mandatory legal regulations to the contrary.

KaVo cannot be held liable for defects and their consequences that have arisen or may arise from natural wear, improper handling, cleaning or maintenance, non-compliance with operating, maintenance or connection instructions, calcination or corrosion, contaminated air or water supplies or chemical or electrical factors deemed abnormal or impermissible in ac-

cordance with KaVo's instructions for use or other manufacturer's instructions.

No liability is assumed when defects or their consequences are derived from manipulations or changes to the product by the customer or a third party.

Service warranty claims will only be accepted if the product is submitted along with proof of purchase in the form of a copy of the invoice/delivery note. The dealer, purchase date, type and serial number must be clearly visible on this document.

2 Safety

2.1.1 Description of safety instructions: Warning symbol



Warning symbol

2.1.2 Description of safety instructions: Structure



 **DANGER**

The introduction describes the type and source of the hazard.

This section describes the potential consequences of non-observance.

- ▶ The optional step includes necessary measures for hazard prevention.

2.1.3 Description of safety instructions: Description of danger levels

The safety instructions listed here, together with the three levels of danger will help avert property damage and injury.



CAUTION

CAUTION

indicates a hazardous situation that can cause damage to property or mild to moderate injuries.



WARNING

WARNING

indicates a hazardous situation that can cause death or serious injury.

**⚠ DANGER****DANGER**

indicates a hazardous situation that can directly cause death or serious injury.

2.2 Purpose – Proper use

This medical device is

- Only intended for dental treatment. Any other type of use or alteration to the product is impermissible and can be hazardous.
The SONICflex can be used in conjunction with KaVo tips for plaque removal, prophylaxis, endodontics, periodontology, surgery and conservative dentistry.
- A medical device according to relevant national statutory regulations.

According to these regulations, this medical device may only be used for the described application by a knowledgeable user. The following must be observed:

- the applicable health and safety regulations
- the applicable accident prevention regulations
- these Instructions for use

According to these regulations, the user is required to:

- only use equipment that is operating correctly,
- use the equipment for the proper purpose,
- protect him or herself, the patient and third parties from danger, and
- avoid contamination from the product..

2.3 Safety instructions



WARNING

Hazard to the care provider and patient

Stop working in case of damage, irregular noise during operation, excessive vibration, unusual build-up of heat or if the tip cannot be firmly held.

- ▶ The instrument must be cleaned, serviced and stored dry if it has not been used for a long period.

**⚠ CAUTION****Injury or damage due to wear.**

Irregular operating noise, excessive or very low vibrations, or if the instrument tip becomes loose.

- ▶ Stop work and seek service support.

**⚠ CAUTION****Premature wear and malfunctioning from improper storage during long periods of nonuse.**

Reduced product life.

- ▶ The medical device should be cleaned, serviced and stored in a dry location, according to instructions, before long periods of nonuse.

The following individuals are authorized to repair and service KaVo products:

- Technicians at KaVo branches throughout the world
- Technicians specially trained by KaVo

**Note**

When the SONICflex is in the holder, the torque wrench should be placed on the tip to protect against injury.

To ensure proper function, the medical device must be set up according to the reprocessing methods described in the KaVo Instructions for Use, and the care products and care systems described therein must be used. KaVo recommends specifying a service interval at the dental office for a licensed shop to clean, service and check the functioning of the medical device. This service interval depends on the frequency of use and should be adjusted accordingly.

Service may only be carried out by KaVo-trained repair shops using original KaVo replacement parts.

2.3.1 Safety instructions: SONICflex tips



CAUTION


Risk of injury and infection when changing the SONICflex tips.
This can substantially endanger the user.

- ▶ To test, use and remove the SONICflex tips, use a glove or other finger protection.



Note

We recommend replacing the SONICflex tips every 9-12 months.

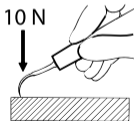

Note

Regularly check the wear of the tip with the tip test card (**Mat. no. 1.001.6958**).

The tips may break from continuous stress or damage (such as being dropped on the floor or being bent). Therefore, check the tip to see if it is safe before each use by applying a slight amount of pressure with your thumb or index finger.

-
- ▶ For an additional safety check, mechanically apply approx. 10 N (1 kg) to the tip without operating the device.

3 Product description



SONICflex 2003, (Mat. no. 1.000.4246)



SONICflex LUX 2003 L (Mat. no. 1.000.4243)

The SONICflex is a dental handpiece that complies with ISO 15606. The vibration is generated by a rotating steel sleeve. In connection with the different KaVo tips, an oscillating elliptical tip movement is generated for the respective application. The internal water cooling (spray cooling) prevents the treatment field from heating up, and keeps the treatment surface clean.



Note

The device should be operated with the described operating pressure. After starting, the intensity can be regulated using the foot switch at maximum drive pressure.

3.1 Technical data

Drive pressure	2.5 (29) to 4.2 (44) bar (psi)
Air consumption	20 – 40 NL/min

Water use	30 - 50 ml/min
Frequency	6 – 6.5 kHz
Recommended pressure	0.1 - 2 N

The SONICflex can be mounted on all MULTIflex couplings.

- Oscillation level 1 = 120 +/- 15 μm
- Oscillation level 2 = 160 +/- 15 μm
- Oscillation level 3 = 240 +/- 15 μm



⚠ CAUTION

Observe the SONICflex recommended setting.

Non-compliance with the recommendations can endanger the patient.

- ▶ When using level 3, the recommended settings must be observed.

3.2 Transport and storage conditions



CAUTION

It is hazardous to start up the medical device after it has been stored strongly refrigerated.

This can cause the medical device to malfunction.



- ▶ Prior to start-up, very cold products must be heated to a temperature of 20°C to 25°C (68°F to 77°F).



Temperature: -20°C to +70°C (-4°F to +158°F)



Relative humidity 5% RH to 95% RH

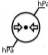 A circular icon representing a pressure gauge. It features a central dot with two horizontal arrows pointing outwards. The label 'hPa' is positioned at the top right and bottom left of the circle, with thin lines connecting them to the gauge's perimeter.	Air pressure: 700 hPa to 1060 hPa (10 psi to 15 psi)
 An icon of an open umbrella with several raindrops falling from its canopy.	Protect from moisture.

4 First use



WARNING

Hazard from non-sterile products.

Infection danger to the care provider and patient.

- ▶ Before first use and after each use, sterilise the medical device.

The SONICflex can be used in conjunction with KaVo tips for plaque removal, prophylaxis, endodontics, periodontology, surgery and conservative dentistry.

The amount of water needs to be set on the dental unit so that the instrument tip sprays the water with the proper oscillating intensity. It is important to remove all the plaque to ensure satisfactory oral hygiene and thorough periodontological treatment. The vibration cleaning of the SONICflex is gentle, fast and easy to use. Using the neighbouring tooth for support

makes the technique easier and offers easier guidance. The instrument must be guided back-and-forth easily, gently and quickly. The instrument is placed on the side of the tooth and guided parallel to the tooth. The instrument should be moved parallel to the surface of the tooth and not the edge to prevent forming notches in the tooth substance. Professionals recommend polishing the tooth surface with the KaVo prophylaxis head using the provided rubber cup and a fine paste to improve the effect of caries prophylaxis.

4.1 Connecting to devices




CAUTION

Damage from soiled and moist cooling air.

Contaminated and moist cooling air can cause malfunctions and lead to premature bearing wear.

- ▶ Make sure that the supply of cooling air is dry, clean and uncontaminated according to ISO 7494-2.

4.2 Attaching the MULTiflex and MULTiflex LUX coupling

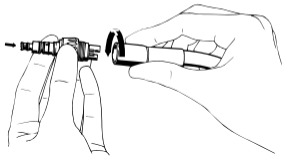

- ▶ Screw the MULTiflex LUX/MULTiflex LED coupling onto the turbine hose and tighten with the wrench.

4.3 Check O-rings




CAUTION

Missing or damaged O-rings.
Malfunctions and premature failure.

- ▶ Make sure that all O-rings are on the coupling and undamaged.

Number of available O-rings: 5

4.4 Check the pressure

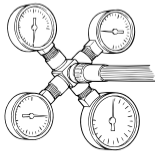

A minimum drive pressure of 2.5 bar (36 psi) is required for operating the SONICflex. A higher drive pressure will be reduced automatically inside the SONICflex. The air consumption is approximately 20-40 NI/min. Insert the test manometer (**Mat. no. 0.411.8731**) between the MULTIflex coupling and the SONICflex.

Pressure displayed:

- Drive air T.R. 2.5 to 4.2 bar (36 to 61 psi)
- Return air R.L. < 0.4 bar (6 psi)
- Spray air Sp.L. = max. 2 (29) bar (psi)
Spray air not required, however.
- - Water W.: 1.0 to 2.0 bar (15 to 29 psi)

5 Operation

5.1 Attaching SONICflex



CAUTION

Ensure that the SONICflex is tightly seated on the coupling.

If the SONICflex unintentionally comes off the coupling during treatment, it can endanger the patient and user.


- ▶ Before each treatment, pull on the SONICflex to see if it is securely seated in the coupling.

**⚠ CAUTION**

Imprecise coupling especially during the afterglow period.


Imprecise coupling can destroy the high-pressure bulb of a MULTiflex (LUX) coupling or reduce its life.

- ▶ Make sure that the coupling is precise.

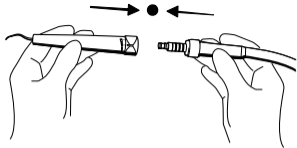

- ▶ Place the SONICflex precisely on the MULTiflex/MULTiflex LUX coupling and push it to the rear until it locks audibly.

5.2 Removing SONICflex

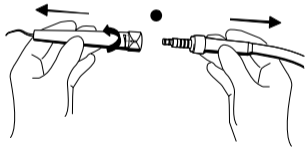

- ▶ Hold the MULTIflex / MULTIflex LUX coupling tightly, and pull the SONICflex forward while twisting slightly.

5.3 Inserting SONICflex tip



CAUTION

Hazard from a tip which has been incorrectly inserted in the torque wrench.


This may result in a risk of injury for the user.

- ▶ When inserting the tip into the torque spanner, make sure that the tip's end always faces into the opening of the torque spanner.



Note

In view of the substantial liability risks, we recommend using exclusively original KaVo SONICflex tips.

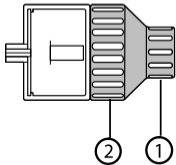

- ▶ Insert the desired tip with the tip end pointing down into the torque wrench and screw it into the handpiece by turning it clockwise.


The torque wrenches are for changing the working tips of the SONICflex and protect against injury. The torque wrench can be screwed in more quickly by holding it at the rear, thin grip area ①. The large diameter ② helps tighten and remove it.

Note

The tip is properly gripped when the torque wrench slips.


**Note**

When the SONICflex is in the holder, the torque wrench should be placed on the tip to protect against injury.

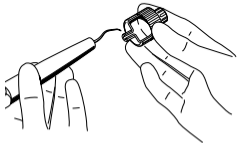5.4 Removing SONICflex tip


- ▶ Insert/attach the torque wrench in/on the SONICflex and unscrew the tip anticlockwise.

5.5 Power setting

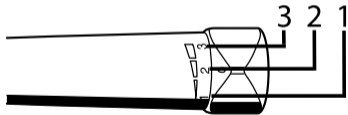

Use the control ring of the SONICflex to set performance levels 1, 2, and 3.

5.6 Regulating spray

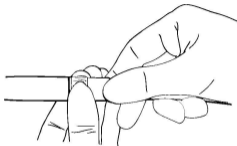

- ▶ Turn the spray ring on the MULITflex (LUX)/ MULTIflex LUX coupling in order to regulate the water content. The water volume can be regulated by selecting different stop positions.

Turn clockwise to reduce the water volume

and anticlockwise to increase the volume.

5.7 Check the amount of water



CAUTION

Overheating of the tooth due to lack of cooling water.

Thermal damage to the dental pulp.

- ▶ Adjust the water amount for the spray cooling to a minimum of 30 cm³/min!

6 Preparation methods according to ISO 17664



Note

The following preparation procedure applies for the SONICflex instrument, torque wrench and nozzle needle.

6.1 Preparations at the site of use



WARNING

Hazard from non-sterile products.

There is a risk of infection from contaminated medical devices.

- ▶ Observe suitable personal protective measures.

- ▶ Remove all residual cement, composite or blood without delay.

- ▶ The medical device must be dry when transporting it to the site of re-processing.
- ▶ Do not place it in a solution or the like.
- ▶ Reprocess the medical device as soon as possible after treatment.
- ▶ Remove the tip from the SONICflex using the torque wrench.

6.2 Cleaning



CAUTION


Malfunctions from cleaning in an ultrasonic unit.
Defects in the product.

- ▶ Only clean manually or in a thermodisinfector.

6.2.1 Cleaning: Manual cleaning - external

Accessories required:

- Tap water $30\text{ °C} \pm 5\text{ °C}$ ($86\text{ °F} \pm 10\text{ °F}$)
- Brush, e.g. medium-hard toothbrush

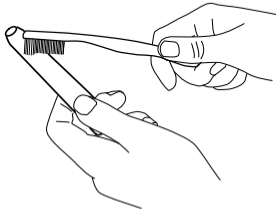

- ▶ Brush it off under running tap water using for example a medium-hard toothbrush.

- ▶ Prepare the tip according to the manufacturer's instructions for use.



6.2.2 Cleaning: Automated external cleaning

KaVo recommends thermodisinfectors in accordance with EN ISO 15883 that are operated with alkaline cleaning agents at a pH of max. 10 (e.g. Miele G 7781 / G 7881 – validation was performed with the "VARIO-TD" programme, "neodisher® mediclean" cleaning agent, "neodisher® Z" neutralisation agent, and "neodisher® mielclear" rinsing agent and extends only to the compatibility of materials with respect to KaVo products).

- ▶ For program settings as well as cleansers and disinfectants to be used, please refer to the Instructions for Use of the thermodisinfectant (complying with max. pH value of 10).
- ▶ In order to prevent negative effects on the medical device, make sure that the interior and the exterior of the medical device are dry, and then lubricate immediately with care agents from the KaVo care system.

6.2.3 Cleaning: Manual cleaning - internal

Can only be done with KaVo CLEANspray or KaVo DRYspray.

- ▶ Cover the medical device with the KaVo CLEANpac bag, and place it on the corresponding care adapter. Press the spray button three times for 2 seconds each time. Remove the medical device from the spray attachment and let the cleaner work for one minute.
- ▶ Afterwards, rinse for 3-5 seconds with KaVo DRYspray.

See also: KaVo CLEANspray / KaVo DRYspray Instructions for Use

**Note**

KaVo CLEANspray and KaVo DRYspray for manual interior cleaning are only available in the following countries:

Germany, Austria, Switzerland, Italy, Spain, Portugal, France, Luxembourg, Belgium, Netherlands, United Kingdom, Denmark, Sweden, Finland and Norway.

In other countries, only automated interior cleaning with thermodisinfectors in accordance with ISO 15883-1.

**Note**

Do not place the tips in the drill bit bath, as the fine capillaries would corrode badly, making it impossible to rinse them under running water.



6.2.4 Cleaning: Automated internal cleaning

KaVo recommends thermodisinfectors in accordance with EN ISO 15883 that are operated with alkaline cleaning agents at a pH of max. 10 (e.g. Miele G 7781 / G 7881 – validation was performed with the "VARIO-TD" programme, "neodisher® mediclean" cleaning agent, "neodisher® Z" neutralisation agent, and "neodisher® mielclear" rinsing agent and extends only to the compatibility of materials with respect to KaVo products).

- ▶ For program settings as well as cleansers and disinfectants to be used, please refer to the Instructions for Use of the thermodisinfectant (complying with max. pH value of 10).
- ▶ In order to prevent negative effects on the medical device, make sure that the interior and the exterior of the medical device are dry, and then lubricate immediately with care agents from the KaVo care system.

6.3 Disinfection




CAUTION

Malfunctioning from using a disinfectant bath or disinfectant containing chlorine.

Defects in the product.

- ▶ Only disinfect in a thermodisinfector or manually.

6.3.1 Disinfection: Manual disinfection - external

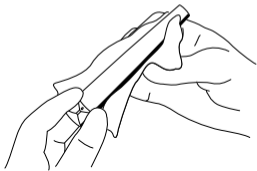

KaVo recommends the following products based on material compatibility. The microbiological efficacy must be ensured by the disinfectant manufacturer.

- Microcide AF from Schülke&Mayr (liquid or cloths)
- FD 322 from Dürr

- CaviCide from Metrex

Tools required:

- Cloths for wiping down the medical device.

- ▶ Spray the disinfectant on a cloth, then thoroughly wipe down the medical device and leave the disinfectant to soak in according to the instructions from the disinfectant manufacturer.

**Note**

Observe the instructions for use for the disinfectant.

6.3.2 Disinfection: Manual disinfection - internal

The efficacy of manual internal disinfection must be demonstrated by the manufacturer of the disinfection agent. With KaVo products, use only disinfection agents that have been released by KaVo with respect to the compatibility of materials (e.g. WL-cid / made by ALPRO).

- ▶ Immediately after internal disinfection, lubricate the KaVo medical device immediately with care agents from the KaVo care system.

6.3.3 Disinfection: Machine disinfection - external and internal



KaVo recommends thermodisinfectors in accordance with EN ISO 15883 that are operated with alkaline cleaning agents at a pH of max. 10 (e.g. Miele G 7781 / G 7881 – validation was performed with the "VARIO-TD" programme, "neodisher® mediclean" cleaning agent, "neodisher® Z" neutralisation agent, and "neodisher® mielclear" rinsing agent and extends only to the compatibility of materials with respect to KaVo products).

- ▶ For program settings as well as cleansers and disinfectants to be used, please refer to the Instructions for Use of the thermodisinfectant (complying with max. pH value of 10).

- ▶ In order to prevent negative effects on the medical device, make sure that the interior and the exterior of the medical device are dry, and then lubricate immediately with care agents from the KaVo care system.

6.4 Drying

Manual drying

- ▶ Blow off the outside and inside with compressed air until water drops are no longer visible.

Machine drying

The drying procedure is normally part of the disinfection program of the thermodisinfectant.

**Note**

Please follow the instructions for use of the thermodisinfectant (compressed air quality - see the Warning under "Start-up").

6.5 Care products and systems - Servicing**⚠ CAUTION**

**Premature wear and malfunctions from improper servicing and care.
Reduced product life.**

- ▶ Perform proper care regularly!

**Note**

KaVo only guarantees that its products will function properly when the care products used are those listed as accessories, as they were tested for proper use on our products.

**Note**

If you are bothered by some oil leaking during the treatment, the technical set-up allows for switching from servicing with oil prior to sterilisation to once weekly servicing.

6.5.1 Care products and systems - Servicing: Torque wrench care




CAUTION

Malfunctions from cleaning in the ultrasonic unit.


Defects in the torque wrench.

- ▶ Do not place the torque wrench in ultrasonic cleaning devices.

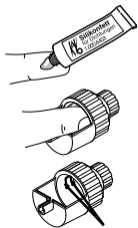

If the torque wrench runs roughly, it should be lubricated with silicone grease (**Mat. no. 1.000.6403**). The silicone grease is pressed into the torque wrench in the slots or grease pockets of the locking springs. Then place a small amount of grease on your fingertip, and press it into the torque wrench at the indicated location (see arrow). Then rotate the torque wrench and regrease if necessary.


6.5.2 Care products and systems - Servicing: Care with KAVOspray

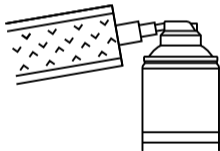

KaVo recommends servicing the product once per week.

- ▶ Remove tip.
- ▶ Cover the product with the Cleanpac bag.
- ▶ Place the product on the cannula, and press the spray button for one second.

6.5.3 Care products and systems - Servicing: Care of the KaVo SPRAYrotor

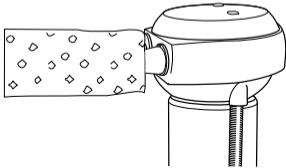


KaVo recommends servicing the product once per week.

- ▶ Remove tip
- ▶ Place the product on the appropriate coupling of the **KaVo SPRAYrotor** and cover it with a Cleanpac bag.
- ▶ Servicing the product.

See also: Instructions for use **KaVo SPRAYrotor**

6.5.4 Care products and systems - Servicing: Care with KaVo QUAT-TROcare

Cleaning and care unit with expansion pressure for effective cleaning and care.

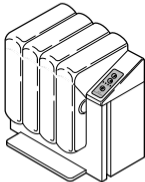

KaVo recommends servicing the product once per week.

- ▶ Remove the tip.

- ▶ Servicing the product.

See also: Instructions for use **KaVo QUATTROcare**

6.6 Packaging



Note

The sterilisation bag must be large enough for the instrument so that the bag is not stretched.

The quality and use of the sterilised product packaging must satisfy applicable standards and be suitable for the sterilisation procedure.

- ▶ Individually weld the medical device in the sterilised item packaging (such as KaVo STERlclave bags **Mat. no. 0.411.9912**)!

6.7 Sterilisation

**Sterilisation in a steam steriliser (autoclave) in accordance with EN 13060/
ISO 17665-1 (e.g. KaVo STERlclave B 2200 / 2200 P)**



CAUTION

**Premature wear and malfunctions from improper servicing and care.
Reduced product life.**

- ▶ Before each sterilisation cycle, service the medical device with KaVo care products. However, if over-oiled, the SONICflex will differ from the guidelines.

**⚠ CAUTION**

Contact corrosion due to moisture.

Damage to product.

- ▶ Immediately remove the product from the steam steriliser after the sterilisation cycle!

**Note**

Remove the tips to be sterilized and dry the medical device.

135 °C



The KaVo medical device has a maximum temperature resistance up to 138 °C (280.4 °F).

(Depending on the available autoclave,) select a suitable procedure from the following sterilisation processes:

- Autoclave with three times initial vacuum:
 - at least 3 minutes at 134 °C -1 °C / +4 °C (273 °F -1.6 °F / +7.4 °F)
- Autoclave using the gravitation method:
 - at least 10 minutes at 134 °C -1 °C / +4 °C (273 °F -1.6 °F / +7.4 °F) or alternatively
 - at least 60 minutes at 121 °C -1 °C / +4 °C (250 °F -1.6 °F / +7.4 °F)

6.8 Storage

Prepared products must be stored, protected from germs (as far as possible) and dust, in a dry, dark, cool room.



Note

Observe the expiration date of the sterilised item.

7 Tools

Obtainable from dental and medical suppliers

Material summary	Mat. no.
Instrument stand 2151	0.411.9501
Cellulose pad 100 units	0.411.9862
Cleanpac 10 units	0.411.9691
Torque wrench	1.000.4887
Coupling	1.006.5966
Open-ended wrench	0.411.0892
Silicone grease	1.000.6403
Nozzle needle	0.410.0911
Insert for SONICflex	0.411.9902

Material summary	Mat. no.
KaVo CLEANspray	1.007.0579
KaVo DRYspray	1.007.0580
KaVo Spray 2112 A	0.411.9640
ROTAspray 2142 A	0.411.7520
QUATTROcare plus Spray 2140 P	1.005.4525

No.	Tip type	Mat. no,
5	Universal scaler	Mat. no. 0.571.5171
6	Crescent scaler	Mat. no. 0.571.5181
7	Perio scaler	Mat. no. 0.571.5191
8	Perio extra long scaler	Mat. no. 0.571.5371
12	Cem	Mat. no. 0.571.5431
-	Cem attachment	Mat. no. 0.571.7142
16	Retro cylinder, left	Mat. no. 0.571.5541
17	Retro cylinder, right	Mat. no. 0.571.5581

No.	Tip type	Mat. no,
20	Retro T-shape left	Mat. no. 0.571.5521
21	Retro T-shape Undercut	Mat. no. 0.571.5561
24	Root planer, button, small left	Mat. no. 0.571.5621
25	Root planer, button, small right	Mat. no. 0.571.5631
26	Root planer, button, small uni- versal	Mat. no. 0.571.5641
27	Rootplaner, button, large perio	Mat. no. 0.571.5651
28	Micro torpedo shape mesial	Mat. no. 0.571.6741
29	Micro torpedo shape distal	Mat. no. 0.571.6731
30	Micro small, hemispherical me- sial	Mat. no. 0.571.6811

No.	Tip type	Mat. no,
31	Micro small, hemispherical distal	Mat. no. 0.571.6801
32	Micro large, hemispherical mesial	Mat. no. 0.571.6831
33	Micro large, hemispherical distal	Mat. no. 0.571.6821
34	Prep CAD-CAM mesial	Mat. no. 1.002.1984
35	Prep CAD-CAM distal	Mat. no. 1.002.1986
42	Cariex D 0.8, D 64	Mat. no. 1.000.7105
43	Cariex D 1.2, D 64	Mat. no. 1.000.7167
45	Seal conical, D 46	Mat. no. 1.000.2568
48	Clean (brush holder)	Mat. no. 0.571.0401
-	Clean brush no. 1 Refill	Mat. no. 1.004.4125
-	Clean brush no. 2 Refill	Mat. no. 1.004.4126
-	Clean brush no. 3 Refill	Mat. no. 1.004.4127
-	Clean brush no. 4 Refill	Mat. no. 1.004.4128

No.	Tip type	Mat. no,
-	Clean brush no. 5 Refill	Mat. no. 1.004.4129
-	Clean brush no. 6 Refill	Mat. no. 1.004.4130
49	Prep gold mesial	Mat. no. 0.571.7212
50	Prep gold distal	Mat. no. 0.571.7222
51	Prep ceram mesial	Mat. no. 0.571.7262
52	Prep ceram distal	Mat. no. 0.571.7272
53	Angle distal	Mat. no. 0.571.7302
54	Angle mesial	Mat. no. 0.571.7292
55	Retro anterior tooth	Mat. no. 0.571.7342
56	Retro finder left	Mat. no. 0.571.7322
57	Retro finder right	Mat. no. 0.571.7332
-	Retro plug left	Mat. no. 0.571.5601
-	Retro plug right	Mat. no. 0.571.5611
58	Bevel mesial	Mat. no. 0.571.7392
59	Bevel distal	Mat. no. 0.571.7382

No.	Tip type	Mat. no,
60	Perio straight	Mat. no. 0.571.7402
61	Period left	Mat. no. 0.571.7412
62	Perio right	Mat. no. 0.571.7422
66	Endo button large 117°	Mat. no. 1.000.5825
67	Endo conical 125°	Mat. no. 1.000.5822
68	Endo conical 112°	Mat. no. 1.000.5823
69	Endo button small 117°	Mat. no. 1.000.5827
70	Endo conical 117°	Mat. no. 1.000.5821
71	Cariex TC 1.0	Mat. no. 1.000.7362
72	Cariex TC 1.4	Mat. no. 1.000.7363
80	Bone rectangular cutting	Mat. no. 1.004.3875
81	Bone sphere large, D 46	Mat. no. 1.004.3876
82	Bone sphere large	Mat. no. 1.004.3877
83	Bone saw sagittal	Mat. no. 1.004.3878

No.	Tip type	Mat. no,
84	Bone saw axial	Mat. no. 1.004.3879
85	Bone elephant foot	Mat. no. 1.006.0645
86	Bone scaler	Mat. no. 1.006.0741
87	Bone saw blade	Mat. no. 1.006.1275
-	Bone saw blade refill	Mat. no. 1.006.1405
-	Implant Set	Mat. no. 1.003.8167
-	Implant Pin Refill	Mat. no. 1.003.8168
-	Swing Set	Mat. no. 1.006.2045
-	Swing 015 Refill	Mat. no. 1.006.2042
-	Swing 020 Refill	Mat. no. 1.006.2043
-	Swing 025 Refill	Mat. no. 1.006.2044



KaVo. Dental Excellence.