Go Green With Envy

Gentle on the skin Gentle on your trays Gentle on the environment





Whip Mix Green Envy[™] tray cleaner is a natural way of removing alginate residues from impression trays.

- Dissolves residue as it interacts, instead of reacts, with soils and surfaces.
- pH Neutral mixture is gentle on skin and biodegradable.
- Does not corrode metal trays.
- The 1 lb. jar provides cleaning for more than 200 trays.

Simply mix Green Envy[™] with water according to the directions, immerse the instruments for 15-30 minutes, then brush or rinse with warm water. The only thing left behind is a fresh, light citrus scent.



pH neu

ral

Mix 2 cap fulls of Green Envy tray cleaner in 2 cups (16 oz.) of lukewarm water.



Immerse instruments for 15-30 minutes.



Rinse or brush off with warm water, and you're done.



Whip Mix

ltem No. 09637 1 lb. (.45 kg) Jar

Call Today at 800-626-5651 to Order.

Connect with us!

Whip Mix Corporation • 361 Farmington Avenue • Louisville, KY 40209 • www.whipmix.com